



## GlucoRx Voyager for Windows Instruction Manual

[Home](#) » [GlucoRx](#) » GlucoRx Voyager for Windows Instruction Manual 

### GlucoRx Voyager for Windows Instruction Manual



#### Contents

- [1 How to import readings from USB Meters to Voyager for Windows](#)
- [2 Documents / Resources](#)
- [3 Related Posts](#)

### How to import readings from USB Meters to Voyager for Windows

1. Navigate to <https://www.glucorx.co.uk/ourservices/software/> and select the Voyager Download button.




A registration form titled "CREATE ACCOUNT" with a light blue background. It contains several input fields: "First Name\*", "Last Name\*", "Select a date" (with a calendar icon), "Email\*", "Password\*", and "Confirm Password\*". There are also radio buttons for "Male" and "Female", with "Female" selected.

Thank you for signing up for GlucoRX. Please click on the following button to verify your email and get started.



Best regards. GlucoRx support team.

2. To login, Input your email address and password

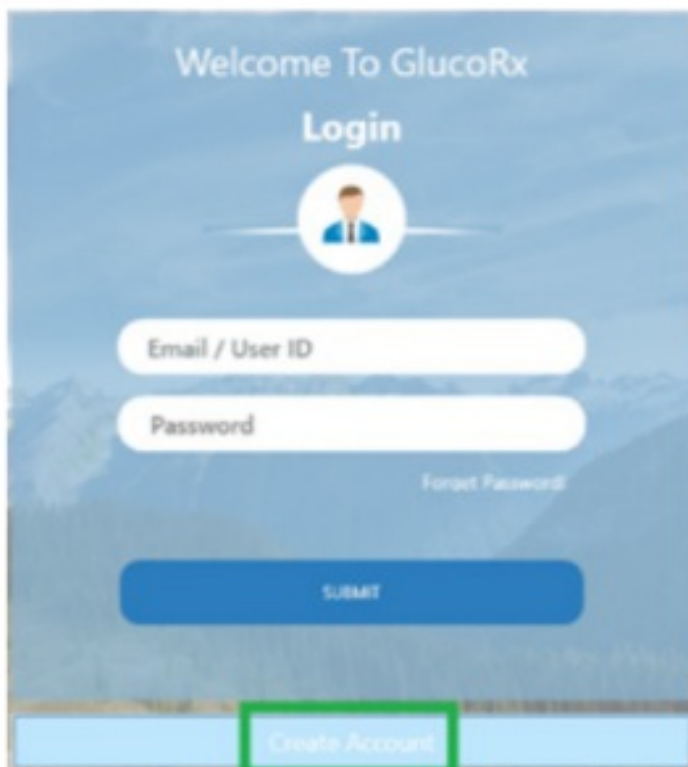

A login form titled "Welcome To GlucoRx Login" with a light blue background. It features a circular profile icon placeholder, input fields for "Email / User ID" and "Password", a "Forgot Password" link, a blue "SUBMIT" button, and a "Create Account" button at the bottom which is highlighted with a green border.

3. The user is prompted to input their Low and high blood glucose limits. This is personal to each person. If you


are unsure what your limits are you should speak with your healthcare professional. (This is only required on the user's first login)

Thresholds	
Units of Measure	mmol
Low Limit	4.1
High Limit	9.5
High Alarm	10.5
Low Alarm	3.5

4. Plug your meter into the computer using a USB cable.
5. When “USB or PC” is displayed on the screen, select the meter name from the devices dropdown and select “Connect”.
6. New readings will automatically be imported.
7. The GlucoRx Voyager portal will open in a web browser.
8. To log into the portal without importing any results, open GlucoRx.exe and click the “visit Portal” button. You can also login from any web browser by going to [www.grxvoyager.com](http://www.grxvoyager.com)



Documents / Resources

	<a href="#">GlucoRx Voyager for Windows</a> [pdf] Instruction Manual Voyager, Voyager, for, windows
---	--

Manuals+.