



IN240701681V01\_GL

712-068V00



IMPORTANT, RETAIN FOR FUTURE REFERENCE: READ CAREFULLY.

## Contents [ [hide](#) ]

[1 712-068V00 Upright Walker Rollator](#)

[2 Product Overview](#)

[3 Assembly Instructions](#)

[4 Operation Instructions](#)

[5 Documents / Resources](#)

[5.1 References](#)

## 712-068V00 Upright Walker Rollator

### Technical Data

Max. user weight	108kg	Handle height(S regulations )	90-102 cm
Total weight approx.	6.9kg	Tray (width x depth)	39.5*27.5 cm
Max. bag load	5kg	Tray height	70.5cm
Length/Overall depth	64.5cm	Front wheels	8inches
Width (open)	56cm	Rear wheels (diameter x width)	8inches
Material of frame	Aluminum	Material of wheels	PVC

### Product Overview



1. Handle
2. Brake handle
3. Parking brake lock
4. Brake cable
5. Height adjustment button
6. Tray
7. Basket
8. Folding button
9. Knob screw
10. Front wheel
11. Rear wheel
12. Knob screw
13. Brake adjustment
14. Velcro tape for brake cable
15. Brake shoe

## **Assembly Instructions**

1. Remove the rollator from the packaging and remove the protective foils.



2. Place the main body on the floor as shown.



3. Remove the knob screws(9) on the front wheel.



4. Insert the front wheel into the frame.



5. Secure the front wheel using the previously loosened knob screw with its black side facing inward.



6. Repeat for the other front wheel. Make sure the black side of the knob screws are on the inside.



7. Flip the product over as shown.



8. Place the rear wheels with the handle as shown.



9. Insert the rear wheel axle into the frame.



10. Please note that the wheels are on the outside.



11. Fasten the rear wheel axle with the previously loosened knob screw (12).



12. Repeat for the other rear wheel.



13. Make sure the rear wheels are on the outside.



14. Put the rollator upright to unfold, using one hand on the handle and another pressing the tray. Ensure it's fully open by listening for a "click" indicating it's locked.



15. Open the red height adjustment button (5) on both sides of the frame.



16. Insert the handles into the frame and close the red height adjustment button (5) so that both sides are at the same height.



17. Our recommendation: Fix the brake cables on both sides of the frame using the velcro tape(14) as shown.



18. Place the tray(6) on the frame.



## Operation Instructions

### 1. Folding

1. Remove the tray (6). Then activate the parking brake lock(3).



2. Fold the basket by pulling the white lever toward your body.



3. The basket is folded as shown.



4. Grasp the rollator by the handle (1) and tilt it in your direction so that it stands on the rear wheels (11).



5. Push the Folding button(8) so that the rollator folds up. If the rollator does not fold automatically, tilt it more.



6. The rollator is folded as shown




## 2. Height Adjustment



Loosen the height adjustment button(S) and adjust the handle to the appropriate height. Repeat for the other side, and Retighten the height adjustment button{S). Make sure that both handles are set to the same height.

The handle height can be adjusted from 90 to 102cm.

 Check that the handle is locked in a secure position.

## 3. Brake System


1. To apply the handbrake pull the brake lever up.



2. Push the brake handle upwards and fix it with the parking brake lock to lock the rear wheels. To release the parking brake, push the brake handle up slightly and pull the parking brake lock forward.

- Brake Adjustment

Step 1: Use one hand to press the iron piece under the spring upward.

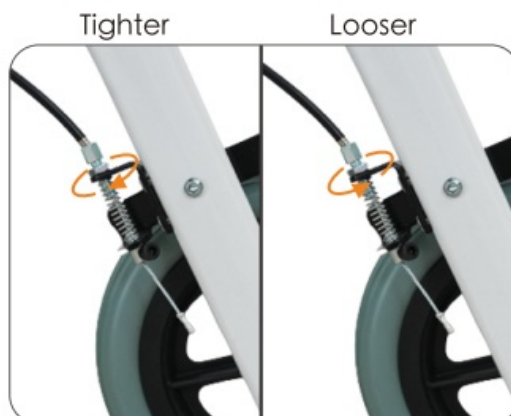

Step 2: Turn the nut clockwise to increase braking force; Turn the nut

counterclockwise to decrease braking force.

If you have any questions, please contact our customer care center, Our contact details are below:

001-s77-644-936e



[customerservice@aosom.com](mailto:customerservice@aosom.com)



Imported by Aosom LLC

27150 SW Kinsman Rd Wilsonville, OR 97070 USA

MADE IN CHINA

If you have any questions, please contact our customer care center, Our contact details are below:



0044-[800-240-4004](tel:800-240-4004)



[endures@mhstar.co.uk](mailto:endures@mhstar.co.uk)

IMPORTER ADDRESS:

MH STAR UK LTD

Unit 27, Perivale Park,

Horsenden lane South

Perivale, UB6 7RH

MADE IN CHINA



## Documents / Resources



[HOMCOM 712-068V00 Upright Walker Rollator \[pdf\]](#) Instruction Manual  
712-068V00 Upright Walker Rollator, 712-068V00, Upright Walker Rollator, Walker Rollator, Rollator

## References

- [User Manual](#)

■ HOMCOM

🔑 712-068V00, 712-068V00 Upright Walker Rollator, HOMCOM, Rollator, Upright Walker Rollator, Walker Rollator

---

## Leave a comment

Your email address will not be published. Required fields are marked \*

Comment \*

Name

Email

Website

☐ Save my name, email, and website in this browser for the next time I comment.

**Post Comment**

**Search:**

**Search**

[Manuals+](#) | [Upload](#) | [Deep Search](#) | [Privacy Policy](#) | [@manuals.plus](#) | [YouTube](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.

