



HUMANTECHNIK Signolux Visual Signal Alert System User Manual

[Home](#) » [HUMANTECHNIK](#) » HUMANTECHNIK Signolux Visual Signal Alert System User Manual 

Contents

- [1 HUMANTECHNIK Signolux Visual Signal Alert System](#)
- [2 Product Information](#)
- [3 Product Usage Instructions](#)
- [4 Contact Information](#)
- [5 Safety instructions](#)
- [6 Functioning principle](#)
- [7 Setup](#)
- [8 Event identification](#)
- [9 ABOUT COMPANY](#)
- [10 Documents / Resources](#)
 - [10.1 References](#)
- [11 Related Posts](#)

HUMANTECHNIK

HUMANTECHNIK Signolux Visual Signal Alert System



Product Information

- **Product Name:** Visual Signal System
- **Languages:** DE, EN, FR, NL, IT, ES
- **Components:** Vibrating pad

Product Usage Instructions

1. Read the user manual carefully and completely before setting up and using the product.
2. Keep the user manual in a safe place for easy access by other users.
3. **Setup:**
 - Connect the vibrating pad to the designated port.
 - Ensure that the LED lights corresponding to different events are active:
 - Doorbell 1: Green LED is active
 - Doorbell 2: Green LED is active
 - Telephone: Orange LED is active
 - Person call: Yellow LED is active
 - Babycry: White LED is active
 - Fire alarm: Red LED is active
 - Water alarm: Blue LED is active
 - Alarm: Red LED is active
 - All values are in seconds (e.g., 1.5 = 1.5 seconds).

Contact Information

- **Germany:**
 - **Tel.:** +49 (0) 76 21/ 9 56 89-0,

- **Fax:** +49 (0) 76 21/ 9 56 89-70,
- **E-mail:** info@humanteknik.com
- **Switzerland:**
 - **Tel.:** +41 (0) 61/ 6 93 22 60,
 - **Fax:** +41 (0) 61/ 6 93 22 61,
 - **E-mail:** info@humanteknik.com
- **France:**
 - **Tel.:** +33 (0) 3 89 44 14 00,
 - **Fax:** +33 (0) 3 89 44 62 13,
 - **E-mail:** sms@audiofr.com
- **Great Britain:**
 - **Tel.:** +44 (0) 1642 247 789,
 - **Fax:** +44 (0) 1642 230 827,
 - **E-mail:** mail@sarabec.co.uk

Safety instructions

Prior to putting the device into operation and using it, kindly read this manual thoroughly and in full. Keep this user manual in a safe place so that it is also accessible for other users at any time.

Functioning principle


This »signolux« vibrating pad has no batteries and can only be used together with the »signolux«-receivers, with exception of the standard receiver A-2614-0 and the plug-in receiver A-2644-0/W! The vibrating pad informs you about incoming event alerts through vibration.

Setup

Connect the vibrating pad with the »AUX« socket of the respective receiver. Trigger an alert to test the function (e.g. press the door- bell button).

Please note:

This vibrating pad works only with signolux-receivers! It is not compatible with with the standard receiver A-2614-0, the flash lamp A-2644-0/W or devices of the »lisa« system!

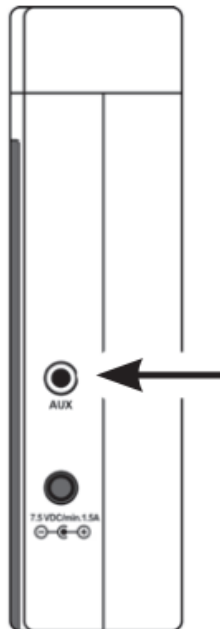

Event identification


- **Doorbell 1:** Doorbell icon 1 (green LED) is active



- **Doorbell 2:** Doorbell icon 2 (green LED) is active



- **Telephone:** Telephone icon (orange LED) is active

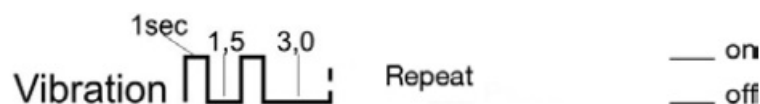


- **Person call:** Person icon (yellow LED) is active



- **Babycry:** Baby icon (white LED) is active



- **Fire alarm:** Fire icon (red LED) is active

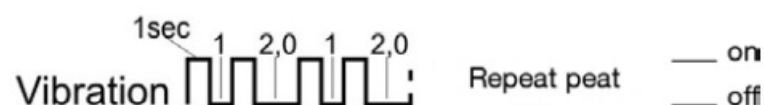

- **Water alarm:** Water icon (blue LED) is active



- **Alarm:** Alarm icon (red LED) is active



- **All values are in seconds:** e.g.: 1,5 = 1,5 seconds

ABOUT COMPANY

- **Germany:**
 - **Tel.:** +49 (0) 76 21/ 9 56 89-0,
 - **Fax:** +49 (0) 76 21/ 9 56 89-70,
 - **E-mail:** info@humantechnik.com
- **Switzerland:**
 - **Tel.:** +41 (0) 61/ 6 93 22 60,
 - **Fax:** +41 (0) 61/ 6 93 22 61,
 - **E-mail:** info@humantechnik.com
- **France:**
 - **Tel.:** +33 (0) 3 89 44 14 00,
 - **Fax:** +33 (0) 3 89 44 62 13,
 - **E-mail:** sms@audiofr.com
- **Great Britain:**
 - **Tel.:** +44 (0) 1642 247 789,
 - **Fax:** +44 (0) 1642 230 827,
 - **E-mail:** mail@sarabec.co.uk

Documents / Resources

	HUMANTECHNIK Signalux Visual Signal Alert System [pdf] User Manual Signalux Visual Signal Alert System, Signalux, Visual Signal Alert System, Signal Alert System, Alert System
--	--

References

- [User Manual](#)

