

[Skip to content](#)

## **Manuals+**

User Manuals Simplified.



# **Inogen One G3 Replacement Columns Instructions**

[Home](#) » [inogen](#) » Inogen One G3 Replacement Columns Instructions

Contents [hide](#)

[1 Inogen One G3 Replacement Columns](#)

[2 Your Equipment Provider](#)

[3 Documents / Resources](#)

[3.1 References](#)

[4 Related Posts](#)



## **Inogen One G3 Replacement Columns**



**To change the columns on your Inogen One G4, first:**

1. Turn off the Inogen One® G4 concentrator by pressing the power button to shut down the device.
2. Remove the Inogen One® G4 concentrator from the carrying case (if attached).




3. Remove the battery from the Inogen One® G4 concentrator.

4. Place the Inogen One® G4 concentrator on its side so that the underside is visible. The metal column assembly




can be seen on one side of the device.


5. Unlock the column assembly by pushing the latch button away from the columns.

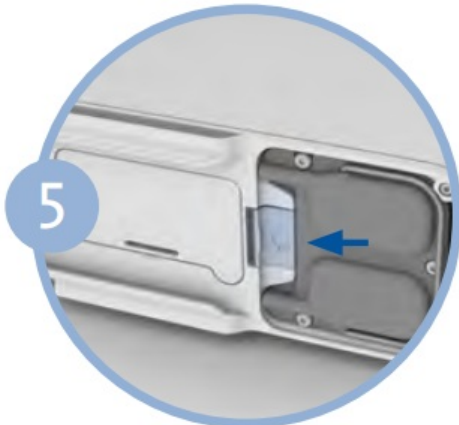


6. While holding the button open, slide the column assembly out of the device by pulling on the column pull handle.

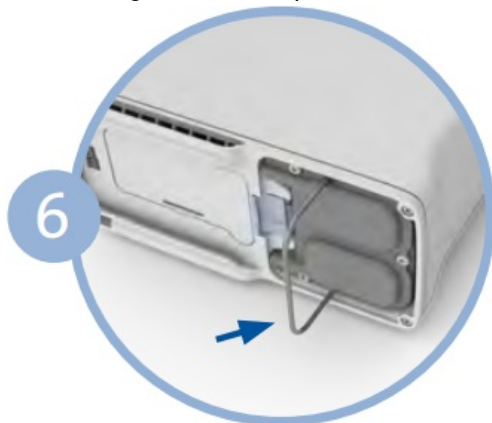

7. Remove the columns completely from the Inogen One® G4. Both columns are removed as one piece.

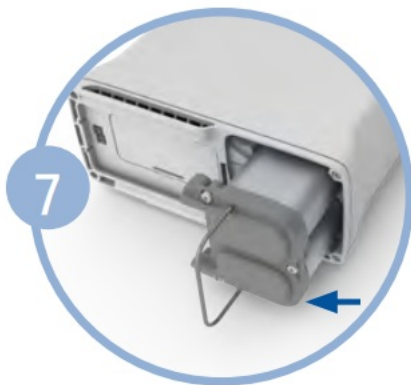

8. Remove dust caps of new coulumn assembly. Make sure there is no dust or debris where the dust caps were

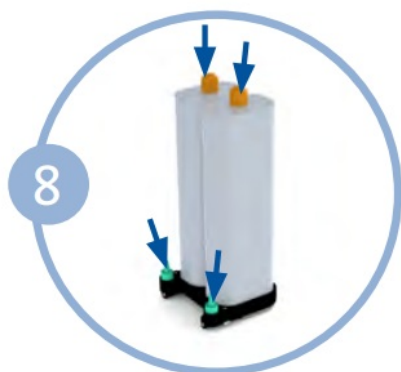

located.

9. Insert column assembly into the Inogen One® G4 concentrator. Do not leave the column ends exposed; column assembly should be inserted into the Inogen One G4 as soon as the dust caps have been removed.



10. Push the column assembly into the device such that the columns are fully seated into the Inogen One® G4 concentrator. The spring loaded latch button should fully return to the closed position.



11. Connect the AC power supply cord to the Inogen One® G4 and plug the power supply AC cord into an electrical



outlet. Do Not Power on the Inogen One® G4 concentrator.

12. Press and hold the plus (+) and (-) minus buttons for 5 seconds. The screen will display “sieve reset” message.



Release buttons once message is displayed on screen.

13. Press the alert/bell button once and screen will display “sieve reset success.” Press the power button to turn on



the Inogen One® G4, and use it normally.

14. If you are using Inogen Connect App, navigate to the Advanced screen, then to Additional Information screen and click on the Column Reset button.


Note: Column change instructions are only to be used when maintenance is required and are not intended for practice purpose.

## Your Equipment Provider

Your Physician:

- 301 Coromar Drive,
- Goleta, CA 93117
- 1-877-4-IMOGEN (Toll Free)
- 1-877-446-6436 (Toll-Free)
- 1-805-562-0515 (International)
- [info@inogen.net](mailto:info@inogen.net)
- [www.inogen.com](http://www.inogen.com)

## Documents / Resources

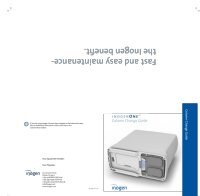

## [Inogen One G3 Replacement Columns](#) [pdf] Instructions

### One G3 Replacement Columns

## References

- [O Inogen Portable Oxygen Concentrators | Oxygen Therapy](#)

## [Manuals+](#),

- [home](#)
- [privacy](#)