



**INTEGREX 2024 Imagine Software**

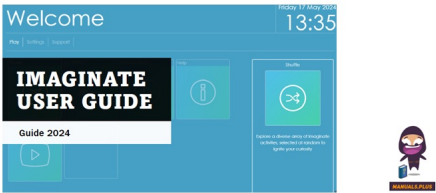

# INTEGREX 2024 Imagine Software User Guide

[Home](#) » [INTEGREX](#) » **INTEGREX 2024 Imagine Software User Guide** 

**Contents**

- [1 INTEGREX 2024 Imagine Software](#)
- [2 Product Information](#)
- [3 Introduction](#)
- [4 Remote](#)
- [5 Overview](#)
- [6 Features](#)
- [7 Instructions for use](#)
- [8 Editor Portal](#)
- [9 Measured Outcomes](#)
- [10 Frequently Asked Questions](#)
- [11 Documents / Resources](#)
  - [11.1 References](#)
- [12 Related Posts](#)



**INTEGREX 2024 Imagine Software**



## Product Information

### Specifications

- Model: Imagine
- Guide Number: 2024
- Connectivity: Wi-Fi
- Remote Control: Included
- Software Updates: Automatic

### Introduction


Welcome to the Imagine Software User Guide! Imagine is a revolutionary platform developed by Integrex, designed to foster creativity and enhance learning experiences through immersive and interactive activities. This software is tailored for a range of environments from a special needs environment, care and education. Imagine is an innovative software solution that transforms traditional learning methods into interactive and engaging experiences. Developed by Integrex, a leader in the field of educational technology, Imagine leverages cutting edge tools to create a rich, immersive environment that promotes active learning and creative exploration.

### Enhance your Journey


Imagine is dedicated to continuous improvement, ensuring that users always have access to the latest and most advanced features. Connecting to the internet unlocks a world of wssüities within the Imaginale platform. Granting you access to the newest version ot Imagine and also ensuring that you can explore and utile the latest functionalities as as they are available\_ Exigence the full ot Imagine with the latest 1\_p30tes and enhancements.

- On first start up you will be asked to connect to the internet. Connecting your machine to the internet ensures you have the latest version of the software and any available software updates will download automatically, this could include new activities, themed activities and bug fixes.
- When you choose connect to the Wi-Fi it will ask you to enter your network password .
- Interactive activities will automatically play and will continue to play until you press the home key on the black remote

## Remote

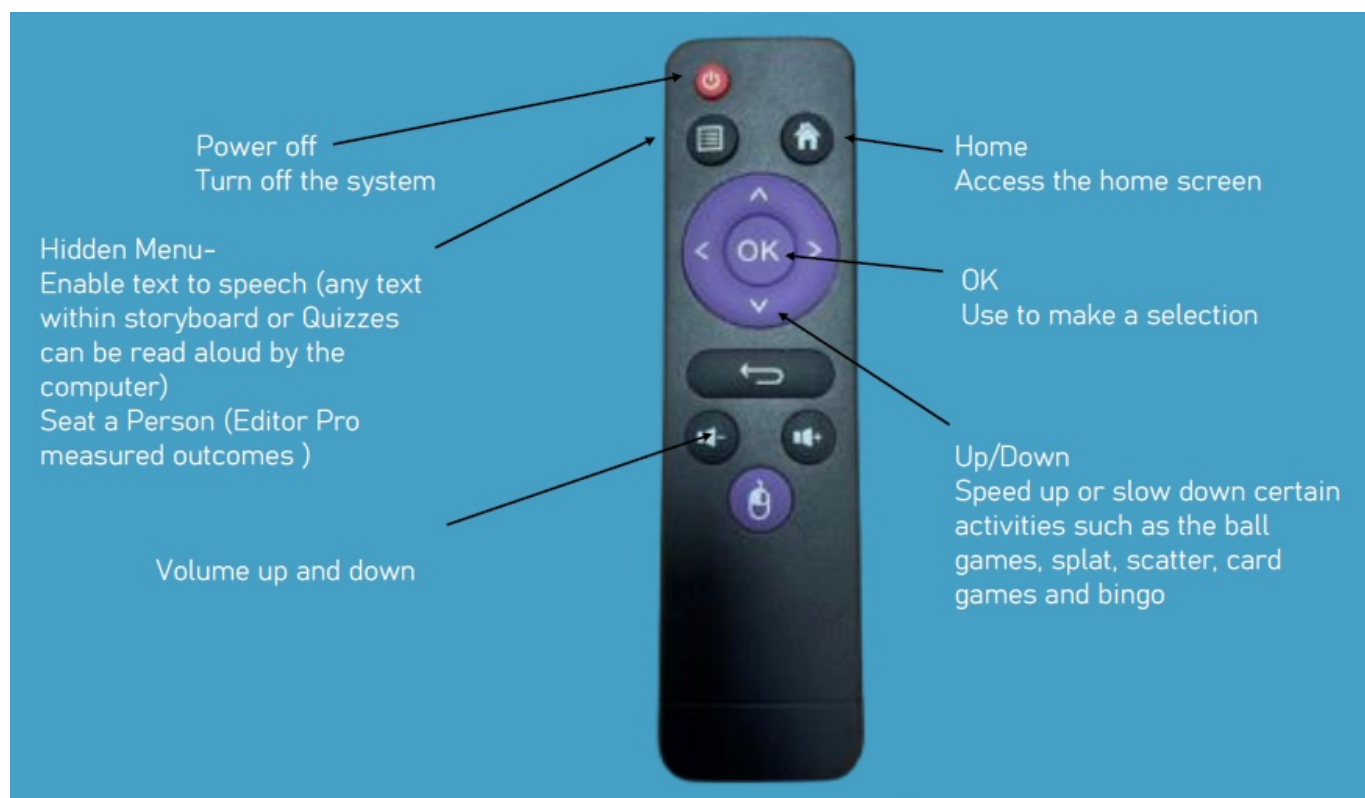

## Overview

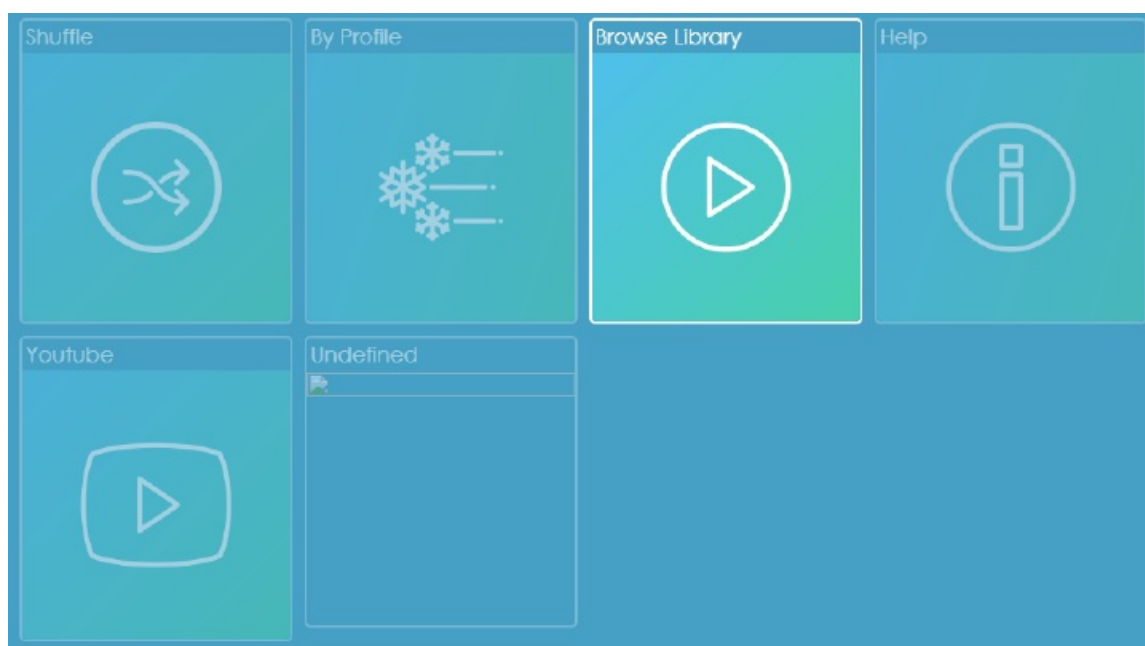

## Features

### Browser Library

Explore our vast Imagine activity library, offering everything from bubble pop games to interactive stones. Whatever your preference, we have got you covered!



### **Shuffle**

Explore a diverse array of Imagineate activates, selected at random to ignite your curiosity



### **By Profile**

Choose from a range of categorized profiles, each containing tailored Imagineate activities to suit your preferences



### **Browser Library**

Explore our vast Imagineate activity library, offering everything from bubble pop games to interactive stories. Whatever your preference, we've got you covered!



### **Help**

Maximize your Imagine journey by delving into our extensive collection of help guides, ensuring you make the most out of your experience



### **YouTube**

Explore a wide array of categorized YouTube videos tailored to your desired category



### **Summer**

Discover the range of Imagine Profiles available, starting with those featured in the Summer event Themed events will appear in this section. There maybe more then one event running so check out this section dial



## Instructions for use


### Setting

USE WITH CAUTION Maximize your Imagine experience by fine-tuning the settings for optimal enjoyment



### Network

Stay up-to-date with the latest Imagine activities while maintaining flexibility by connecting to a WiFi network

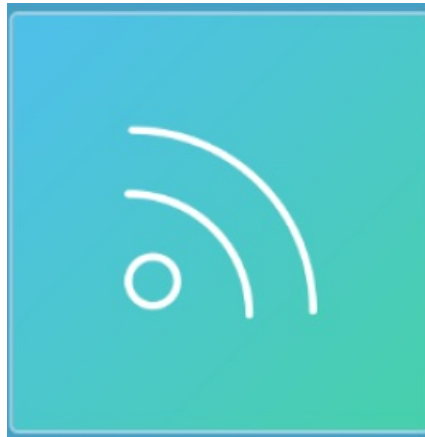

### Resize

Enhance your Imagine journey by customizing the playing area to fit your preferred surface, ensuring maximum enjoyment and comfort




### **Close**

It's time to wrap up the fun! Le close Imagine and return to Windows



### **Restart**

Let's restart Imagine for a refreshed and enjoyable experience

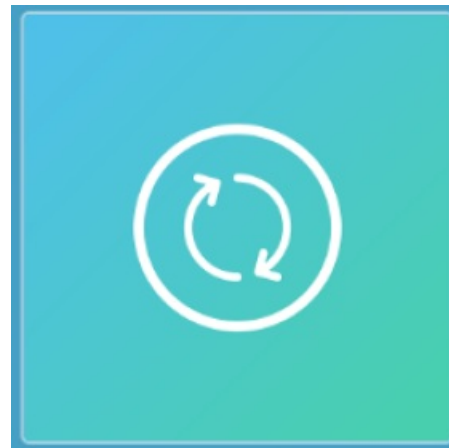

### **Shutdown**

The fun concludes here. Let's shut down Imagine



## General Information

🌟 Unlock Your Creative Potential with 'magnate Pro' 🚀

- 🧠 **Digital Storyboard Creator:**

Experience storytelling with our digital storyboard creator Whether pro or Beginner, craft compelling narratives effortlessly. Bring your ideas to life with ease\_

- 🎯 **Interactive Quiz Creator:**

Captivate with dynamic quizzes! Imagine Pro empowers you to create engaging quizzes that inspire and entertain. From trivia to assessments, spark curiosity and make a lasting impression.

- 📊 **Full Reporting Interface:**

Gain valuable Insights Into user behavior and preferences with our comprehensive reporting interface. Track performance metrics, analyze user feedback, and identify opportunities for improvement with precision.

## ImagineSuite

- system 'D': reoatd51-7aöd-G0d-86ts-5d05et94/Æ52
- Serial: EU-I-I O
- Version: 1.2.1.93
- 09/07/2094
- communicated: ff2/07/2024
- P Address: 192.10.10.244
- MAC: C8D9D2185CFE

## Editor Portal

- With your unit or room, you will have been provided with a laminated sheet containing the username and password for your Cloud Editor To access the Cloud Editor visit [www.touchthefuture.co.uk](http://www.touchthefuture.co.uk) and login with the details provided.
- If you have lost these details or have any problems logging on, then please call 01283 551551. (Office hours Mon Fri 9am 5pm)
- When in the Editor, you can create new activities and profiles as well as import your own images, videos and




sounds

- Please visit our website for short user friendly clips on how to use the editor


<https://www.integrex.co.uk/information/faqs/sense/imagineeditorportal2/>

## IOS or Android App


Download the app via the google play store or the apple store

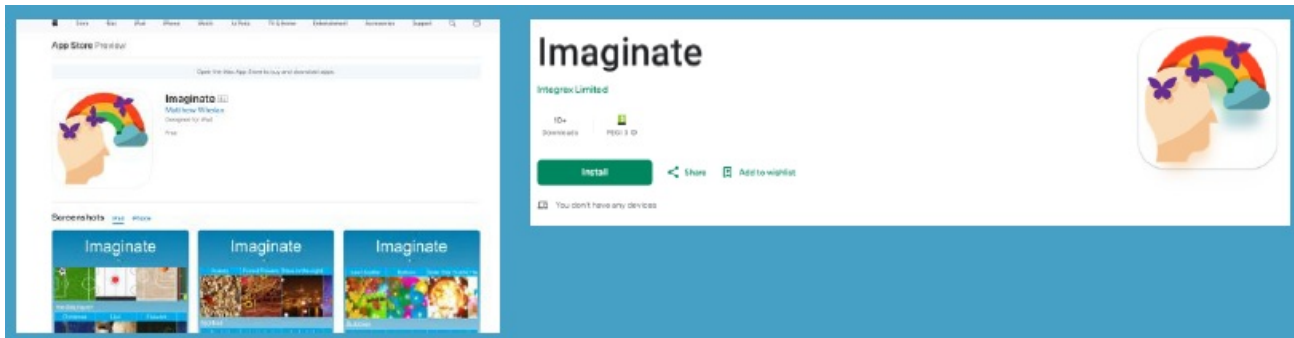


- Access your online editor portal via [www.touchthefuture.co.uk](http://www.touchthefuture.co.uk) you will have been provided a user name and password on a laminated sheet. If you cant locate this please call 01283 551551 or email [support@integrex.co.uk](mailto:support@integrex.co.uk)
- Once in the editor access the admin screen, find your machine in the list and press the i button on the right hand side.

Name	Description	Information
M4-10001	M4-10001	

A pop up box will appear. Press the view link code and a QR code will pop up.

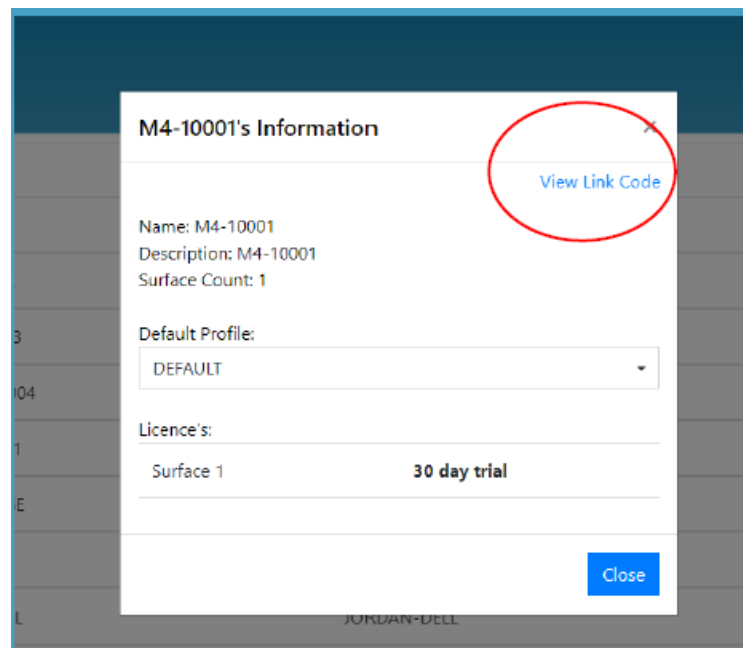


- Open the Imagine app on your device and scan the QR code. Make sure your room or mobile interactive projection unit is turned on and connected to the internet.
- This will now allow you to control the software in your room or on you interactive projection unit via the device you downloaded the APP onto, you can choose an instant play activity or select a profile. If you have an Immersive Room you can control the Hardware such as Lighting and Aromatherapy from within the App.

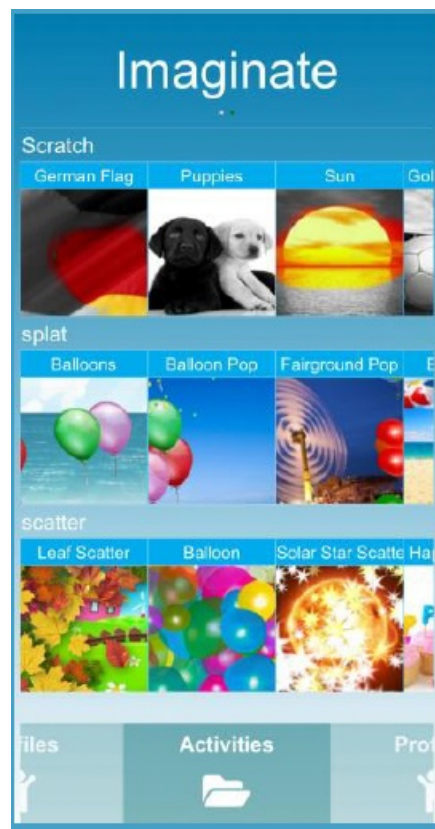


## Activity Wizard

Utilize the “instant Activity Wizard” to create personalized activities by incorporating a variety of your own images

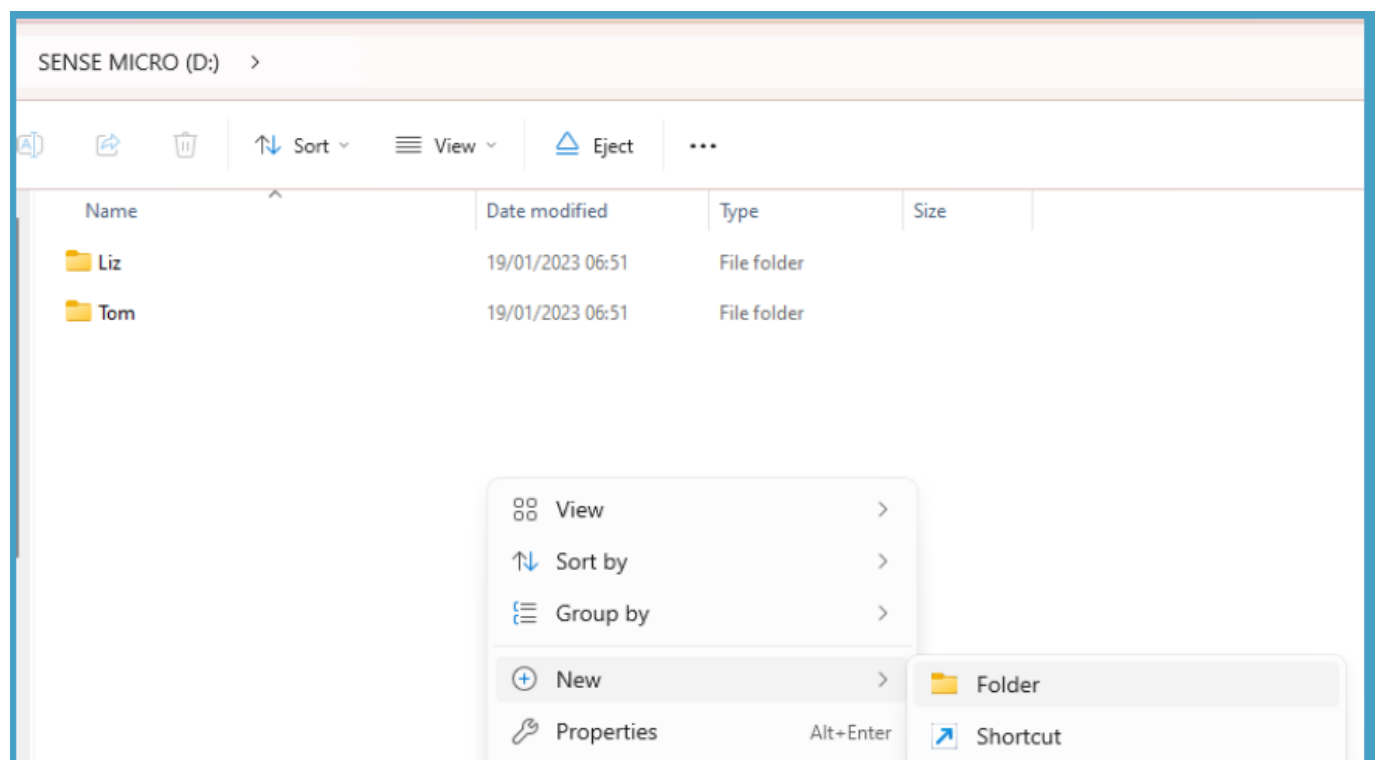

## Create a folder on a USB memory stick

Create a folder on a USB memory stick and make sure you give it a descriptive name for easier identification when using the images later

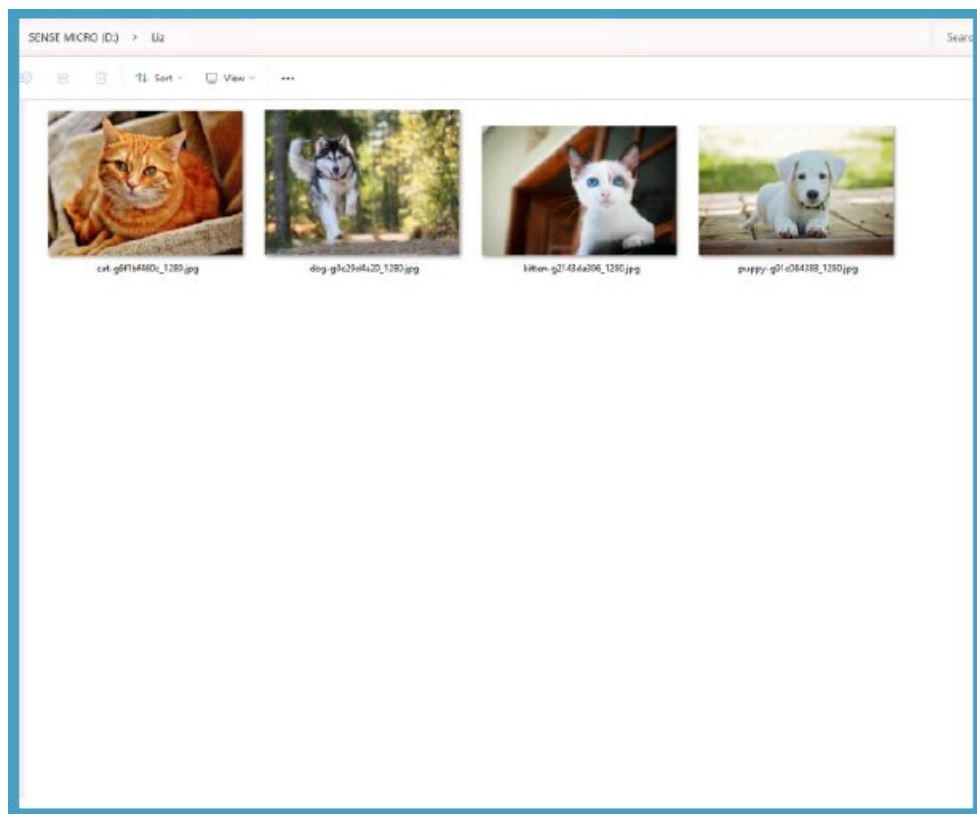


### Compile a collection of your preferred images

- Gather a set of your favourite images for use in your customised activities. Make certain that these images are relevant to the activities you are designing, adding these images to the appropriate folder on the USB memory stick.
- Utilize the “instant Activity Wizard” to create personalized activities by incorporating a variety of your own images



### Begin the instant Activity Wizard Journey

Insert the prepared USB Memory Stick into an accessible USB port on the device.

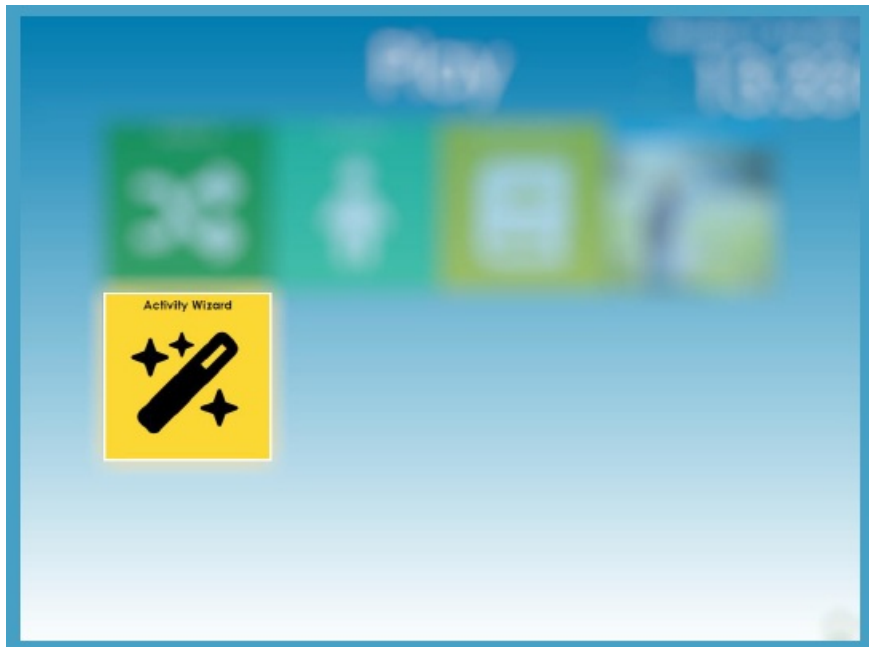

### Select Activity Wizard

- This immediately assembles a range of activities using your images, resulting in a genuinely personalized and immersive experience.
- Utilize the “instant Activity Wizard” to create personalized activities by incorporating a variety of your own images

### Start your activity

Pick one of the numerous personalized activates by exploring the list



### Navigate through your personalized activities

Simply Navigate through your images by using the left or Right arrow buttons on the remote control




### **Turning off the system**

- Press the home button on the remote
- Choose the Settings section on the welcome screen
- Select the Shutdown option within here and this will shutdown the software as well as the mobile unit or the room pc and projectors


### **Measured Outcomes Editor Pro Required**

- The Sense Micro Interactive Projector offers a wealth of benefits by providing measurable outcomes through its advanced tracking capabilities. To begin, simply seat a resident or group of residents at the start of a session. Subsequently, the Well Being Suite will prompt you to rate how they are feeling. During the session, the Well Being Suite or Imagine Suite will discreetly monitor various outcomes. Consequently, you can generate detailed reports over a selected date range.
- Furth more, by monitoring a variety of metrics, the Sense Micro delivers comprehensive well being scores. This, in turn, allows users to gain valuable insights into their physical, mental and emotional health, thereby promoting a balanced and holistic approach to well being.
- You will be provided with a username and password to access your personalized portal

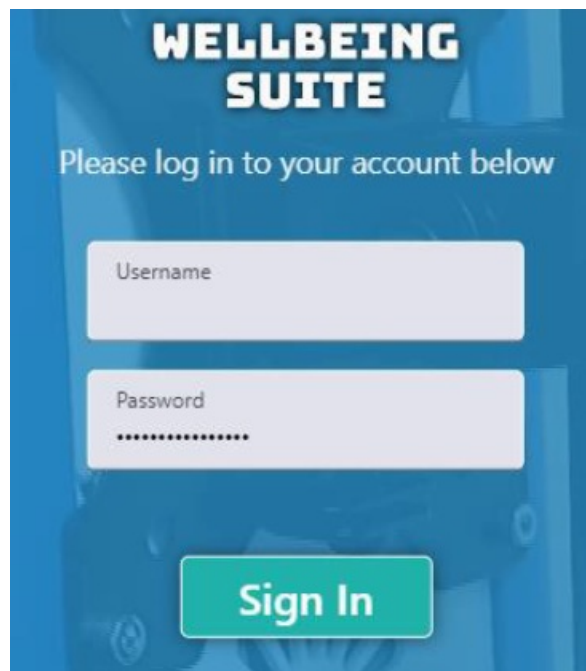


## Measured Outcomes

- Once inside the portal, you can start to add each user via the add new button.
- It will ask you to type in the user's name, once all users are entered into the portal we can start to use the tracking capabilities of the Well Being Suite.

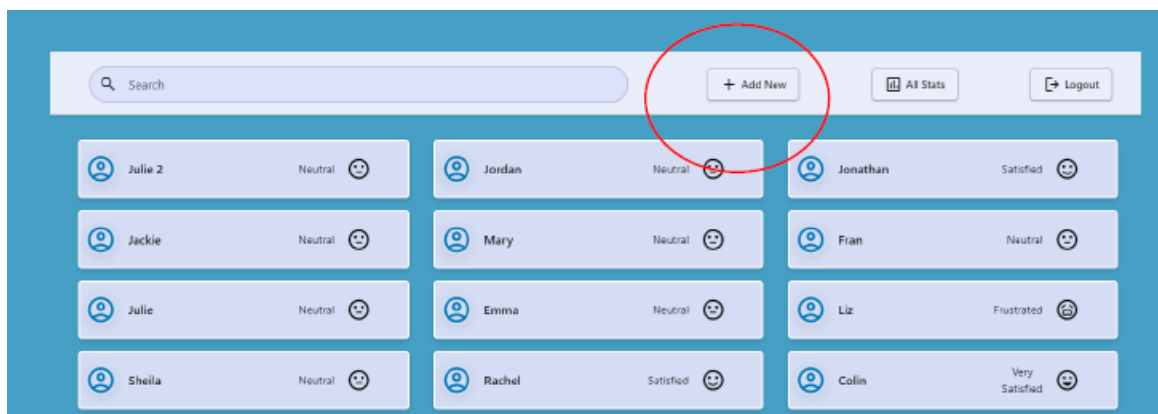


- To allow the system to track an individual you will need to seat them at the table within the Software. If you press the button on the remote with the four lines, it will bring up a box to allow you to seat a user or multiple users. When seating a user, it will ask for a well being score at the beginning of the session, it will ask again at the end and this can be selected via the OK button on the remote, the user or caregiver can make this assessment. This allows for the system to track wellbeing scores at the beginning of a session and at the end.

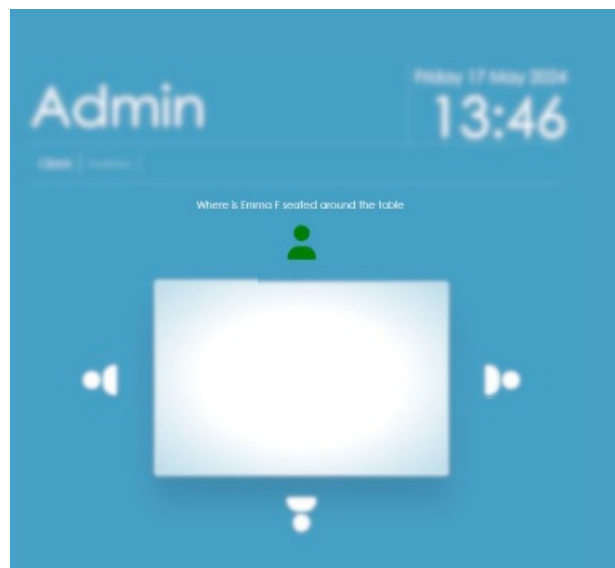

## Measured Outcomes

### Accessing the Data

We can access the data the system has been tracking through the portal, either for the setting as a whole or an individual

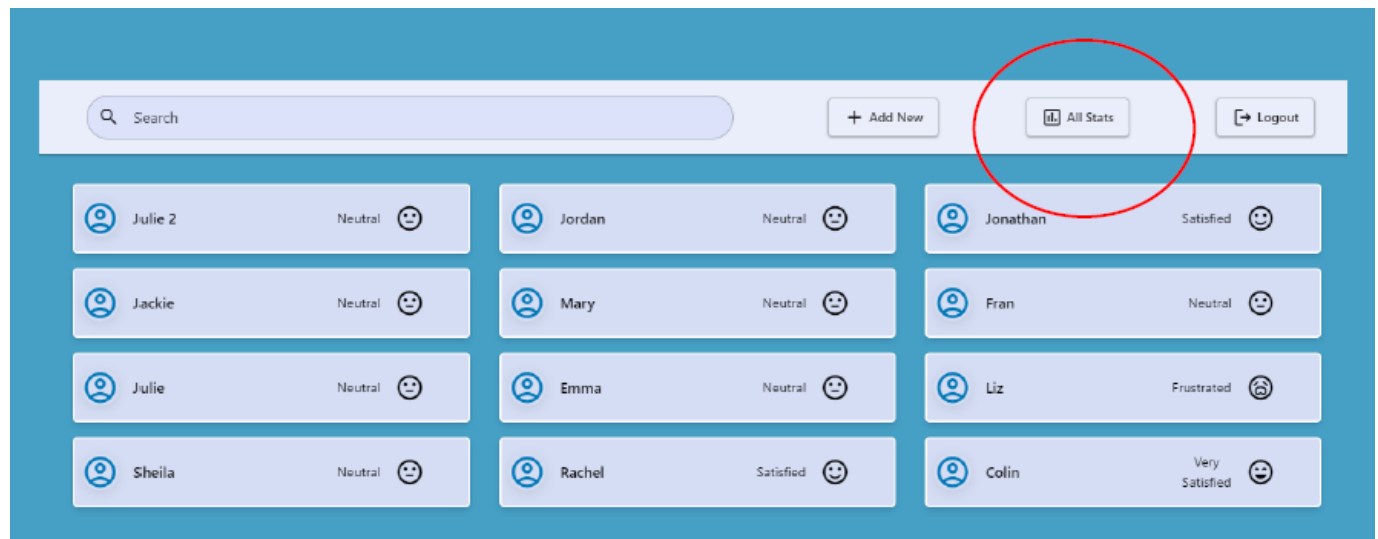

### What is measured?

The Well Being suite will measure:

- Wellbeing scores
- Average reaction time
- Slowest and fastest reactions
- Playtime
- Left and right hand usage
- Interaction areas
- Popularity of activities





## Help

- The help guide can be found in the top right hand corner and will bring up a video user guide of everything covered in this guide
- If you are experiencing any technical difficulties with the machine or software please call 01283 551551
- Remote training sessions are available please call 01283 551551 to arrange.
- Office Hours are 9am 5pm.

## Frequently Asked Questions

- **Q: How do I access new themed events?**
  - A: Themed events are available in the Editor Portal section where you can check daily for new events.
- **Q: How do I control hardware functions with the app?**
  - A: If you have an Immersive Room, you can control hardware functions such as lighting and aromatherapy from within the app.

## Documents / Resources

	<p><b><a href="#">INTEGREX 2024 Imagine Software</a></b> [pdf] User Guide  2024, 2024 Imagine Software, Imagine Software, Software</p>
--	--

## References

- [Integrex Imagine](#)
- [Imagine Editor Portal - Integrex](#)
- [User Manual](#)

### Manuals+. [Privacy Policy](#)

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.