



## Contents

- [1 Journey Soft Secure Rotating Shower Chair](#)
- [2 INTRODUCTION AND NOTES](#)
- [3 Safety Instructions](#)
- [4 PRODUCT DESCRIPTION](#)
- [5 MICROBAN® ANTIMICROBIAL PRODUCT PROTECTION](#)
- [6 INSTRUCTIONS FOR USE](#)
- [7 SPECIFICATIONS AND WARRANTY](#)
- [8 Warranty](#)
- [9 FAQ](#)
- [10 Documents / Resources](#)
  - [10.1 References](#)
- [11 Related Posts](#)



## Journey Soft Secure Rotating Shower Chair



## INTRODUCTION AND NOTES

### Welcome to Journey Health & Lifestyle

Thank you for purchasing your SoftSecure Rotating Shower Chair. You've invested in a high-quality, premium materials product that will facilitate comfort and safety.

### Safety Instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Approved for indoor use only.
- Clothing or body parts can be pinched when sitting down or standing up, as well as when raising the armrests and locking them into place.
- Be careful on the seat; be aware of wet and slippery surfaces.
- Prevent unauthorized use, for example, by children.
- Take note of the maximum weight permitted.

## PRODUCT DESCRIPTION

### Parts of the Product



1. Backrest
2. Seat
3. Flip Up Armrest
4. Rotating Plate with Knob
5. Push Button

## MICROBAN® ANTIMICROBIAL PRODUCT PROTECTION

### Microban® Antimicrobial Product Protection

- Microban® antimicrobial\* product protection is built-in to make your Rotating Shower Chair last longer and look cleaner
- Microban® antimicrobial\* product protection helps prevent the uncontrolled growth of bacteria, mold and mildew on the Rotating Shower Chair and provides 24/7 cleanliness protection



Microban® is a registered trademark of Microban Products Company

\* These antimicrobial properties are built-in to protect the Rotating Shower Chair. The Rotating Shower Chair does not protect users or others against bacteria, viruses, germs or other disease organisms

## INSTRUCTIONS FOR USE

### Assembly

#### Step 1

- Attach the legs to the frame by depressing push buttons into the tubes located on each side of the legs.
- Make sure that push buttons “snaps” correctly through holes and are stable and adjusted to the same height.



#### step 2

- Attach backrest to the frame, by depressing push buttons and sliding the backrest into the tubes located on each side and on the back of the seat.



### step 3

Check that the seat is stable and level prior to use.

1. Pull the knob and turn the seat to face the user, and lock the seat into position by locking the knob and keep swiveling the seat until it locks in position.



2. Unlock the seat by pulling the knob.
3. When user is seated on the chair, position the seat to ensure that it is locked into position and no longer rotates freely.
4. Lock the shower chair when the user faces the open side of the bath. The user can now stand up carefully and slowly.
5. To flip up the armrest, push the button shown below.



## SPECIFICATIONS AND WARRANTY

### Product Specifications

Product Dimension	(18"-19") x (18"-19") x (30"-34")
Weight Capacity	300 lbs
Packaging Dimensions	25" x 18" x 16"
Net Weight	21 lbs
Material on Product	Aluminum

### Warranty

Journey Health & Lifestyle warrants the SoftSecure Rotating Shower Chair frame to be free from defects in materials, workmanship and assembly for twelve (12) months from the original purchase date. The warranty does not extend to non-durable components such as the rubber tips.

If you have any questions, call us at our toll free number: 1-[800-958-8324](tel:800-958-8324)

### FAQ


• **Q: What is the weight capacity of the rotating shower chair?**

**A:** The weight capacity is 300 lbs.

• **Q: How long is the warranty for the frame?**

**A:** The frame warranty is twelve (12) months from the original purchase date. If you have any further questions, please call our toll-free number: 1-[800-958-8324](tel:800-958-8324).

### Documents / Resources

	<p><a href="#">Journey Soft Secure Rotating Shower Chair [pdf]</a> Instruction Manual Soft Secure Rotating Shower Chair, Secure Rotating Shower Chair, Rotating Shower Chair, Shower Chair, Chair</p>
---	---

### References

- [Shop Lightweight Wheelchairs, UPWalkers, and Mobility Scooters Online – Journey Health & Lifestyle](#)
- [User Manual](#)