

Home » Journey » journey SoftSecure Commode with Backrest Instruction Manual







## **Contents**

- 1 SoftSecure Commode with Backrest
- **2 INTRODUCTION AND NOTES**
- **3 PRODUCT DESCRIPTION**
- **4 MICROBAN® ANTIMICROBIAL PRODUCT**


**PROTECTION** 

- **5 INSTRUCTIONS FOR USE**
- **6 SPECIFICATIONS AND WARRANTY**
- 7 Documents / Resources
  - 7.1 References

# **SoftSecure Commode with Backrest**

#### Scan Here

With Your Phone To

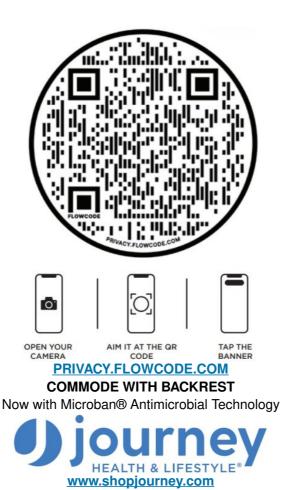

## **INTRODUCTION AND NOTES**

### Welcome to Journey Health & Lifestyle

Thank you for purchasing your SoftSecure Commode with Backrest. You've invested in a high-quality, premium materials commode with backrest that will facilitate comfort and safety.

#### **Safety Instructions**

- Read these instruction for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Approved for indoor use only.
- Clothing or body parts can be pinched when sitting down, standing up, or inserting the toilet bucket.
- Prevent unauthorized use, for example, by children.
- Take note of the maximum user weight permitted.

#### PRODUCT DESCRIPTION

#### Parts of the Product



	1. Armrest
<b>O</b>	2. Seat
	3. Height Adjustable Leg
	4. Rubber Tip
<b>O</b>	5. Backrest
	6. Commode Bucket

## **MICROBAN® ANTIMICROBIAL PRODUCT PROTECTION**

### Microban® Antimicrobial Product Protection

- Microban® antimicrobial\* product protection is built-in to make your Commode With Backrest last longer and look cleaner
- Microban® antimicrobial\* product protection helps prevent the uncontrolled growth of bacteria, mold and mildew on the Commode With Backrest and provides 24/7 cleanliness protection



Microban® is a registered trademark of Microban Products Company

\* These antimicrobial properties are built-in to protect the Commode With Backrest. The Commode With Backrest does not protect users or others against bacteria, viruses, germs or other disease organisms

#### **INSTRUCTIONS FOR USE**

#### **Assembly**

#### Step 1

Attach armrests to the frame by turning the knobs and sliding the armrest into the tubes located on each side of the seat.





that the push buttons "snap" correctly through holes and are stable and adjusted to the same height.



**Step 3** Attach the backrest tube to the frame.



**Step 4**Slide Commode Bucket into guide rails under the seat.



## **SPECIFICATIONS AND WARRANTY**

# **Product Specifications**

Product Dimension	(26"-27") x 18" x (31"-35")
Weight Capacity	300 lbs
Packaging Dimensions	22" x 10" x 25"
Net Weight	20 lbs
Material on Product	Aluminum

### Warranty

Journey Health & Lifestyle warrants the SoftSecure Commode With Backrest frame to be free from defects in materials, workmanship assembly for twelve (12) months from the original purchase date. The warranty does not extend to non-durable components such as the rubber tips.





### **COMMODE WITH BACKREST**

If you have any questions, call us at our toll free number: 1-800-958-8324

**Documents / Resources** 



journey SoftSecure Commode with Backrest [pdf] Instruction Manual SoftSecure Commode with Backrest, SoftSecure, Commode with Backrest, Backrest, Commod

#### References

- Shop Lightweight Wheelchairs, UPWalkers, and Mobility Scooters Online Journey Health & Lifestyle
- User Manual

Manuals+, Privacy Policy | @manuals.plus | YouTube

This website is an independent publication and is neither affiliated with nor endorsed by any of the trademark owners. The "Bluetooth®" word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. The "Wi-Fi®" word mark and logos are registered trademarks owned by the Wi-Fi Alliance. Any use of these marks on this website does not imply any affiliation with or endorsement.