



## Juzo Dynamic Varin Soft In Prosthetic Above Knee Stump Shrinker Instructions

[Home](#) » [Juzo](#) » Juzo Dynamic Varin Soft In Prosthetic Above Knee Stump Shrinker Instructions 

### Contents

- [1 Juzo Dynamic Varin Soft In Prosthetic Above Knee Stump Shrinker](#)
- [2 Prosthetic Shrinker](#)
- [3 Care and Maintenance](#)
- [4 Instructions for Use](#)
- [5 Documents / Resources](#)
- [6 Related Posts](#)



**Juzo Dynamic Varin Soft In Prosthetic Above Knee Stump Shrinker**




## Prosthetic Shrinker

- Therapeutic compression
- Soft, breathable knit

20-30 and 30-40 mmHg

### Size Chart for Juzo Above Knee Shrinker

	I	II	III	IV	V	VI
G	44 cm 17 1/4"	48 cm 19"	52 cm 20 1/2"	56 cm 22"	60 cm 23 3/4"	64 cm 25 1/4"
F	41 cm 16 1/4"	44 cm 17 1/4"	47 cm 18 1/2"	50 cm 19 3/4"	53 cm 20 3/4"	56 cm 22"
E	31 cm 12 1/4"	34 cm 13 1/2"	37 cm 14 1/2"	40 cm 15 3/4"	43 cm 17"	46 cm 18"

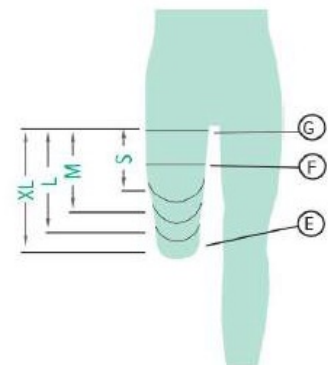

AK Lengths: Extra Long - 14" / 35 cm Long - 12" / 30 cm Medium - 10" / 25 cm Short - 8" / 20 cm

## 20-30 and 30-40 mmHg

### Size Chart for Juzo Below Knee Shrinker

	0	I	II	III	IV	V	VI	
F	38 cm 15 1/4"	41 cm 16 1/4"	44 cm 17 1/4"	47 cm 18 1/2"	50 cm 19 3/4"	53 cm 20 3/4"	56 cm 22"	
E	28 cm 11 1/4"	31 cm 12 1/4"	34 cm 13 1/2"	37 cm 14 1/2"	40 cm 15 3/4"	43 cm 17"	46 cm 18"	
C	26 cm 10 1/2"	29 cm 11 1/2"	32 cm 12 1/2"	35 cm 13 3/4"	38 cm 15"	41 cm 16 1/4"	44 cm 17 1/2"	

BK Lengths: Long - 18" / 46 cm Medium - 15" / 38 cm Short - 12" / 30 cm

Juzo USA Manufactured by Julius Zorn, Inc [www.juzousa.com](http://www.juzousa.com) PO Box 1088 Juzo Canada LTD. Cuyahoga Falls, OH 44223 [www.juzo.ca](http://www.juzo.ca) Juzo and Keep Moving are registered trademarks of Julius Zorn, Inc 2019 Julius Zorn Incorporated IN STUMP-1019

### Freedom to Make the Most of Every Day

Juzo prosthetic shrinkers are designed to improve your quality of life. not inhibit it. Family-owned since 1912. Juzo has led the compression therapy garment industry by creating high-quality products that offer you the freedom to maintain a more active lifestyle. Thank you for choosing Juzo.

### Care and Maintenance

To maintain your Juzo shrinker, we recommend the following:

- Hand or machine wash warm daily to maximize therapeutic effectiveness and prolong the life of your Juzo garment.
- When machine washing and drying, use the lowest permanent press settings. A garment bag is strongly recommended and available in the Juzo Compression Therapy
- Car accessory package.
- Wash dark colors separately from other clothing.
- Use Juzo washing detergent or another mild soap.
- DO NOT USE bleach, chlorine, fabric softeners, or other laundry additives. These may damage your garment.
- Please refer to the washing symbols on the label that seem into your Juzo shrinker.
- Store your Prosthetic Shrinker in a dry place at room temperature.

### Return Policy

Should it become necessary to return your garment for evaluation, for hygienic reasons Juzo requires that the garment be laundered prior to returning to the original place of purchase?

### Fabric

Details are provided on a label sewn inside your prosthetic shrinker.

## Instructions for Use

It is recommended that you consult with your medical professional and fitter for proper guidelines on the use and application of this product. As therapy progresses or new symptoms develop, it may be necessary to take new measurements for a new prosthetic shrinker.

1. Turn the shrinker inside out. except for its end part.
2. Take the shrinker. folded over itself, in both hands and slip it over the residual limb.
3. Turn the shrinker back and pull it evenly over the residual limb.
4. Now stroke the whole shrinker gradually upwards, smoothing it to your leg with the palm of your hand.

**Important:** Never pull at the upper edge of the prosthetic shrinker

### Indications

Age or trauma-related amputation, prevention of edemas after removal of prosthesis improving circulation in the residual limb. The compression class must be determined by your physician. Regular medical visits will help to ensure that your therapy is as successful as possible.

### Contraindications


Open wounds, weeping skin areas, arterial insufficiency, stasis dermatitis, acute thrombophlebitis, non, reduced edemas, and non-compensated cardiac insufficiency. Julius Zönn Inc assumes no liability as a result of any contraindicated use of this product

### Cautions

It is recommended that Juzo Silver Garments be removed prior to cardiac defibrillation and MRI scans. No side effects are currently known. However a negative result should occur (Le. skin irritations) from the use of our product, please consult with a medical professional The manufacturer,s not Hable for damage/ Injuries which have been caused by improper handling or misuse.

Always use a Juzo donning glove when applying compression garments.

## Documents / Resources

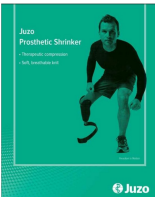	<p><a href="#">Juzo Dynamic Varin Soft In Prosthetic Above Knee Stump Shrinker</a> [pdf] Instructions Dynamic Varin Soft In Prosthetic Above Knee Stump Shrinker, Prosthetic Above Knee Stump Shrinker, Above Knee Stump Shrinker, Knee Stump Shrinker, Stump Shrinker, Shrinker</p>
---	--