



# LAVA Pulse Build in Heart Rate & BP Sensor Phone User Manual

[Home](#) » [LAVA](#) » LAVA Pulse Build in Heart Rate & BP Sensor Phone User Manual 

## Contents

- [1 LAVA Pulse Build in Heart Rate & BP Sensor Phone](#)
- [2 USING INSTRUCTION](#)
- [3 Documents / Resources](#)
- [4 Related Posts](#)

**LAVA** Pulse

## LAVA Pulse Build in Heart Rate & BP Sensor Phone



## USING INSTRUCTION

Simple steps to measure your Heart Rate/BP

1. Press the\* key on dial pad and unlock your phone.



2. Long press the centre navigation key to open Lava Pulse app.



3. Once the app opens choose option 1 and place your finger on camera.



4. Keep your finger on the camera till the phone screen shows 100%.




5. Now remove your finger from camera and see the result on phone screen.



#### Disclaimer:

1. This mobile phone has a health feature that calculates heart rate and blood pressure rate.
2. The health feature incorporated in the mobile phone is only for assistance in measuring the well-being of the user and not a substitute of medical equipment (such as heart rate calculator and sphygmomanometer). Before acting upon the result based on the mobile phone, user may resort to medical practitioner for professional check-up. While all efforts are made to provide accuracy in the results, it is subject to technical configurations of the sensor.
3. The recommendation is as per ACC/AHA.

#### Documents / Resources

	<p><a href="#">LAVA Pulse Build in Heart Rate &amp; BP Sensor Phone</a> [pdf] User Manual Pulse, Build in Heart Rate BP Sensor Phone, Heart Rate BP Sensor Phone, Build in BP Sensor Phone, BP Sensor Phone, Sensor Phone, Phone</p>
---	--