



LECKEY Harness Range Instruction Manual

[Home](#) » [LECKEY](#) » LECKEY Harness Range Instruction Manual 

LECKEY

Harness Range
User Instructions



Contents

1 FITTING THE HARNESS

1.1 LECKEY Kit

1.2 LECKEY Mygo Seat

1.3 LECKEY Squiggles Seat

1.4 LECKEY Everyday Activity Seat

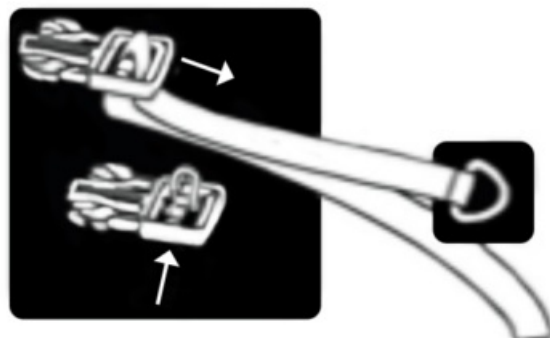
2 Documents / Resources

3 Related Posts

FITTING THE HARNESS






Please note that when fitting the harness strap to the Camlock, you must follow the following safety measures:

- Ensure the male buckle is positioned with the center bar flat side up (triangular point facing downwards) and the longest edge of the buckle to the top.
- Thread strap through the rear slot (furthest away from the buckle)
- Push strap down through the front slot (closest to the buckle)
- Thread strap through Camlock and close to secure
- Use the D ring to adjust the position of the strap

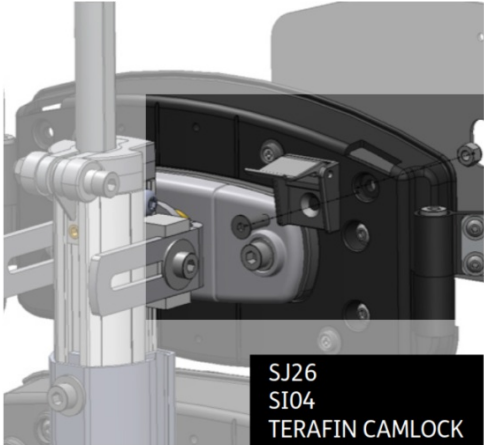
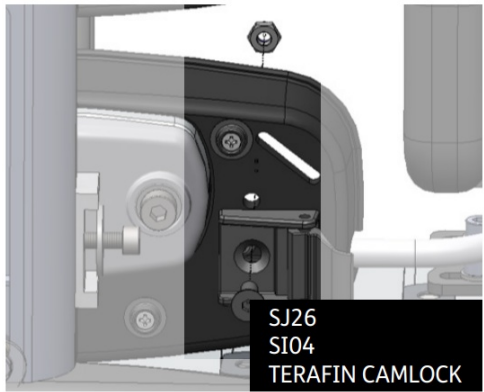


The pack should contain the following

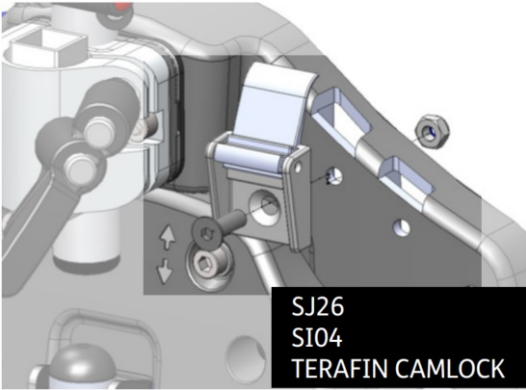
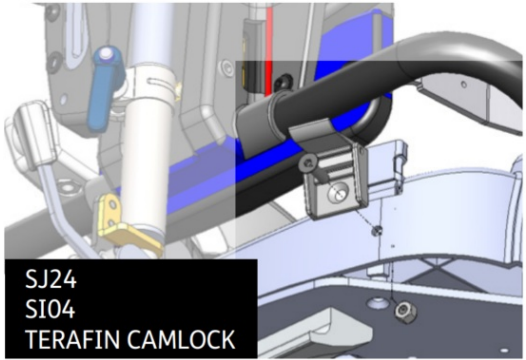
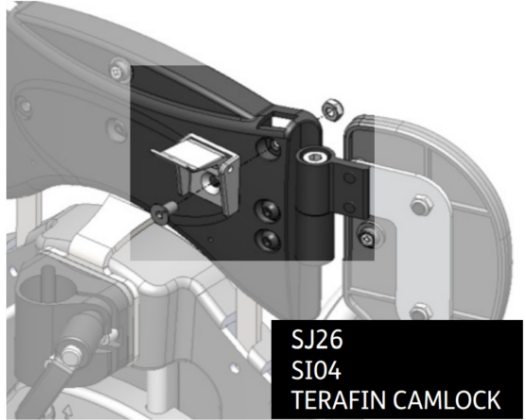
(Screws below are scaled 1:1 so can be used as a size reference)

ags	Used In
 <p>SJ117 (4OFF)</p>	Leckey Everyday Activity Seat
 <p>SJ26 (4OFF)</p>	Leckey Squiggles Seat Leckey Mygo Seat Leckey KIT
 <p>SJ24 (2OFF)</p>	Leckey Squiggles Seat Leckey Mygo Seat
 <p>SJ25 (2OFF)</p>	Leckey Squiggles Seat
 <p>S104 (4OFF)</p>	Leckey Squiggles Seat Leckey Mygo Seat Leckey KIT Leckey Everyday Activity Seat

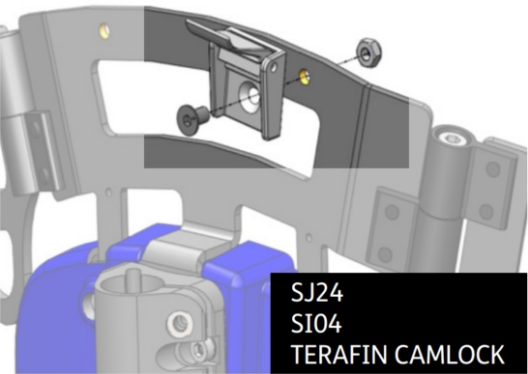
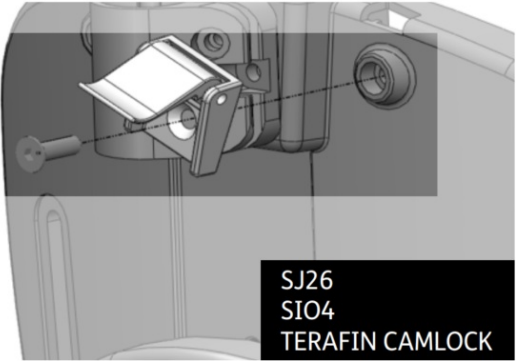
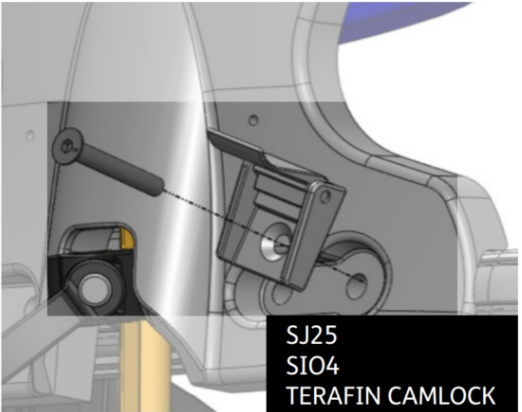
LECKEY Kit

<p>KIT upper</p>	 <p>SJ26 SI04 TERAFIN CAMLOCK</p>
<p>KIT Lower</p>	 <p>SJ26 SI04 TERAFIN CAMLOCK</p>

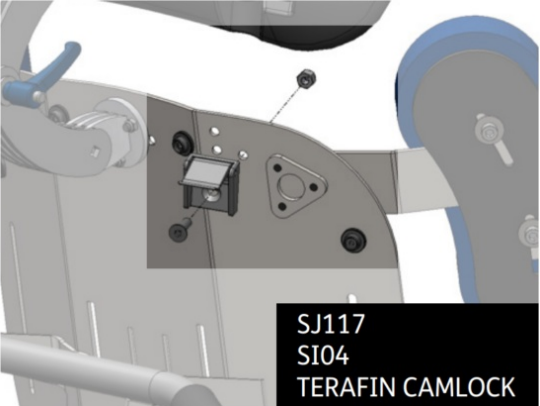
LECKEY Mygo Seat

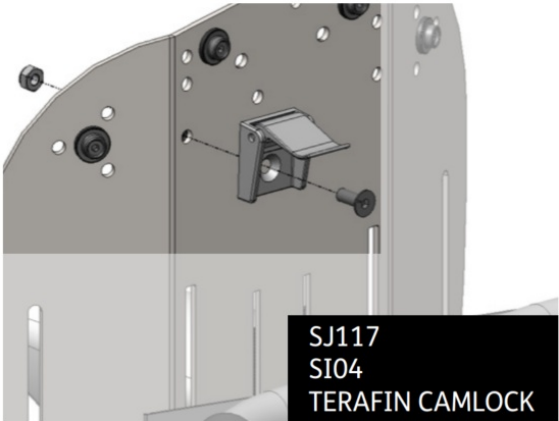
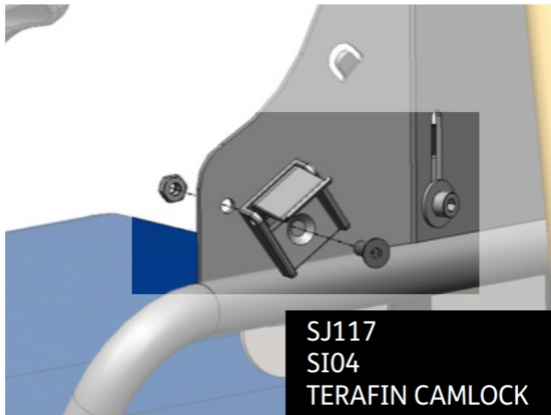

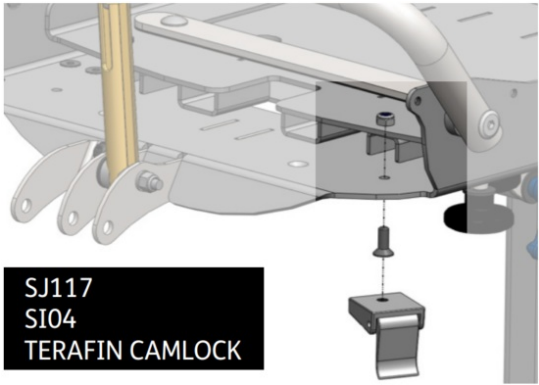
<p>Mygo Upper</p>	
<p>Mygo Lower</p>	
<p>Mygo shoulder ext</p>	

LECKEY Squiggles Seat

<p>SQSE shoulder ext</p>	 <p>SJ24 SIO4 TERAFIN CAMLOCK</p>
<p>SQSE upper</p>	 <p>SJ26 SIO4 TERAFIN CAMLOCK</p>
<p>SQSE lower</p>	 <p>SJ25 SIO4 TERAFIN CAMLOCK</p>

LECKEY Everyday Activity Seat

<p>EDS 3 upper</p>	 <p>SJ117 SIO4 TERAFIN CAMLOCK</p>
---------------------------	--

<p>EDS 2 upper</p>	 <p>SJ117 SI04 TERAFIN CAMLOCK</p>
<p>EDS 2&3 lower</p>	 <p>SJ117 SI04 TERAFIN CAMLOCK</p>
<p>EDS 1 upper</p>	 <p>SJ117 SI04 TERAFIN CAMLOCK</p>
<p>EDS 1 lower</p>	 <p>SJ117 SI04 TERAFIN CAMLOCK</p>



Harness Range
User Instructions



[LECKEY Harness Range](#) [pdf] Instruction Manual Harness Range