



# lifelines neuro R40 EEG Amplifier User Guide

[Home](#) » [lifelines neuro](#) » lifelines neuro R40 EEG Amplifier User Guide 

## lifelines neuro R40 EEG Amplifier User Guide



Contents

1 Quick-start guide

2 Documents / Resources

2.1 References

3 Related Posts

### Quick-start guide

The R-40 EEG Amplifier is a multi-channel electroencephalograph designed for use in routine EEG and lab monitoring applications. When supplied as part of an EEG System it includes a PC for control and storage, a video monitor, a medical-grade mains isolation transformer and an optional USB Photoc Stimulator and arm, all mounted on a special wheeled cart.

- Only use the PC supplied or authorised by Lifelines.
- If a cart is supplied, assemble it in accordance with its enclosed instructions. Ensure that the voltage rating of the isolation transformer matches the local mains supply voltage.
- The User Manual, which will be found on the enclosed CD, must be read before operating the equipment. Free PDF reader software is available here: <https://get.adobe.com/uk/reader>, if required. The User Manual is available from the llinesvisitor FTP site below and a hardcopy can be requested by using the email contact address below.
- The CD contains a stand-alone control program. Follow the instructions in the Readme file for installation instructions.
- The USB drivers will be found on the enclosed CD. After connecting the device to the host PC for the first time, at the Windows prompt, browse to the folder CD Drive:\USB Drivers. From there Windows will find the correct drivers for the version of Windows being used.
- If a Photoc Stimulator is supplied, follow the installation instructions in the Readme file on its CD.



### Cautions and Warnings


1. The R-40 EEG Amplifier must only be used by a healthcare professional, within a hospital or clinical setting, who has the training and knowledge to undertake EEG examinations and is familiar with EEG equipment and practice. The User Manual must be read in its entirety before the equipment is used.
2. **WARNING:** Do not modify this equipment without the authorization of the manufacturer.
3. **CAUTION:** Federal (USA) law restricts this device to sale by or on the order of a physician.

51272-006 – R-40 EEG Amplifier & System Quick Start Iss2.1.docx  
Part No. 1357




Lifelines Ltd, 7 Clarendon Court, Over Wallop, N2 Stockbridge, Hampshire SO20 8HU, UK Web: <http://www.llines.com> • FTP: <ftp://ftp.llines.com> • Email: [sales@LLines.com](mailto:sales@LLines.com) Tel: +44 (0)1264 782226



### Documents / Resources

	<p><a href="#">lifelines neuro R40 EEG Amplifier</a> [pdf] User Guide R40, EEG Amplifier, R40 EEG Amplifier, Amplifier</p>
---	--

### References

-  [EEG Recording Devices for the Diagnosis of Neurological and Sleep Disorders](#)
-  [Adobe - Download Adobe Acrobat Reader DC](#)
-  [Adobe - Download Adobe Acrobat Reader DC](#)

Manuals+.