

Lucid HEARING LBL-1063 Hearing App User Guide


[Home](#) » [Lucid HEARING](#) » Lucid HEARING LBL-1063 Hearing App User Guide 

Contents

- 1 [Lucid HEARING LBL-1063 Hearing App](#)
- 2 [The Lucid App Volume control:](#)
- 3 [Programs](#)
- 4 [LucidShape](#)
- 5 [Settings \(Continued\)](#)
- 6 [FREQUENTLY ASKED QUESTIONS](#)
- 7 [Documents / Resources](#)
 - 7.1 [References](#)
- 8 [Related Posts](#)



Lucid HEARING LBL-1063 Hearing App

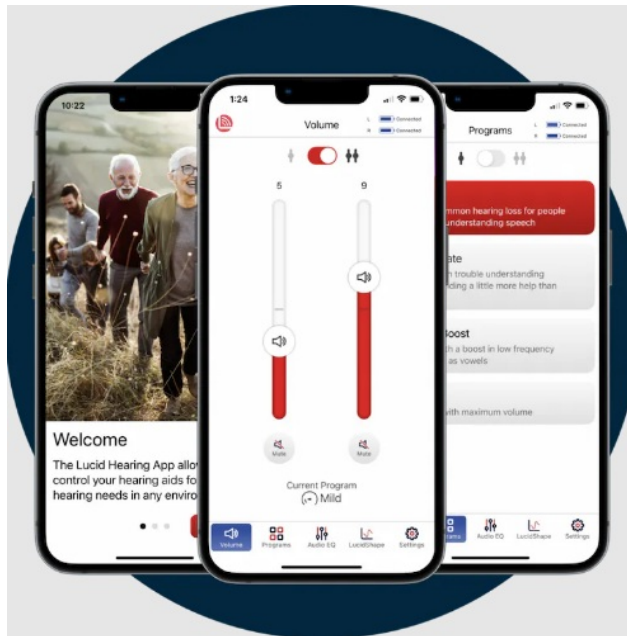

Product Information: Lucid Hearing App for Android

Thank you for purchasing Lucid Hearing's over-the-counter hearing aids. The Lucid Hearing App for Android allows you to easily set up and connect your hearing aids to your mobile device for app control and streaming.

We're Here to Help

We understand the importance of not missing any important moments, so we are always available to support your hearing journey.

Step 1: Check Your Phone Compatibility

You can check if your phone is compatible with the Lucid Hearing App by visiting www.lucidhearing.com/mobile-phone-compatibility.

Step 2: Download the Lucid Hearing App

When your hearing aids are powered on, you can download the Lucid Hearing App from Google Play. Alternatively, you can use your phone camera to scan the provided code for direct download.

Step 3: Pairing/Connecting To the App

Follow these steps to pair and connect your hearing aids to the Lucid Hearing App:

1. Open the app and click on "Start Using Lucid Hearing" on the welcome screen to begin the sign-up process.
2. If you already have a Lucid Hearing account, use the email associated with your profile. If you have a password set up, log in with that password. Otherwise, create a new password to protect your account.
3. If you do not have a Lucid Hearing account, use any email you prefer. After creating a password, check your email (including junk mail) for a confirmation link. Click on the link to confirm your account.
4. Once confirmed, create your account and start navigating the app.
5. In the app, go to the "Pair Your Device" screen where your hearing aids will be displayed.
6. Make sure your hearing aids are charged and powered on. Accept the app permissions to find nearby devices. Check the box and click "Pair" at the bottom of the screen. If prompted, click "Pair" on the pairing request dialog.

Product Usage Instructions: Lucid Hearing App for Android

Volume Control

After pairing your hearing aids, follow these steps to adjust the volume:

1. A prompt will ask you to insert the devices into the charging case (if applicable) and then remove them.
2. The Volume screen will appear, allowing you to adjust the volume of both hearing aids together.
3. To adjust independently, toggle the “Both” switch off. Slide the bar up and down to control volume or click to mute your aids if needed.

Hearing Aid Status

In the top right corner of the Volume screen, you will see a green and white battery icon box. Click on this box to view the battery level charge of your connected hearing aids. To exit the screen, click “Close”.

Programs

On the Programs tab, you can select the desired listening profile for your hearing aids. You have the option to select the same program for both or choose a unique setting for each individual side.

ANDROID CONNECTIVITY GUIDE CONNECTING (PAIRING) YOUR HEARING AIDS TO YOUR MOBILE DEVICE FOR APP CONTROL & STREAMING

Thank you for your purchase of Lucid Hearing’s over-the-counter hearing aids. This guide will help you easily setting up and connect your hearing aids.

WE’RE HERE TO HELP

We know you don’t want to miss any important moments, so we’re letting you know that we’re always available to support your hearing journey. Lucidhearing.com for Chat Support MON-SAT 8AM–6PM CST Email:

Support@lucidhearing.com

Call: (800) 785–6052 MON-SAT 8AM–6PM CST

STEP 1: Check Your Phone Compatibility

You can check for compatibility at: www.lucidhearing.com/mobile-phone-compatibility

STEP 2: Download the Lucid Hearing App

When your hearing aids are powered on, you can connect to the Lucid Hearing App to customize your hearing experience. You’ll find it in Google Play.


For ease, you can use your phone camera to scan this code to download the Lucid Hearing app for Android.




STEP 3: Pairing/Connecting To App

Open the app and follow these steps:

Once your app is downloaded, open the app and your first screen will be a welcome screen. Click on Start Using Lucid Hearing and begin the sign-up process.

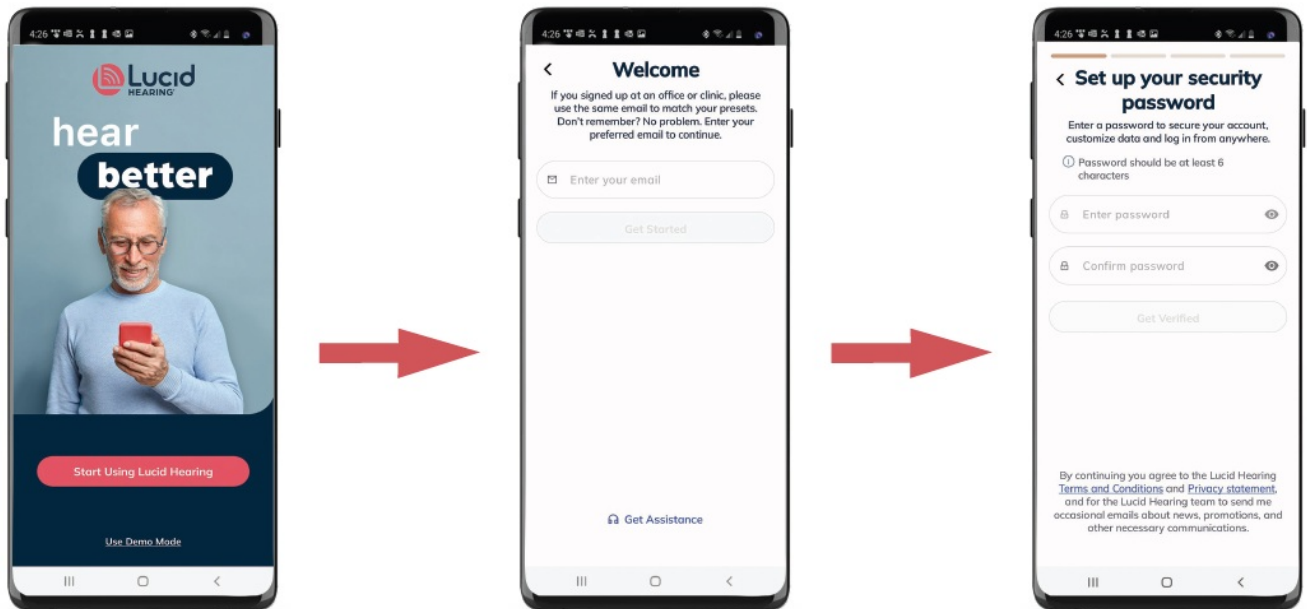


Customers with a current Lucid Hearing account: Use the email associated with your profile. If you already have a password set up, log in and enter that password. If you do not have a password currently set up, then create a new password to protect your account.

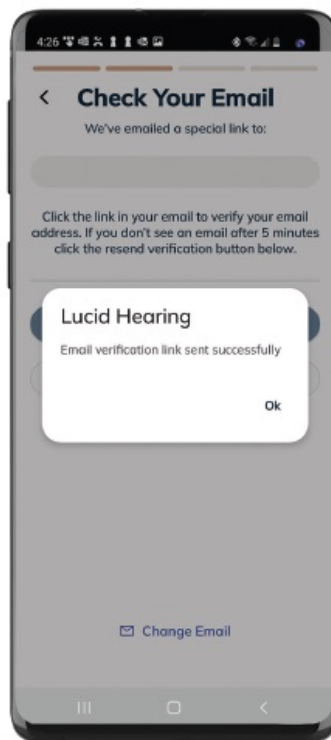


Customers without a current Lucid Hearing account: If this is your first time as a customer, use any email you prefer. After creating a password, the system will generate an email to confirm your account. Check your junk mail if you don't see it.

STEP 3: Pairing/Connecting To App (Continued)

Once you've received the email and clicked on the link to confirm, you're ready to create your account and start navigating the app.

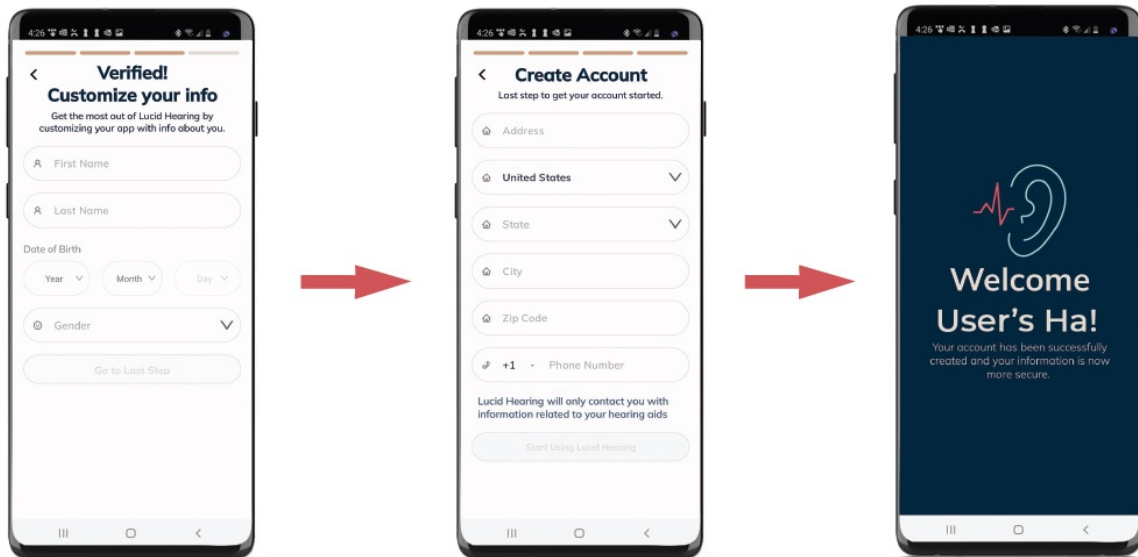


Once complete, you'll be greeted by a welcome screen. Within the Lucid Hearing app, your hearing aids will appear on the Pair Your Device screen.

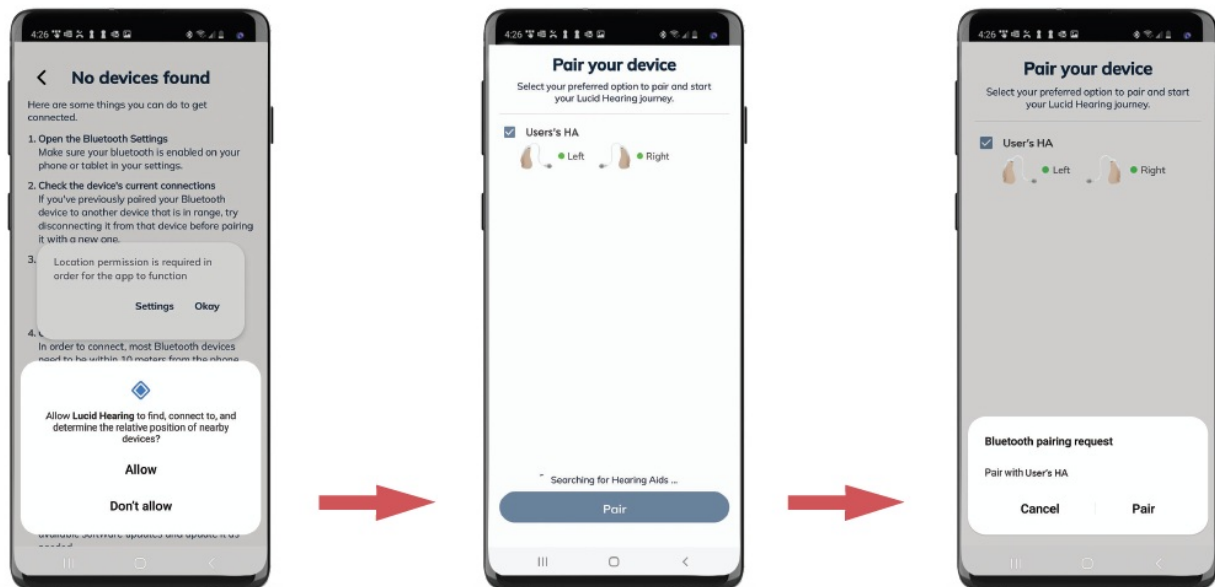


Pairing: Your hearing aids should be charged and taken out of their case (if applicable) to power on. Accept the app permissions to find nearby devices. Check the box, then click Pair at the bottom of the screen. Click Pair on the pairing request dialog if prompted.

The Lucid App Volume control:

After pairing your hearing aids, a prompt will ask you to insert the devices into the charging case (if applicable) and then remove them. Next, the Volume screen will allow you to adjust the volume of both hearing aids together.

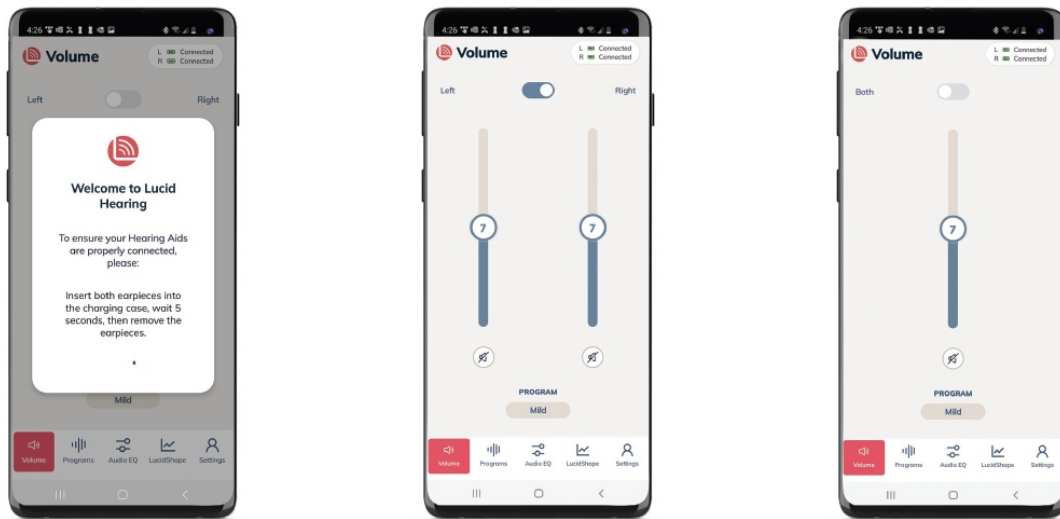


To adjust independently, toggle Both switch off. Slide the bar up and down to control volume or click to mute your aids if needed.

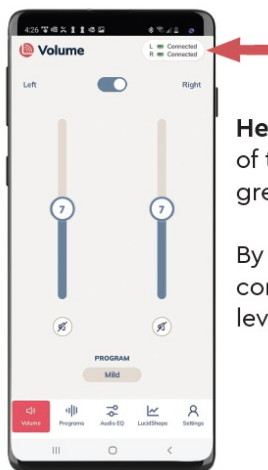

Hearing Aid Status: In the top right corner of the **Volume** screen you will see the green and white battery icon box.


By clicking on this box, it will show your connected hearing aids and their battery level charge. To exit the screen, click **Close**.

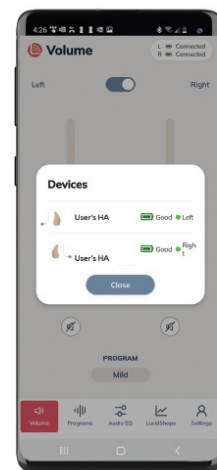

Hearing Aid Status: In the top right corner of the Volume screen you will see the green and white battery icon box. By clicking on this box, it will show your connected hearing aids and their battery level charge. To exit the screen, click Close.


Programs

On the Programs tab, you will select the desired listening profile for your hearing aids. You can select the same program for both, or select a unique setting for each individual side.

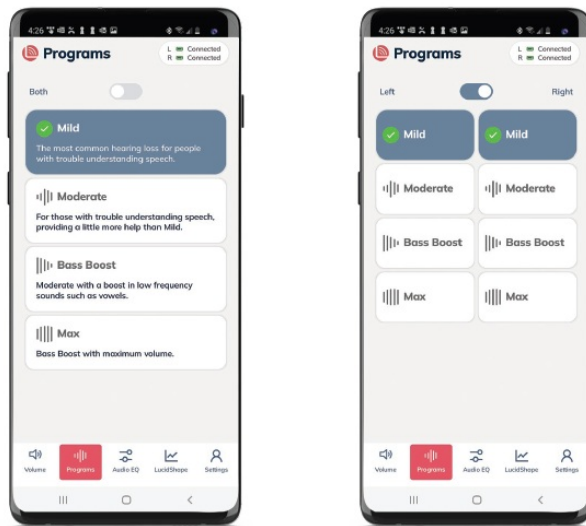

To learn more about each program, visit the instruction manual included with your device.

Both settings will appear when both hearing aids are set to the same program and disappear when unique programs are selected.

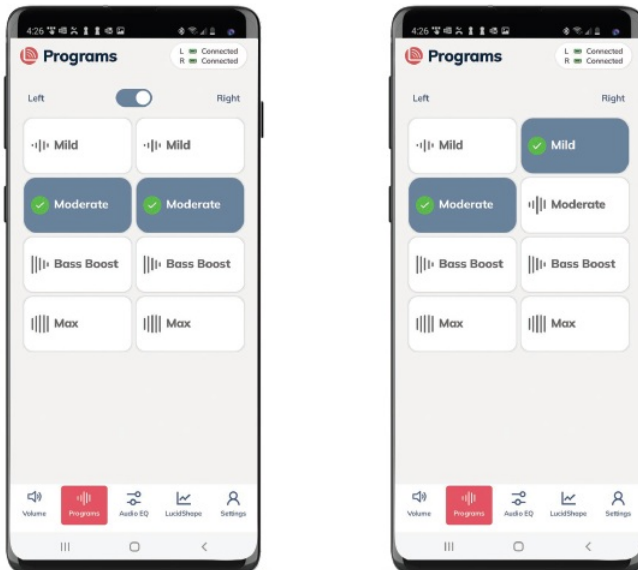


Note: program names will vary depending on product.

The Programs tab will allow you to change between listening settings pre-programmed for your convenience. Explore programs for both aids or toggle to allow independent control of the left and right aid if needed.


Audio EQ:

The Audio EQ offers two very useful features. Explore Audio Mix and the Audio Equalizer sections below to learn how to customize your listening experience.

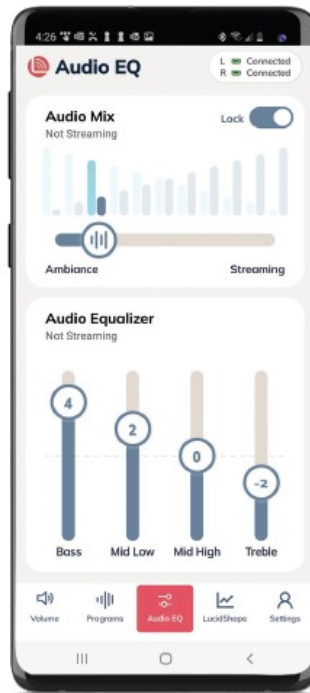

Audio Mix


The Audio Mix allows you to balance Ambiance with Streaming volume in real time. Drag the switch to the right to hear more of what you're streaming and less ambient background noise, or drag it further to the left to allow more environmental awareness and less audio signal.

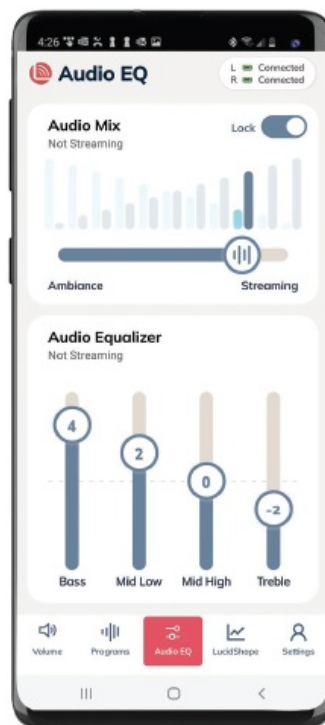

The Audio Equalizer allows you to manipulate four independent frequency ranges (Bass, Mid Low, Mid High, Treble) designed to customize or fine-tune your individual streaming preference. You achieve this by simply touching the slide bar and sliding it to the desired position.

Unlock/Lock

To adjust the audio while media is paused or stopped, unlock by sliding the toggle switch next to the Audio Mix. Your changes will be made once you resume listening.

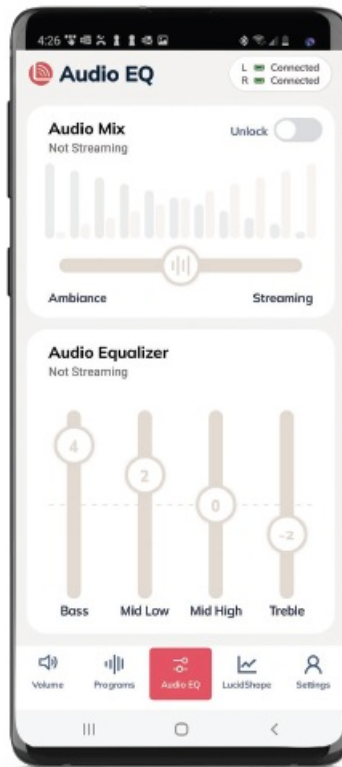


LucidShape

LucidShape gives you the ability to adjust specific frequencies according to your listening preferences and needs. Here we will demonstrate how to use LucidShape to adjust your hearing aids.

First, select which hearing aid you would like to adjust at the top of the screen. You can choose to adjust the left and right individually, or both together. LucidShape automatically defaults to both.



Live Environment mode: listen to the environment around you as you begin to adjust your listening program.

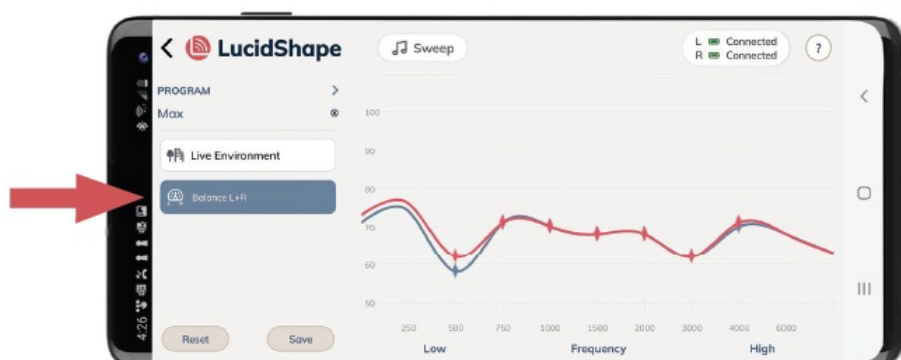

Balance L+R mode: both left and right hearing aids will play a tone simultaneously to help you adjust the balance between the hearing aids.

Note that when you open Lucidshape you are automatically adjusting the program you are currently in, but you can manually select to a different program.



Sweep button: Within the L+R screen, press Sweep to run through all the frequencies and play tones from low to high. Listen to each frequency and make adjustments to volume as needed. You can also use the Balance L+R screen to touch an individual frequency, which will play that tone as you adjust the volume at that frequency.




To increase volume and sensitivity, just press, hold, and drag upwards.




To decrease volume and sensitivity, drag down.

Once you are satisfied with the sound, click Save at the bottom left of the screen. To start over, click Reset and your hearing aids will be reset to their original setting.

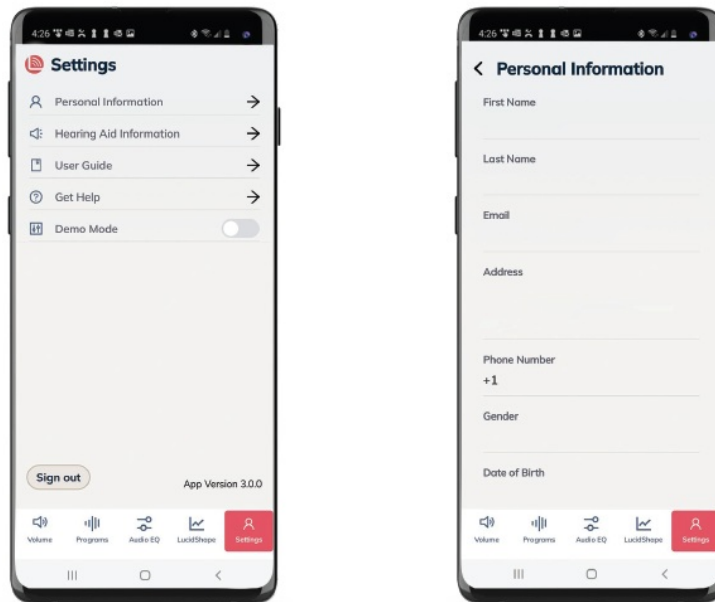

Settings

Here you will find information about your hearing aids, a helpful User Guide for the Lucid Hearing app, LucidShape and support.

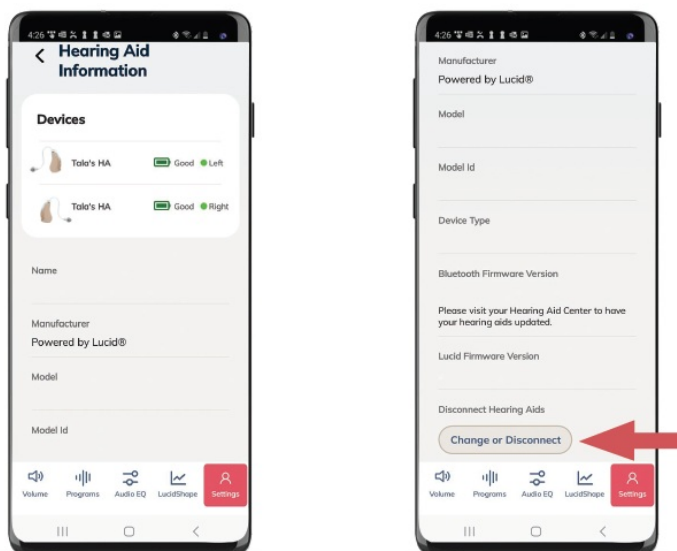


To disconnect, scroll to the bottom of the screen and select **Change or Disconnect**. This will unpair your current hearing aids with the app.


Personal Information: Access this tab to update your information.

Within the Hearing Aid Information screen, you can view firmware information, view the model of your hearing aids, as well as disconnect your hearing aids from the app.

Settings (Continued)

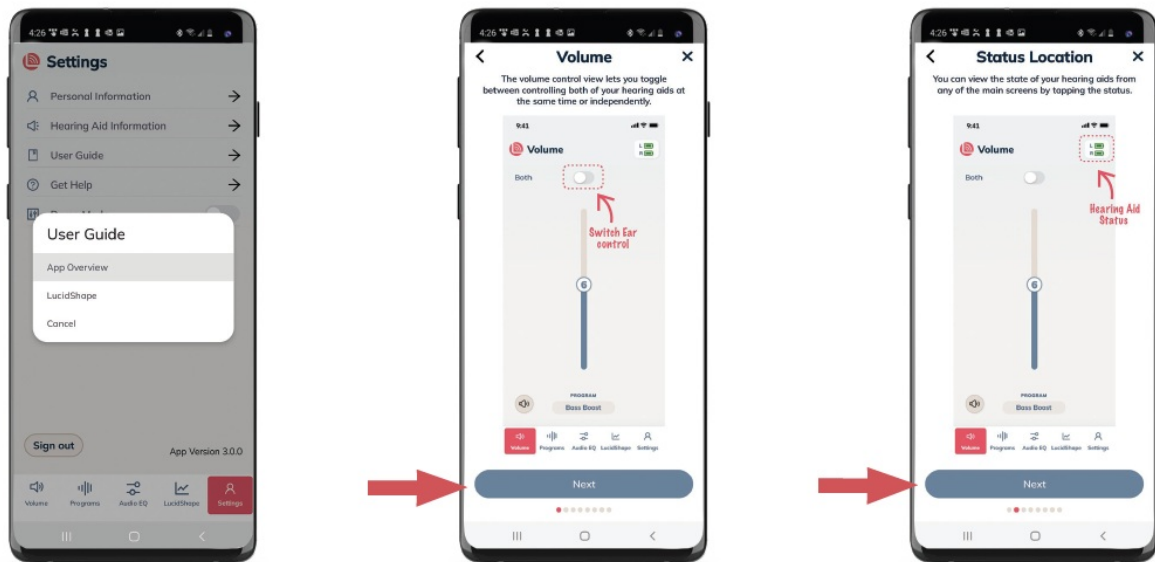

Inside the User Guide, select Overview to access useful slides by clicking Next to walk you through the app.

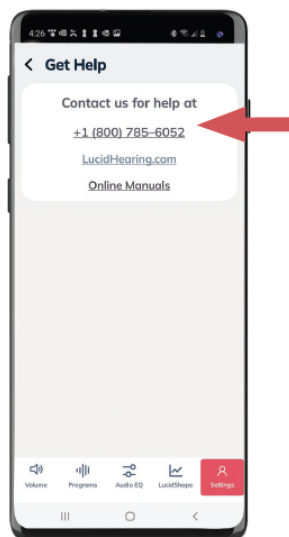

The **Get Help** tab will connect you with our website as well as our customer support number.

Click on the phone number to reach our representatives and find business hours.

FREQUENTLY ASKED QUESTIONS

Q: I'm struggling to use the app. How can I get a representative to help me?

A: You can get access to support in three ways:

1. Lucidhearing.com for Chat Support MON-SAT 8AM–6PM CST
2. Email: Support@lucidhearing.com
3. Call: (800) 785–6052 MON-SAT 8AM–6PM CST

Q: I bought Over-The-Counter hearing aids but still have problems hearing. Why?

A: You may need a hearing exam to determine the recommended steps. You can schedule a free, in-depth examination by visiting one of our hearing centers. Find a location and schedule online at lucidhearing.com or call (800) 785–6052.

Q: I hear weird noises when I insert my hearing aid. Why?

A: When inserting your hearing aids for the first time, there can be feedback or whistling if your hearing aids are not fully inserted or you do not have a good seal. Make sure they are completely inserted.

Q: What's the difference between an Over-the-Counter (OTC) and a prescription hearing aid?

A: OTC hearing aids are designed to help people with perceived mild to moderate hearing loss and require no exam or prescription to purchase. Prescription hearing aids are suitable for anyone with perceived mild and moderate hearing loss as well, but also for those with severe or profound hearing loss. They do require a hearing test and prescription provided by a licensed hearing professional. The best way to determine which type of hearing aid is right for you is to have a hearing test to accurately determine your level of hearing impairment.

Q: Where can I get my hearing tested?

A: Lucid Hearing is proud to offer free hearing tests for people age 19+. We have nearly 500 hearing aid centers nationwide, mainly inside Sam's Clubs. Go to lucidhearing.com to find a center nearest you. You can also call (800) 785-6052 to schedule a hearing test.


- **Mild hearing loss:** If one-on-one conversations are fine, but you're having difficulty understanding some words when there's a lot of background noise, you may have mild hearing loss. Technically speaking, it's defined as having hearing loss between 26 – 40 decibels.
- **Moderate hearing loss:** At this level, you're asking people to repeat themselves a lot during conversations – in person and on the telephone. People with this degree of hearing loss cannot hear sounds lower than 41 – 55 decibels.

Prescription hearing aids are your best solution if you have severe or profound hearing loss:



- **Severe hearing loss:** If you can't hear people speaking without the use of a hearing aid or other amplification, or you tend to rely on social cues to understand the conversation, you likely have severe hearing loss. People with this degree of hearing loss cannot hear sounds lower than 71 – 90 decibels.
- **Profound hearing loss:** If you have profound hearing loss, you can only hear extremely loud conversations or sounds – and even then, it's difficult to understand without a hearing aid or cochlear implant. People with this degree of hearing loss cannot hear sounds lower than 91+ decibels.

©2023 Lucid Hearing Holding Company, LLC. Lucid Hearing is a registered trademark of Lucid Hearing Holding Company, LLC. All rights reserved. LBL-1063 Rev 01

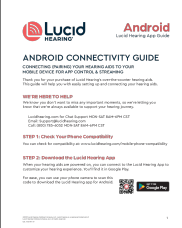Documents / Resources

	Lucid HEARING LBL-1063 Hearing App [pdf] User Guide LBL-1063 Hearing App, Hearing App, App
---	---

References

-  [Home | Lucid Hearing - Lucid Hearing](#)
-  [Mobile Phone Compatibility - iPhone and Android list - Lucid Hearing](#)