

For further assistance, please
visit [Tenovi.com/support](https://tenovi.com/support).


tenovi

© 2025 Tenovi Co.
All rights reserved. Republication, duplication or modification
in whole or in part is prohibited without prior written permission
from Tenovi. All trademarks are owned by or licensed to
Tenovi, Co. unless otherwise noted.
Revision 05.01.2024

1

Remove Pull Tab


Remove **pull tab** from the battery compartment to power on the device.

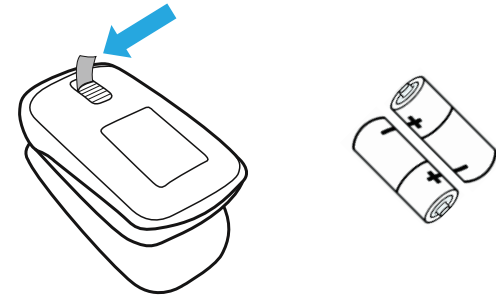

2

Insert Finger Into Pulse Ox


Insert one finger completely into the measurement section of the Oximeter with your fingernail **pointing up** towards the screen.

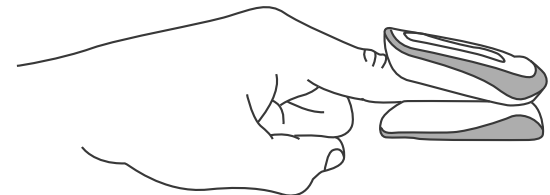

Press Button

3

Press the **power button** on the oximeter. Keep your finger in the oximeter for 20 seconds

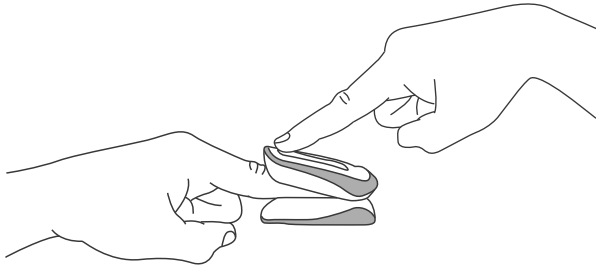

Measurement Taken

5


"**Success**" will display when the reading has been transmitted. When "**FAILED**" is shown, the reading will be saved and the device will try and send again on your next measurement.

Success

Wait 20 seconds

4

Once the measurement has been completed, the device will then lock the reading and begin to transmit the data over the cell network.

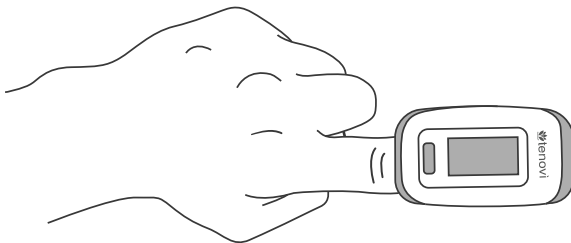

For further assistance, please visit [Tenovi.com/support](https://tenovi.com/support).

tenovi