

BRAUN


Nasal aspirator 1
브라운
코 흡인기 1

Quick-start
guide


간단
사용설명서



- 1 Insert batteries.
배터리를 삽입합니다.

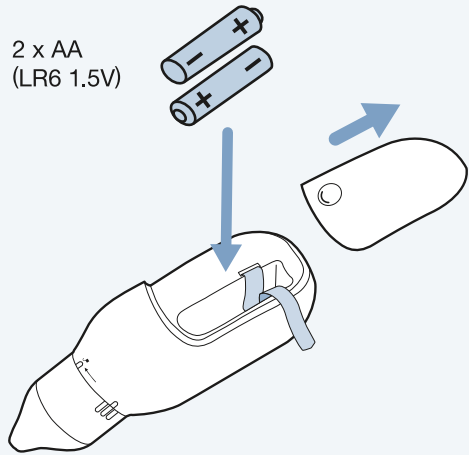

- 2 Select tip & press in place.
팁을 선택하여 끼워 넣습니다.

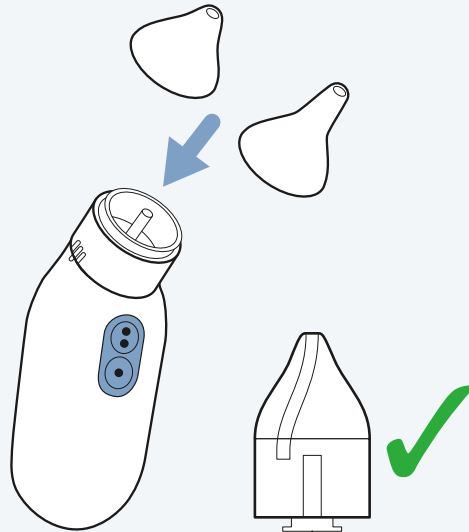

- 3
-
- High
강
Low
약
- A diagram showing a hand holding the nasal aspirator and inserting it into a baby's nose. A blue arrow points to the device, and a legend indicates that the top button is for 'High' (강) and the bottom button is for 'Low' (약).



Do not overfill
reservoir.
용기를 과도하게
채우지 마십시오.







Cleaning instructions:

Twist to lock/unlock reservoir.

세척 방법:

용기를 잡고 돌려줌으로써 탈착합니다.

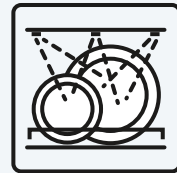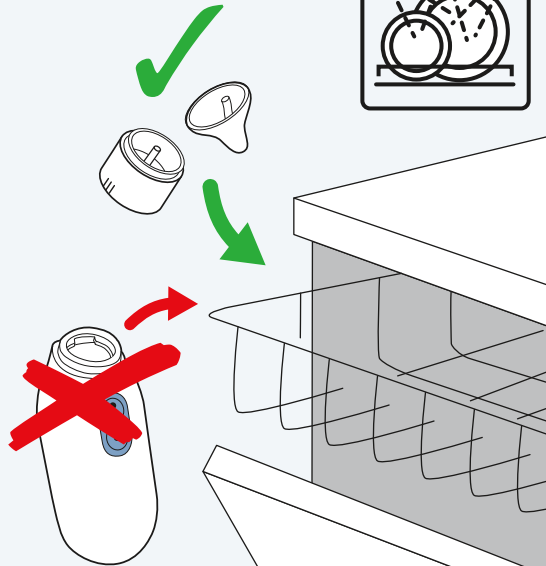

For additional information visit:

www.braunnasalaspirator.com

더 자세한 내용은 www.braunnasalaspirator.com에서 확인하실 수 있습니다.

Certain trademarks used under license from The Procter & Gamble Company or its affiliates.

© 2020 All rights reserved. Made in China.

BNA100

09DEC19

P/N: A002264R0