

# WIDEX SMARTRIC PORTABLE CHARGER (WPP401)

## QUICK GUIDE

The Widex SmartRIC™ is a fully rechargeable hearing aid, using a sleek and stylish portable charger for charging on the go. A fully charged Widex portable charger allows for five full hearing aid charges, providing the wearer over a week of freedom.

### CHARGING THE WIDEX SMARTRIC HEARING AIDS



From 0% battery status, a full hearing aid charge will take about **four hours**.



If the hearing aid battery is less than 40%, **fast charging** is activated and a **30-minute charge** will provide up to **eight hours of hearing aid use**.



#### Hearing Aid Light Indicators:

	Solid green	The hearing aid is fully charged.
	Flashing green	The hearing aid is charging.

Note: If the charger is not connected to a power source, short press the on/off button on the bottom of the charger to check the battery status. The light indicators will turn off after 12 seconds.

#### Other light indications that may appear:

	Fading green	After a long press of the on/off button – charger is turning off.
	Flashing orange	When charging via Qi-certified charging pad, high hearing aid temperature. Charging will continue when temperature reduces.
	Flashing red	When charging via USB-C, high hearing aid temperature. Charging will continue when temperature reduces.

**i** Each hearing aid will charge independently, e.g. if one hearing aid is fully charged and the other is not, only the hearing aid with low battery will charge. Hearing aids will charge with the charger lid open.



SOUND LIKE NO OTHER

# WIDEX SMARTRIC PORTABLE CHARGER (WPP401)

## QUICK GUIDE

### CHARGING THE PORTABLE CHARGER



Fully charging the charger will take four hours (via USB-C) or 6 hours (via Qi-certified charging pad).

#### Light indicator

There are three LED light indicators at the bottom of the charger, showing power and battery status of the charger.



#### When the charger is charging:

#### Charger not connected to a power source:

● ● ●	● ● ●	Three + full hearing aid charges available.
● ● ●	● ● ○	Minimum two full hearing aid charges available.
● ● ○	● ○ ○	Minimum one full hearing aid charge available.
● ○ ○	● ○ ○	Less than one hearing aid charge left.
	● ○ ○	Internal battery is discharged

Note: If the charger is not connected to a power source, short press the on/off button to check the battery status. The light indicators will turn off after 12 seconds.

#### Other light indications that may appear:

● ● ●	Fading green	After a long press of the on/off button – charger is turning off.
● ● ●	Flashing orange	High temperature, charging paused (Qi-certified pad charging). Charging will continue when temperature reduces.
● ● ●	Flashing red	If using a Qi pad for charging, a charger error may have occurred, and the voltage is out of range. Remove the charger from Qi pad and try again. The internal battery temperature is too high or too low, charging stopped. Charging will continue when charger returns to room temperature.

For more information on charger use, please refer to the User Instructions booklet provided with the product.

