

OMRON

Automatic Blood Pressure Monitor

Model HEM-7140T1

Instruction Manual



5637561-3C  
Issue Date: 2022-05-26

Table of Contents

Introduction

Important Safety Information

1. Know Your Monitor

2. Preparation

3. Using Your Monitor without a Smart Device

4. Using Your Monitor with a Smart Device

5. Using Memory Function

6. Other Settings

7. Error Messages and Troubleshooting

8. Maintenance

9. Optional Medical Accessories

10. Other Optional Parts

11. Specifications

12. Guidance and Manufacturer's Declaration

Introduction

Thank you for purchasing the OMRON Automatic Blood Pressure Monitor. This blood pressure monitor uses the oscillometric method of blood pressure measurement. This means this monitor detects your blood movement through your brachial artery and converts the movements into a digital reading.

**Safety Instructions**

This instruction manual provides you with important information about the OMRON Automatic Blood Pressure Monitor. To ensure the safe and proper use of this monitor, READ and UNDERSTAND all of these instructions. **If you do not understand these instructions or have any questions, contact your local OMRON representative before attempting to use this monitor. For specific information about your own blood pressure, consult with your physician.**

**Intended Use**

This device is a digital monitor intended for use in measuring blood pressure and pulse rate in the adult patient population. It is mainly designed for general household use.

**Receiving and Inspection**

Remove this monitor from the packaging and inspect for damage. If this monitor is damaged, DO NOT USE and consult with your local OMRON representative.

**Important Safety Information**

Read the Important Safety Information in this instruction manual before using this monitor. Follow this instruction manual thoroughly for your safety.

Keep for future reference. For specific information about your own blood pressure, CONSULT WITH YOUR PHYSICIAN.

**⚠Caution** Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

- DO NOT use this monitor on infants, toddlers, children or persons who cannot express themselves.
- DO NOT adjust medication based on readings from this blood pressure monitor. Take medication as prescribed by your physician. ONLY a physician is qualified to diagnose and treat high blood pressure.
- DO NOT use this monitor on an injured arm or an arm under medical treatment.
- DO NOT apply the arm cuff on your arm while on an intravenous drip or blood transfusion.
- DO NOT use this monitor in areas containing high frequency (HF) surgical equipment, magnetic resonance imaging (MRI) equipment, computerized tomography (CT) scanners. This may result in incorrect operation of the monitor and/or cause an inaccurate reading.
- DO NOT use this monitor in oxygen rich environments or near flammable gas.
- Consult with your physician before using this monitor if you have common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation; arterial sclerosis; poor perfusion; diabetes; pregnancy; pre-eclampsia or renal disease. NOTE that any of these conditions in addition to patient motion, trembling, or shivering may affect the measurement reading.
- NEVER diagnose or treat yourself based on your readings. ALWAYS consult with your physician.
- To help avoid strangulation, keep the air tube and AC adapter cable away from infants, toddlers and children.
- This product contains small parts that may cause a choking hazard if swallowed by infants, toddlers and children.

Data Transmission

- This product emits radio frequencies (RF) in the 2.4 GHz band. DO NOT use this product in locations where RF is restricted, such as on an aircraft or in hospitals. Turn off the Bluetooth® feature in this monitor, remove batteries and/or unplug the AC adapter when in RF restricted areas.

AC Adapter (optional accessory) Handling and Usage

- DO NOT use the AC adapter if this monitor or the AC adapter cable is damaged. If this monitor or the cable is damaged, turn off the power and unplug the AC adapter immediately.
- Plug the AC adapter into the appropriate voltage outlet. DO NOT use in a multi-outlet plug.
- NEVER plug in or unplug the AC adapter from the electric outlet with wet hands.
- DO NOT disassemble or attempt to repair the AC adapter.


•Periodically check batteries to ensure they are in good working condition.

### 1. Know Your Monitor

**1.1 Contents/Product Includes**  
Monitor, arm cuff, 4 "AA" batteries, instruction manual, setup instructions, warranty card


### 1.2 Monitor and Arm Cuff

Front of the monitor

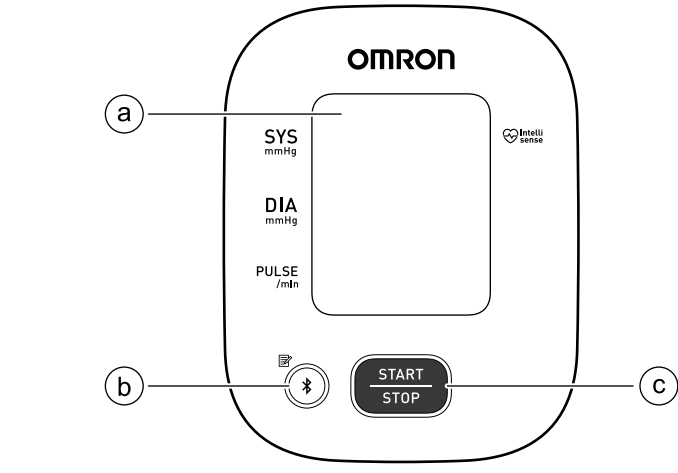

- Ⓐ Display
- Ⓑ [Connection] button (Memory)
- Ⓒ [START/STOP] button


Left side, back and bottom of the monitor

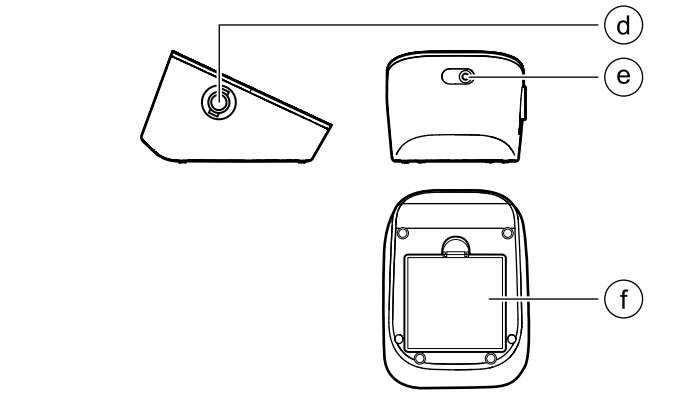


- Ⓐ Air jack
- Ⓔ AC adapter jack (for optional AC adapter)
- Ⓕ Battery compartment

**Arm cuff**  
(Upper arm circumference: 22 - 32 cm)

- Ⓓ Air plug
- Ⓗ Air tube

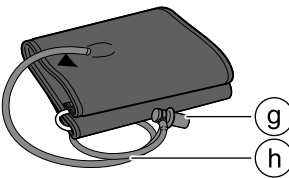

### 1.3 Display and Symbols

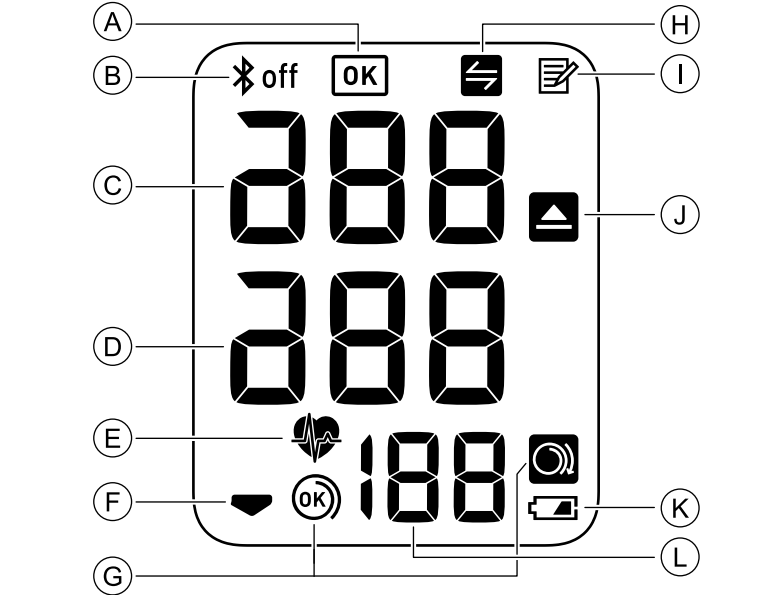

|   |                                                                                                                                                                                                                                                                                                                                                           |
|---|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Ⓐ | <b>OK symbol</b><br>Flashes when your monitor is connected to your smart device successfully.                                                                                                                                                                                                                                                             |
| Ⓑ | <b>Bluetooth ON symbol</b><br>Appears when your readings are being transferred.                                                                                                                                                                                                                                                                           |
| Ⓒ | <b>Bluetooth OFF symbol</b><br>Appears when Bluetooth is disabled.                                                                                                                                                                                                                                                                                        |
| Ⓓ | <b>Systolic blood pressure reading</b>                                                                                                                                                                                                                                                                                                                    |
| Ⓔ | <b>Diastolic blood pressure reading</b>                                                                                                                                                                                                                                                                                                                   |
| Ⓕ | <b>Heartbeat symbol</b><br>Flashes while taking a measurement.                                                                                                                                                                                                                                                                                            |
| Ⓖ | <b>Deflation symbol</b><br>Appears during cuff deflation.                                                                                                                                                                                                                                                                                                 |
| Ⓖ | <b>Cuff wrap guide symbol (OK)</b><br>Appears if the arm cuff is wrapped around the arm correctly while taking a measurement. It also appears when viewing past readings.                                                                                                                                                                                 |
| Ⓖ | <b>Cuff wrap guide symbol (loose)</b><br>Appears if the arm cuff is not wrapped around the arm correctly while taking a measurement. It also appears when viewing past readings.                                                                                                                                                                          |
| Ⓗ | <b>Sync symbol</b><br>Flashes/appears when your data needs to be transferred because the stored memory is either almost, or completely full. Once you pair your monitor with your smart device, transfer your readings immediately before the monitor deletes the oldest reading. Up to 14 readings can be stored in the internal memory of your monitor. |
| Ⓘ | <b>Memory symbol</b><br>Appears when viewing readings stored in the memory.                                                                                                                                                                                                                                                                               |
| Ⓙ | <b>Hypertension symbol</b><br>Appears if your systolic blood pressure is 135 mmHg or above and/or the diastolic blood pressure is 85 mmHg or above.                                                                                                                                                                                                       |
| Ⓚ | <b>Low battery symbol</b><br>Flashes when batteries are low.                                                                                                                                                                                                                                                                                              |
| Ⓚ | <b>Depleted battery symbol</b><br>Appears when batteries are depleted.                                                                                                                                                                                                                                                                                    |
| Ⓛ | <b>Pulse display / Memory number</b><br>Pulse rate appears after the measurement.<br>When pressing the  button, the memory number appears for approximately 1.5 seconds before the pulse rate shows again on the display.                                                                                                                                 |

| 2018 ESH/ESC* Guidelines for the management of arterial hypertension |            |            |
|----------------------------------------------------------------------|------------|------------|
| Definitions of hypertension by office and home blood pressure levels |            |            |
|                                                                      | Office     | Home       |
| Systolic Blood Pressure                                              | ≥ 140 mmHg | ≥ 135 mmHg |
| Diastolic Blood Pressure                                             | ≥ 90 mmHg  | ≥ 85 mmHg  |

These ranges are from statistical values for blood pressure.

\* European Society of Hypertension (ESH) and European Society of Cardiology (ESC).

**⚠Caution**

- NEVER diagnose or treat yourself based on your readings. ALWAYS consult with your physician.

### 2. Preparation

#### 2.1 Installing Batteries

1. Push down on the hook of the battery cover and pull upward.
2. Insert 4 "AA" batteries as indicated in the battery compartment.
3. Close the battery cover.

**Note**

- When the " " symbol flashes on the display, it is recommended you replace batteries.
- To replace batteries, turn your monitor off and remove all batteries. Then replace with 4 new batteries at the same time.
- Replacing batteries will not delete previous readings.
- The supplied batteries may have a shorter life span than new batteries.
- Disposal of used batteries should be carried out in accordance with local regulations.

**2.2 Blood Pressure Measurement Tips**

To help ensure an accurate reading, follow these directions:

- Stress raises blood pressure. Avoid taking measurements during stressful times.
- Measurements should be taken in a quiet place.
- It is important to take measurements at the same times each day. Taking measurements in the morning and in the evening is recommended.
- Remember to have a record of your blood pressure and pulse readings for your physician. A single measurement does not provide an accurate indication of your true blood pressure. You need to take and record several readings over a period of time.

**⚠Caution**

- Avoid bathing, drinking alcohol or caffeine, smoking, exercising and eating for at least 30 minutes before taking a measurement.
- Rest for at least 5 minutes before taking a measurement.

#### 2.3 Applying the Arm Cuff

**Note**

- The following steps are for applying the arm cuff to your left arm. When you take a measurement on your right arm, follow the cuff wrap instructions for a right arm use mentioned under "To take a measurement on your right arm" located at the end of this sub-section.
- The blood pressure can differ between the right arm and the left arm, and the measured blood pressure readings can be different. OMRON recommends to always use the same arm for measurement. If the readings between both arms differ substantially, check with your physician to determine as to which arm to use for your measurements.

**⚠Caution**

- Remove tight-fitting or thick clothing from your arm while taking a measurement.

1. Plug the arm cuff into your monitor by inserting the air plug into the air jack securely until it clicks.
2. Place your hand through the cuff loop. Pull the cuff until it reaches your upper left arm.
3. Make sure that the air tube is positioned on the inside of your arm and wrap the cuff securely, so it can not move around your arm.

**To take a measurement on your right arm**

When you take a measurement on the right arm, the air tube should be at the side of your elbow. Be careful not to rest your arm on the air tube.

### 2.4 Sitting Correctly

To take a measurement, you need to be relaxed and comfortably seated in a room with a comfortable temperature. Place your arm on the table.

- Sit in a comfortable chair with your back and arm supported.
- Keep your feet flat and your legs uncrossed.
- The arm cuff should be placed on your arm at the same level as your heart, with the arm resting comfortably on a table.



### 3. Using Your Monitor without a Smart Device

**3.1 Taking a Measurement**

You can use your monitor without pairing to a smart device. To pair your monitor with a smart device, refer to sub-section 4.1.

**Note**

- To stop the measurement, press the [START/STOP] button once to deflate the arm cuff.

**⚠Caution**

- DO NOT use this monitor with other medical electrical (ME) equipment simultaneously. This may result in incorrect operation of the devices and/or cause an inaccurate reading.
- Remain still and DO NOT talk while taking a measurement.

1. Press the [START/STOP] button.  
All symbols appear on the display before starting the measurement.
2. Remain still and do not talk until the entire measurement process is complete.


The " " symbol flashes at every heartbeat.

**Note**

- The " " symbol appears if the arm cuff is wrapped around the arm correctly.
- When the " " symbol is displayed, the arm cuff is not applied correctly. Press the [START/STOP] button to turn your monitor off, then apply the cuff correctly.

After your monitor has detected your blood pressure and pulse rate, the cuff automatically deflates. Your blood pressure and pulse rate are displayed.

If either the systolic or the diastolic reading is high (refer to sub-section 1.3), the " " symbol appears.

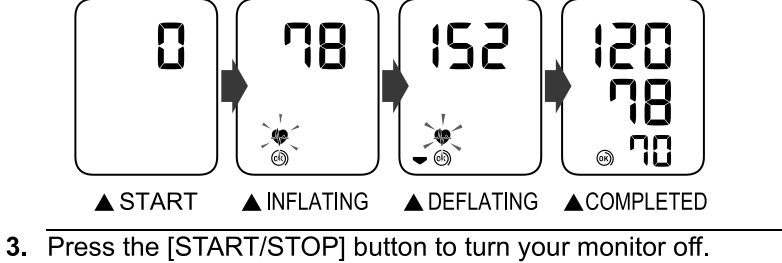

3. Press the [START/STOP] button to turn your monitor off.
- Note**
- Your monitor automatically turns off after 2 minutes.
  - Wait 2 - 3 minutes between measurements. The wait time allows the arteries to decompress and return to their pre-measurement state. You may need to increase the wait time depending on your individual physiological characteristics.

**■ If your systolic pressure is more than 210 mmHg**

If your systolic pressure is more than 210 mmHg, take a measurement manually.

After the arm cuff starts to inflate, press and hold the [START/STOP] button until the monitor inflates 30 to 40 mmHg higher than your expected systolic pressure.

**Note**

- Do not inflate above 299 mmHg.

**⚠Caution**

- Inflating to a higher pressure than necessary may result in bruising of the arm where the cuff is applied.

### 4. Using Your Monitor with a Smart Device

#### 4.1 Pairing Your Monitor with a Smart Device

Review the list of compatible smart devices at [www.omronconnect.com/devices/](http://www.omronconnect.com/devices/)

1. Enable Bluetooth on your smart device.
2. Download and install the "OMRON connect" app onto your smart device.
3. Open the app and follow the pairing instructions shown on your smart device.
4. Confirm that your monitor is connected successfully.

When your monitor is connected successfully to your smart device, the OK symbol appears.

**Note**

- If the "Err" symbol appears, follow the instructions in the "OMRON connect" app.
- Your monitor automatically turns off after 10 seconds.
- Be aware that OMRON will not be responsible for the loss of data and/or information in the app.
- "OMRON connect" is the only app that we recommend to use with your monitor to transfer data correctly.

5. Press the [START/STOP] button to turn your monitor off.

**Note**

- If the "Err" symbol appears, follow the instructions in the "OMRON connect" app.
- Your monitor automatically turns off after 10 seconds.
- Be aware that OMRON will not be responsible for the loss of data and/or information in the app.
- "OMRON connect" is the only app that we recommend to use with your monitor to transfer data correctly.

**4.2 Taking a Measurement**

Refer to sub-section 3.1 to start a measurement. Open the app on your smart device and follow the instructions.

**Note**

- Make sure that your monitor is within 5 m of your smart device.

**■ Transferring Data Automatically**

When Bluetooth is enabled, the readings will be transferred to your smart device after taking a measurement.

**4.3 Disabling/Enabling Bluetooth**

Disable Bluetooth in your monitor in the following areas where use of wireless equipment is prohibited.

- On aircraft • In hospitals • While abroad

Bluetooth is enabled as default.

1. When your monitor is off, press and hold the button for more than 10 seconds.  
"oFF" flashes 3 times.

2. Press the [START/STOP] button to turn your monitor off.

**Note**


- To enable the Bluetooth, press and hold the button for more than 3 seconds. "on" appears on the display.
- Your monitor automatically turns off 10 seconds after the completion.
- When Bluetooth is disabled, the " " symbol appears.

5. Using Memory Function

Your monitor automatically stores up to 14 readings.

5.1 Viewing the Readings Stored in Memory


1. Press the  button.

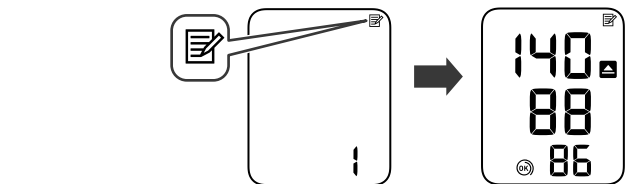

The Memory number appears for 1.5 seconds before the pulse rate is displayed. The most recent reading stored in the memory will be numbered as "1".

Note

- If the reading is high (refer to sub-section 1.3), the  symbol appears.
- If there are no readings stored in the memory, the screen to the right is displayed.
- If the memory is full, new readings will replace the old readings, starting with the oldest reading.

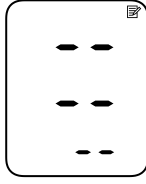

2. Press the  button repeatedly to scroll through the previous readings stored in the memory.

6. Other Settings

6.1 Restoring Your Monitor to the Default Settings

To delete all the information stored in your monitor, follow the instructions below. Make sure that your monitor is turned off.


1. Press the  button.

After a memory number appears, the latest reading will appear with a past reading.

2. While holding the


 button down, press and hold the [START/STOP] button for more than 4 seconds.

The display to the right flashes.

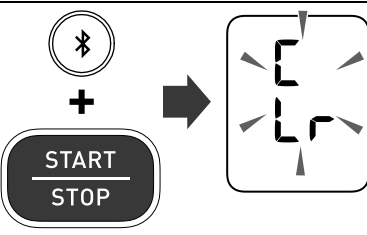

3. While holding the

 button down again, press and hold the [START/STOP] button for more than 4 seconds. The display stops flashing, then your monitor is restored to the default settings.

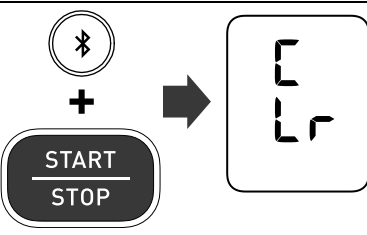



4. Press the [START/STOP] button to turn your monitor off.

Note

- Reverting to the default setting of your monitor does not delete the information in the app.
- Your monitor automatically turns off after 10 seconds.

7. Error Messages and Troubleshooting

If any of the below problems occur during measurement, check to make sure that no other electrical device is within 30 cm. If the problem persists, please refer to the table below.
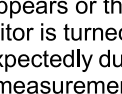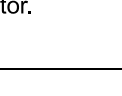
| Display/Problem                                                                                                                                                | Possible Cause                                                                        | Solution                                                                                                                                                                                                          |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <br>appears or the arm cuff does not inflate.                                 | The [START/STOP] button was pressed while the arm cuff is not applied.                | Press the [START/STOP] button again to turn the monitor off.                                                                                                                                                      |
|                                                                                                                                                                | Air plug is not completely plugged into the monitor.                                  | Insert the air plug securely.                                                                                                                                                                                     |
|                                                                                                                                                                | The arm cuff is not applied correctly.                                                | Apply the arm cuff correctly, then take another measurement. Refer to sub-section 2.3.                                                                                                                            |
| <br>appears or a measurement cannot be complete after the arm cuff inflates.  | Air is leaking from the arm cuff.                                                     | Replace the arm cuff to the new one. Refer to section 9.                                                                                                                                                          |
|                                                                                                                                                                | You move or talk during a measurement and the arm cuff does not inflate sufficiently. | Remain still and do not talk during a measurement. If "E2" appears repeatedly, inflate the arm cuff manually until the systolic pressure is 30 to 40 mmHg above your previous readings. Refer to sub-section 3.1. |
| <br>appears                                                                   | The arm cuff is inflated exceeding the maximum allowable pressure.                    | Do not touch the arm cuff and/or bend the air tube while taking a measurement. If inflating the arm cuff manually, refer to sub-section 3.1.                                                                      |
| <br>appears                                                                   | You move or talk during a measurement. Vibrations disrupt a measurement.              | Remain still and do not talk during a measurement.                                                                                                                                                                |
| <br>appears                                                                 |                                                                                       | Apply the arm cuff correctly, then take another measurement. Refer to sub-section 2.3. Remain still and sit correctly during a measurement.                                                                       |
| <br>appears                                                                 | The pulse rate is not detected correctly.                                             |                                                                                                                                                                                                                   |
| <br>does not flash during a measurement                                     |                                                                                       |                                                                                                                                                                                                                   |
| <br>appears                                                                 | The monitor has malfunctioned.                                                        | Press the [START/STOP] button again. If "Er" still appears, contact your local OMRON representative.                                                                                                              |
| <br>appears                                                                 | The monitor cannot connect to a smart device or transmit data correctly.              | Follow the instructions shown in the "OMRON connect" app. If the "Er" symbol still appears after checking the app, contact your local OMRON representative.                                                       |
| <br>flashes                                                                 | The monitor is waiting for pairing with the smart device.                             |                                                                                                                                                                                                                   |
| <br>flashes                                                                 | More than 11 readings are not transferred.                                            | Pair or transfer your readings to the "OMRON connect" app so you can keep them in memory in the app, and this error symbol disappears.                                                                            |
|                                                                                                                                                                | Your monitor is not paired or not connected with your smart device.                   |                                                                                                                                                                                                                   |
| <br>appears                                                                 | There are 14 readings in memory to be transferred                                     |                                                                                                                                                                                                                   |
| <br>flashes                                                                 | Batteries are low.                                                                    | Replacing all 4 batteries with new ones is recommended. Refer to sub-section 2.1.                                                                                                                                 |
| <br>appears or the monitor is turned off unexpectedly during a measurement. | Batteries are depleted.                                                               | Immediately replace all 4 batteries with new ones. Refer to sub-section 2.1.                                                                                                                                      |
|                                                                                                                                                                | Batteries are depleted.                                                               |                                                                                                                                                                                                                   |
| Nothing appears on the display of the monitor.                                                                                                                 | Battery polarities are not properly aligned.                                          | Check the battery installation for proper placement. Refer to sub-section 2.1.                                                                                                                                    |

| Display/Problem                       | Possible Cause                                                                                                                                                                                                                                                               | Solution |
|---------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------|
| Readings appear too high or too low.  | Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review sub-sections 2.2 - 2.4 and section 3.                                                                                |          |
| Any other communication issue occurs. | Follow the instructions shown in the smart device, or visit the "Help" section in the "OMRON connect" app for further help. If the problem still persists, contact your local OMRON representative.                                                                          |          |
| Any other problems occur.             | Press the [START/STOP] button to turn the monitor off, then press it again to take a measurement. If the problem continues, remove all batteries and wait for 30 seconds. Then re-install batteries. If the problem still persists, contact your local OMRON representative. |          |

8. Maintenance

8.1 Maintenance

To protect your monitor from damage, follow the directions below:

- Changes or modifications not approved by the manufacturer will void the user warranty.

 Caution

- DO NOT disassemble or attempt to repair this monitor or other components. This may cause an inaccurate reading.

8.2 Storage

Store your monitor and other components in a clean, safe location.

1. Remove the arm cuff from the monitor.


 Caution

- To unplug the air plug, pull on the plastic air plug at the base of the tube, not the tube itself.

2. Gently fold the air tube into the arm cuff.

Note

- Do not bend or crease the air tube excessively.

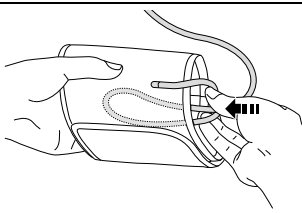

Do not store your monitor and other components:

- If your monitor and other components are wet.
- In locations exposed to extreme temperatures, humidity, direct sunlight, dust or corrosive vapors such as bleach.
- In locations exposed to vibrations or shocks.

8.3 Cleaning

- Do not use any abrasive or volatile cleaners.
- Use a soft dry cloth or a soft cloth moistened with mild (neutral) detergent to clean your monitor and arm cuff and then wipe them with a dry cloth.
- Do not wash or immerse your monitor and arm cuff or other components in water.
- Do not use gasoline, thinners or similar solvents to clean your monitor and arm cuff or other components.

8.4 Calibration and Service


- The accuracy of this blood pressure monitor has been carefully tested and is designed for a long service life.
- It is generally recommended to have the unit inspected every two years to ensure correct functioning and accuracy. Please consult your authorised OMRON dealer or the OMRON Customer Service at the address given on the packaging or attached literature.

9. Optional Medical Accessories



Note

- Do not throw the air plug away. The air plug can be applicable to the optional cuff.

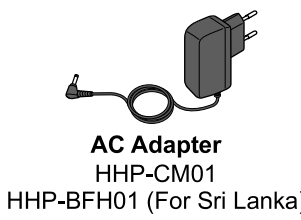


Using the AC Adapter (optional accessory)

Note

- Make sure not to place your monitor in a location where it is difficult to plug and unplug the AC adapter.

1. Insert the AC adapter plug into the AC adapter jack on the back side of your monitor.

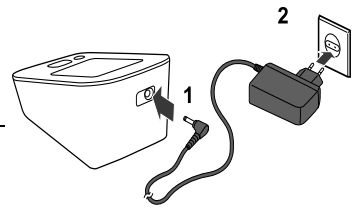


2. Plug the AC adapter into an electrical outlet.

To unplug the AC adapter, unplug the AC adapter from the electrical outlet and then remove the AC adapter plug from the monitor.

 Caution

- ONLY use the AC adapter, arm cuff, batteries and accessories specified for this monitor. Use of unsupported AC adapters, arm cuffs and batteries may damage and/or may be hazardous to this monitor.

10. Other Optional Parts

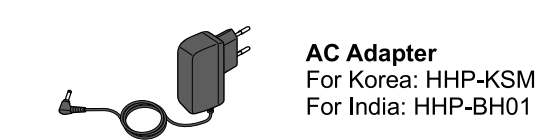

11. Specifications

|                                              |                                                                                                                                     |
|----------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------|
| Product description                          | Automatic Blood Pressure Monitor                                                                                                    |
| Model                                        | HEM-7140T1                                                                                                                          |
| Display                                      | LCD digital display                                                                                                                 |
| Cuff pressure range                          | 0 to 299 mmHg                                                                                                                       |
| Blood pressure measurement range             | SYS: 60 to 260 mmHg<br>DIA: 40 to 215 mmHg                                                                                          |
| Pulse measurement range                      | 40 to 180 beats / min.                                                                                                              |
| Accuracy                                     | Pressure: ±3 mmHg<br>Pulse: ±5% of display reading                                                                                  |
| Inflation                                    | Automatic by electric pump                                                                                                          |
| Deflation                                    | Automatic pressure release valve                                                                                                    |
| Measurement method                           | Oscillometric method                                                                                                                |
| Transmission method                          | Bluetooth® Low Energy                                                                                                               |
| Wireless communication                       | Frequency range: 2.4 GHz - 2483.5 MHz<br>Modulation: GFSK<br>Effective radiated power: < 20 dBm                                     |
| Operating mode                               | Continuous operation                                                                                                                |
| IP classification                            | Monitor: IP20<br>Optional AC adapter: IP22 for HHP-BH01 and HHP-BFH01, IP21 for other AC adapters                                   |
| Rating                                       | DC6 V 4 W                                                                                                                           |
| Power source                                 | 4 "AA" batteries 1.5 V or optional AC adapter (INPUT AC 100 - 240 V 50 - 60 Hz 0.12 - 0.065 A)                                      |
| Battery life                                 | Approximately 900 measurements (Using new alkaline batteries and included arm cuff. Depending on the type of battery and arm cuff.) |
| Durable period (Service life)                | Monitor: 5 years<br>Arm cuff: 5 years<br>Optional AC adapter: 5 years                                                               |
| Operating conditions                         | +10 to +40°C / 15 to 90% RH (non-condensing) / 800 to 1060 hPa                                                                      |
| Storage / Transport conditions               | -20 to +60°C / 10 to 90% RH (non-condensing)                                                                                        |
| Contents                                     | Monitor, arm cuff, 4 "AA" batteries, instruction manual, setup instructions, warranty card                                          |
| Protection against electric shock            | Internally powered ME equipment (when using only batteries)<br>Class II ME equipment (optional AC adapter)                          |
| Weight (not including batteries)             | Monitor: approximately 250 g<br>Arm cuff: approximately 110 g                                                                       |
| Dimensions (approximately value)             | Monitor: 103 mm (W) × 82 mm (H) × 140 mm (L)<br>Arm cuff: 146 mm × 466 mm (air tube: 610 mm)                                        |
| Cuff circumference applicable to the monitor | 17 to 42 cm (included arm cuff: 22 to 32 cm)                                                                                        |

|              |                          |
|--------------|--------------------------|
| Memory       | Stores up to 14 readings |
| Applied part | Type BF (arm cuff)       |

Note

- These specifications are subject to change without notice.
- This monitor is clinically investigated according to the requirements of EN ISO 81060-2:2014 and complies with EN ISO 81060-2:2014 and EN ISO 81060-2:2019+A1:2020. In the clinical validation study, K5 was used on 85 subjects for determination of diastolic blood pressure.
- IP classification is degrees of protection provided by enclosures in accordance with IEC 60529. This monitor and optional AC adapter are protected against solid foreign objects of 12.5 mm diameter and greater such as a finger. The optional AC adapters (except for HHP-BH01 and HHP-BFH01) are protected against vertically falling water drops which may cause issues during a normal operation. The optional AC adapter HHP-BH01 and HHP-BFH01 are protected against oblique falling water drops which may cause issues during a normal operation.
- Operation mode is classification in accordance with IEC 60601-1.






About a wireless communication interference



This product operates in an unlicensed ISM band at 2.4 GHz. In the event this product is used near other wireless devices such as microwave and wireless LAN, which operate on the same frequency band as this product, there is a possibility that interference may occur. If interference occurs, stop the operation of the other devices or relocate this product away from other wireless devices before attempting to use it.

12. Guidance and Manufacturer's Declaration

CE0197

- This blood pressure monitor is designed according to the European Standard EN1060, Non-invasive sphygmomanometers Part 1: General Requirements and Part 3: Supplementary requirements for electromechanical blood pressure measuring systems.
- Hereby, OMRON HEALTHCARE Co., Ltd., declares that the radio equipment type HEM-7140T1 is in compliance with Directive 2014/53/ EU.
- The full text of the EU declaration of conformity is available at the following internet address: <https://www.omronhealthcare-ap.com/ap/support>
- This OMRON product is produced under the strict quality system of OMRON HEALTHCARE Co., Ltd., Japan. The Core component for OMRON blood pressure monitors, which is the Pressure Sensor, is produced in Japan.
- Please report to the manufacturer and the competent authority of the Member State in which you are established about any serious incident that has occurred in relation to this device.
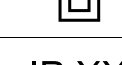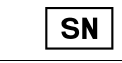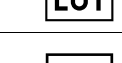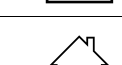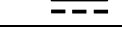
| Symbols description                                                                   |                                                                                         |
|---------------------------------------------------------------------------------------|-----------------------------------------------------------------------------------------|
|    | Applied part - Type BF<br>Degree of protection against electric shock (leakage current) |
|    | CE Marking                                                                              |
|    | Class II equipment. Protection against electric shock                                   |
| IP XX                                                                                 | Ingress protection degree provided by IEC 60529                                         |
|  | Serial number                                                                           |
|  | LOT number                                                                              |
|  | Medical Device                                                                          |
|  | For indoor use only                                                                     |
|  | Temperature limitation                                                                  |
|  | Humidity limitation                                                                     |
|  | Atmospheric pressure limitation                                                         |
|  | Indication of connector polarity                                                        |
|  | Identifier of cuffs compatible for the device                                           |
|  | OMRON's trademarked technology for blood pressure measurement                           |
|  | Arm circumference                                                                       |
|  | Cuff positioning indicator for the left arm                                             |
|  | Marker on the cuff to be positioned above the artery                                    |
|  | Range pointer and brachial artery alignment position                                    |
|  | Range indicator of arm circumferences to help selection of the correct cuff size        |
|  | Need for the user to consult this instruction manual.                                   |
|  | Need for the user to follow this instruction manual thoroughly for your safety.         |
|  | Direct current                                                                          |
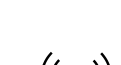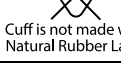
| Symbols description                                                                 |                                                                                                                                                                                                                                                                                 |
|-------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|   | Alternating current                                                                                                                                                                                                                                                             |
|  | Date of manufacture                                                                                                                                                                                                                                                             |
|  | To indicate generally elevated, potentially hazardous, levels of non-ionizing radiation, or to indicate equipment or systems e.g. in the medical electrical area that include RF transmitters or that intentionally apply RF electromagnetic energy for diagnosis or treatment. |
|  | KC mark                                                                                                                                                                                                                                                                         |
|  | BIS mark                                                                                                                                                                                                                                                                        |
|  | Efficiency level of power supply                                                                                                                                                                                                                                                |
|  | Not made with natural rubber latex                                                                                                                                                                                                                                              |

Important information regarding Electromagnetic Compatibility (EMC)

HEM-7140T1 manufactured by OMRON HEALTHCARE Co., Ltd. conforms to EN60601-1-2:2015 Electromagnetic Compatibility (EMC) standard. Further documentation in accordance with this EMC standard is available at:  
Asia Pacific: <https://www.omronhealthcare-ap.com/emc-information>  
Korea: <https://www.omron-healthcare.co.kr/product/HEM-7140T1>  
Refer to the EMC information for HEM-7140T1 on the website.

Correct Disposal of This Product (Waste Electrical & Electronic Equipment)

This marking shown on the product or its literature, indicates that it should not be disposed of, with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this product from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources.



Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can return this item for environmentally safe recycling. Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial waste for disposal.

The **Bluetooth**® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by OMRON HEALTHCARE Co., Ltd. is under license. Other trademarks and trade names are those of their respective owners.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries and regions. App Store is a service mark of Apple Inc. Google Play and Google Play logo are trademarks of Google LLC.

<https://www.omronhealthcare-ap.com/>

|                               |                                                                                                                                                                                                                             |
|-------------------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| Manufacturer                  | OMRON HEALTHCARE Co., Ltd.<br>53, Kunotsubo, Terado-cho, Muko, KYOTO, 617-0002 JAPAN                                                                                                                                        |
| EU-representative             | OMRON HEALTHCARE EUROPE B.V.<br>Scorpius 33, 2132 LR Hoofddorp, THE NETHERLANDS<br><a href="http://www.omron-healthcare.com">www.omron-healthcare.com</a>                                                                   |
| Importer in EU                |                                                                                                                                                                                                                             |
| Asia Pacific HQ               | OMRON HEALTHCARE SINGAPORE PTE LTD.<br><a href="http://www.omronhealthcare-ap.com">www.omronhealthcare-ap.com</a>                                                                                                           |
| Production Facility           | OMRON HEALTHCARE MANUFACTURING VIETNAM CO., LTD.<br>No.28 VSIP II, Street 2, Vietnam-Singapore Industrial Park II, Binh Duong Industry-Services-Urban Complex, Hoa Phu Ward, Thu Dau Mot City, Binh Duong Province, Vietnam |
| Importer in Republic of Korea | OMRON HEALTHCARE KOREA Co., Ltd.<br>A-dong 18F, 465, Gangnam-daero, Seocho-gu, Seoul, Korea<br>Customer Service Center(1544-5718)<br><a href="http://www.omron-healthcare.co.kr">www.omron-healthcare.co.kr</a>             |