
Quick Start Guide

iTalk2™ communication device

www.ablenetinc.com

Updated: December 10, 2020 4:21 PM

Colored switch top
activation surface

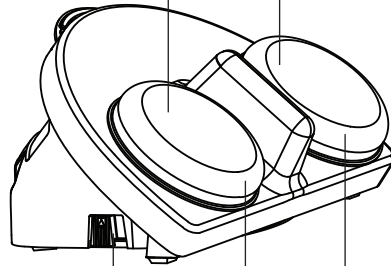Levels
switch

Clear snap cap for
symbol overlays

Recording indicator
light

Sound output speaker

Microphone

Record button

On/off and
volume control

1/8-in (3.5-mm) accessibility
switch input/output jacks

#6-32 threaded inserts
for AbleNet Universal
Mounting Plate
attachment (mounting
plate not included)

Requires one
alkaline 9-volt
battery (battery
not included)

Getting Started

1. Add one alkaline 9-volt battery to device (battery not included). To open battery door, rotate D-ring counterclockwise and lift up. To lock battery door, rotate D-ring clockwise.
2. Turn device on
3. Select desired level
4. Press and hold record button until recording indicator light turns on
5. Press and hold one colored switch top. Begin speaking into microphone after you hear beep.
6. When done recording message, release colored switch top
7. Repeat steps 5 and 6 for other colored switch top
8. Press the record button to turn the recording indicator light off
9. Adjust volume as needed
10. Your device is ready to use

Picture Symbol Overlays

A picture symbol can be placed under the clear snap cap to give additional context to what will happen when the colored switch top is activated. Symbol overlays should fit within a 2-in (5-cm) circle area.

Activate Switch Adapted Toys or Appliances

By default, the accessibility switch input/output jacks of this device will activate a switch adapted toy or appliance.

1. Plug your toy or appliance into one or both of the 1/8-in (3.5-mm) accessibility switch input/output jacks on the device. Some toys or appliances may require the included plug-to-plug adapter to be used to connect to the device.
2. The toy or appliance will activate as long as the message is playing.

Note: Some electrical appliances will require a PowerLink to be activated.

Play Message With an Accessibility Switch

By default, the accessibility switch input/output jacks of this device will activate a switch adapted toy or appliance. The following steps will change the accessibility switch input/output jacks so that they can be used with an accessibility switch.

1. Turn device off
2. Plug an AbleNet accessibility switch with 1/8-in (3.5-mm) switch plug into the accessibility switch input/output jacks on the device. One or two switches may be used.
3. Activate the AbleNet accessibility while turning device on

Note: When the device is turned off, the accessibility switch input/output jacks will revert to the toy/appliance functionality. To use accessibility switches again, repeat the above steps.

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Warranty & Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(800) 322-0956
(651) 294-2200
www.ablenetinc.com

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For private households:

Information on Disposal of Users of WEEE

This symbol  on the product and/or accompanying documents means that used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product to designated collection points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. Please contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

For professional users in the European Union:

If you wish to discard electrical and electronic equipment (EEE), please contact your dealer or supplier for further information.

For disposal in countries outside of the European Union:

This symbol is only valid in the European Union (EU). If you wish to discard this product, please contact your local authorities or dealer and ask for the correct method of disposal.

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