


Setting up & using the CareLink™ Connect app

For Care Partners

- 1 Check the [Compatibility Chart](#) found on your local website FAQ.
- 2 Download the [CareLink™ Connect app](#) on the Apple® or Google Play™ app store.
Important - We recommend to turn off your Operating system auto updates to help ensure that you won't be using an unverified version of the app.
- 3 Follow the [On Screen](#) prompts.
- 4 Read the [Dosing Disclaimer](#) that states dosing decisions should not be made based on the app then get started.
- 5 Create your [Partner Account](#). Once created proceed to login.
- 6 Send a [Follow Request](#) by inputting the CareLink™ Personal account username of the person you would like to follow.
- 7 Pump User [Approves Request](#) via their CareLink™ Personal account then you are connected.

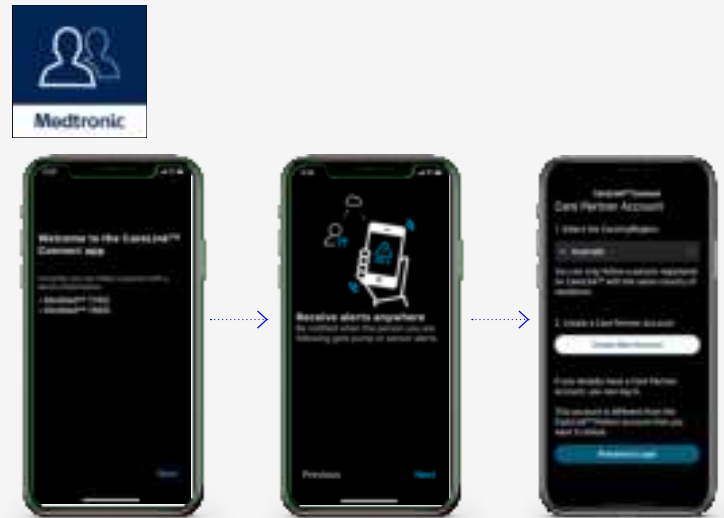

The app is now ready to use.

You are now connected to your partner.

Home Screen View




Alerts



History



Time in Range

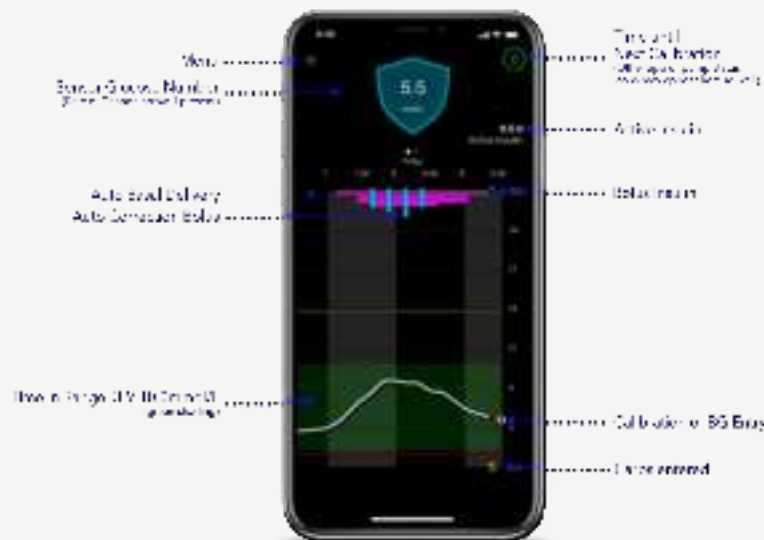

Personalised diabetes insights [anytime, anywhere.](#)