# Testset 909

## Reizdarmprofile (A670, A71) im Stuhl: Histamin (A410), Tryptophan (A651), Serotonin (A652), GABA (A653)

### Testanleitung

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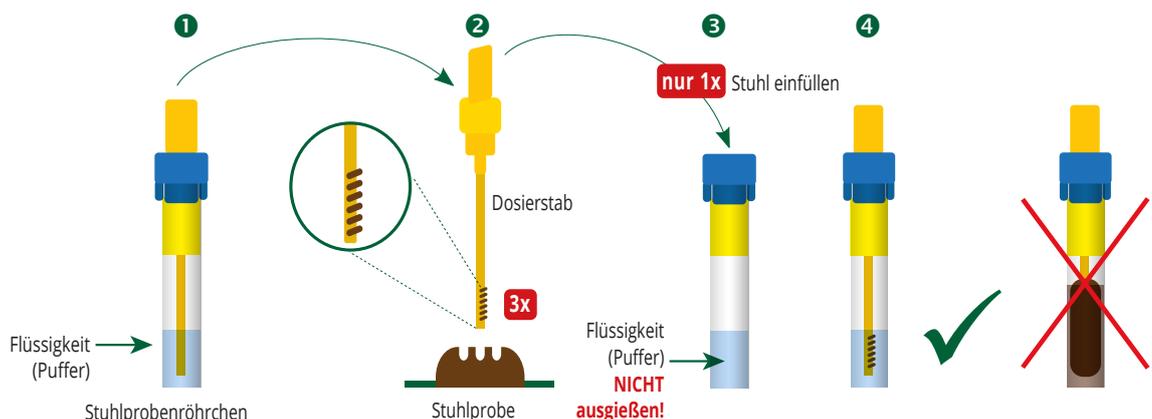
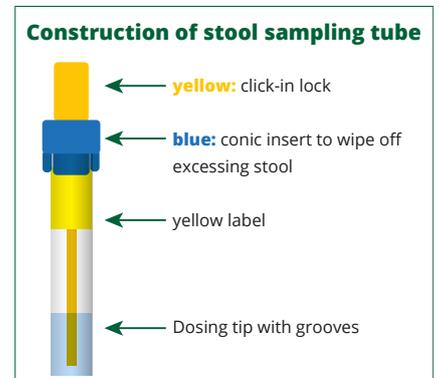
## Test Instructions

### The test set includes:

- 1 x Request Form
- 1 x Test Instructions
- 1 x Special Stool Sampling Kit with yellow label
- 1 x Stool Catcher
- 1 x Transporting Tube

## Use of the tube for stool sampling

### 1. Pull out the dosing rod

Hold the stool sample tube onto the blue conic insert (**the blue conic insert remains closed on the stool sample tube**). Then untwist the yellow dosing rod and pull it out. **Do not pour out the liquid from the stool sampling tube!**

### 2. Take stool sample

Pierce with the dosing rod at different places of a stool sample **3 times**. The grooves of the dosing tip must be filled with stool sample.

### 3. Return dosing rod

Put the dosing rod with the stool sample attached to it back into the stool sampling tube **one time** and **do not pull it out again**. Wipe off excessing stool at the narrow opening of the blue conic insert.

**Please note: only this small, defined amount of stool** on the dosing rod may enter the inside of the stool sampling tube.

### 4. Close the stool sampling tube

Then turn the dosing rod so that the click-in lock clicks audibly.

**Please note: Your stool sample does hardly lose color** inside the stool sampling tube. However, if you introduce the dosing rod into the stool sample tube more than once, the stool solution turns brown due to too much stool **and your sample can no longer be measured or you will receive a false positive result!**

### 5. Packaging

Insert the stool tube into the enclosed transport tube and close securely.

### 6. Shipping the sample

Then pack the transport tube and the completed request form in the return envelope and preferably hand it in at a post office.

**Important: Please do not send on Fridays, weekends or before public holidays.**