
ADAPTER FOR
HOSE AND MASK MAINTENANCE
COMPATIBLE WITH MOST CPAP HOSES

1 - SoClean Adapter

2 - Hose Sleeves
[For use with 12 mm hose]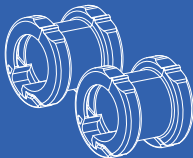

INSTRUCTIONS FOR USE
PNA1604-23-CA

If you need help connecting your adapter, contact SoClean.

For SoClean operation information,
please see the User Manual
provided with the device.

An electronic version of the User
Manual is available at SoClean.com
or you may contact Customer Care.

IMPORTANT: This adapter needs to fit
snugly. If the fit appears too tight, do not
force adapter into tubing. If the adapter is
loose, call Customer Care.

US: 1-866-501-3705 | **CA:** 1-800-916-4577
SoClean.com | info@soclean.com

Installation Instructions

SoClean Adapter

Shown
connected to
popular CPAP
hoses.

Detach **Ozone Tube** from **Injection Fitting** or your previous **SoClean Adapter**. Save **Injection Fitting** in case you need it in the future.

Move on to Step 1.


Ozone Tube

Injection Fitting


Check Valve

*** This end of the Ozone Tube connects to the SoClean Adapter**

Tube is pre-connected to the back of the SoClean 2 unit.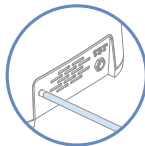


*** This end of the Ozone Tube is pre-connected to the back of the SoClean 2 unit.**

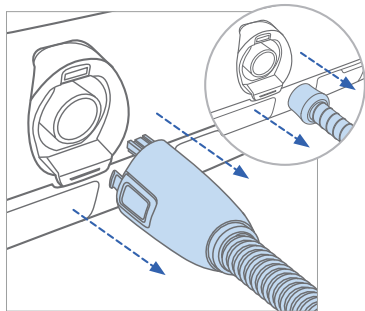
1

Remove the hose from your sleep equipment.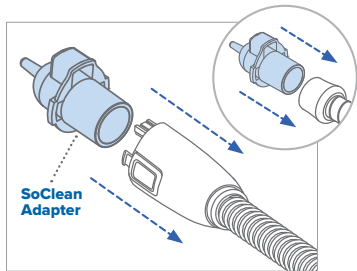

2

Install the SoClean Adapter onto the hose.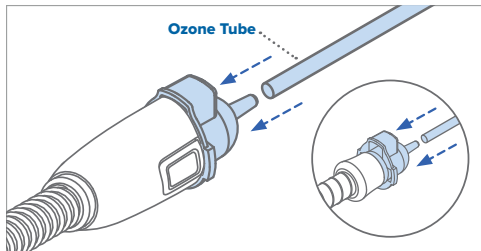

3

Connect **Ozone Tube** to the **SoClean Adapter**. You are now ready to use your SoClean.

Place mask into the **SoClean Chamber** and run the maintenance cycle.

NOTE: SoClean is not responsible for damage to your sleep equipment machine or personal property resulting from equipment mishandling.


WARNING: The **Ozone Tube** should always be connected to the **SoClean Adapter** and the back of the SoClean unit when operating the SoClean.

Sleep Equipment Hose Compatibility


For use with hose diameters of 15 mm and 19 mm and length up to 6' 6".

A 12 mm hose diameter will require a SoClean Hose Sleeve.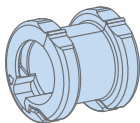


The images shown are actual size and can be used to determine which hose you have.

If you determine you require the Hose Sleeve, contact SoClean Customer Care or your authorized reseller.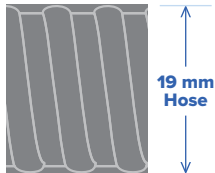


Reconnecting Your Sleep Equipment

Once the **Maintenance Cycle** is complete and you are ready to use your sleep equipment: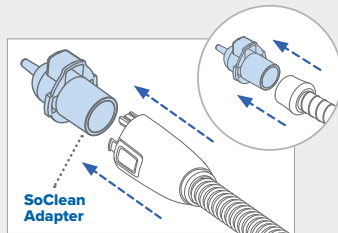

1

Remove the **SoClean Adapter** from your hose.

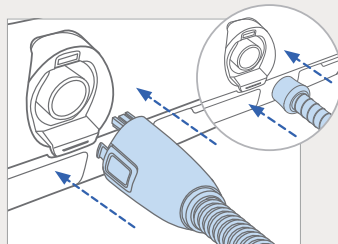2

Reconnect your hose to the sleep equipment hose port.

Using Your SoClean

The below images show both SoClean devices connected.

Once you have completed the setup steps, you are ready to begin the daily maintenance of your sleep equipment accessories with your SoClean.

Safety Checklist Before Using Your SoClean

- ☐ Sleep equipment accessories are washed in sink and dried before first use
- ☐ SoClean Adapter is installed on sleep equipment hose
- ☐ Ozone tube is connected to the SoClean Adapter
- ☐ Sleep equipment hose is still connected to the mask
- ☐ SoClean is powered on
- ☐ Filter is installed

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SoClean Inc.