

# Waterless Milk Warmer™ Quick Start Instructions

Review instruction manual for important safeguards prior to first use.

## Set-up




Expand liner prior to installing in warmer.




Bend insert at hinge and place into position.



Evenly spread liner within warmer cavity.

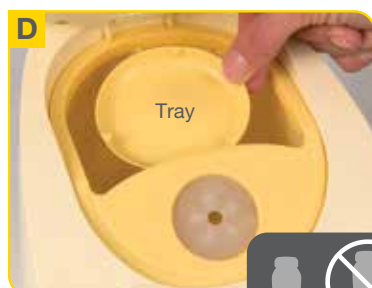


Push tray, flat-side down, into bottom of warmer.

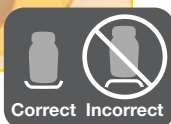

Load feed.


**NOTE:** Always use insert tray to warm container or syringe.




Press power button to turn on the device. 

## Program


### Select Mode

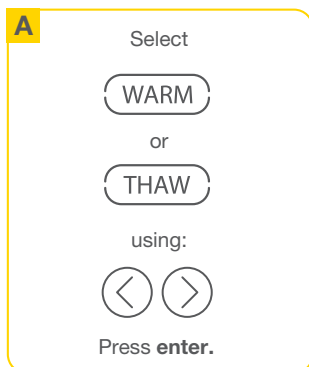


### Select Starting Temperature

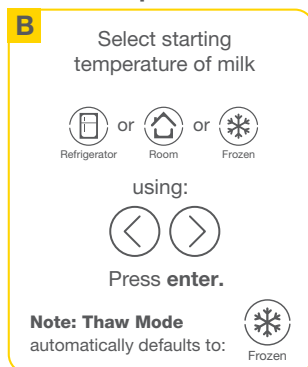

### Select Volume

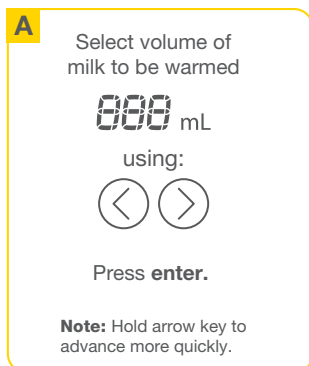

### Confirm & Begin

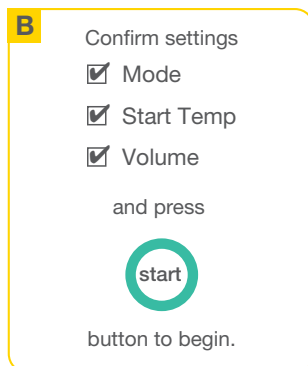

## Ready

Flashing green light indicates milk is ready.

