


Cleaning and Maintenance Part I (Product Guide)

It is recommended to clean the pulse oximeter regularly. To clean, use a soft cloth lightly dampened with water and a separate soft cloth lightly dampened with isopropyl alcohol.

1. Make sure that the device is off and remove the batteries.
 2. Wipe the outer surface of the device and the finger chamber using lightly dampened soft cloth.
 3. Allow the device to air dry thoroughly before use.
- Caution: Do not use any strong dissolving agent such as acetone.
 - Caution: Do not rub the body of the device using materials such as steel wire balls or polished metal objects.
 - Caution: Do not immerse or soak any part of the device in any liquid.
 - Caution: Do not pour or spray liquid onto the device.
 - Caution: Do not allow any liquid to seep into the device during cleaning.
 - Caution: Do not clean the device using high-temperature and/or high-pressure disinfecting gas.

Cleaning and Maintenance Part II (Product Guide)

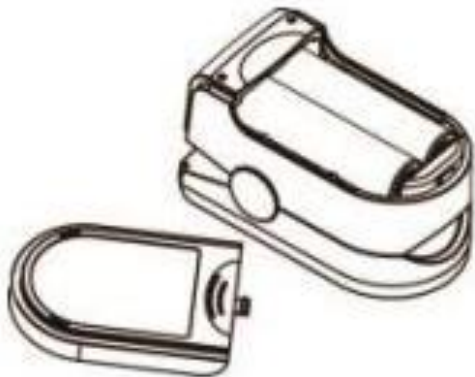

- Remove the batteries from the battery compartment if the pulse oximeter will not be used for an extended period of time.
- Replace the batteries if they are low on power.
- Clean the pulse oximeter and the fingertip before every use to ensure accurate reading.
- Store the pulse oximeter between -13°F and 158°F (-25°C and +70°C) and at humidity levels no greater than 93%.
- Periodically check the pulse oximeter for damage.
- Do not use the pulse oximeter in an environment with flammable gases and/or where the temperature or humidity is excessively high or low