

Adjusting the receiver gain can be done with the Roger Touchscreen Mic, Roger Select and Roger On

Each press on the + or – changes the EasyGain by 2dB

Roger Touchscreen Mic

Place the hearing aid with the powered receiver (attached or installed) within 10cm/4inches of the Roger Touchscreen Mic. Tap **[Settings]** then **[Roger receiver settings]** on the Roger Touchscreen Menu. The transmitter will read the Roger receiver and then open the receiver settings menu. Tap **[EasyGain level]** and adjust as required.



Roger Select

For Roger Select, tap on the plus or minus as indicated in the image. Note: these are not buttons on the Select. They are 'touch sensitive' areas which change the EasyGain when touched with the finger.



Roger On

Simply press the function button to increase the volume by 2 dB.

Once +8 dB is achieved and you press the center button again, the EasyGain will jump to -8 dB. The display shows the actual volume, e.g.:

