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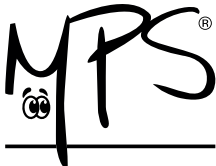
Safety and warranty manual MPS-Protective Top Shirt Dog.	
Product type: (Medical) Recovery wear for dogs.	
Read all safety and product information carefully before using the MPS-Protective Top Shirt on your pet. Failure to follow these instructions may cause injury. MPS products are not toys or fashion items. Adult supervision is required during the use of this product. Improper use may lead to hotspots, chafing, skin irritation, inflammation, or infection. In case of emergency or doubt, consult your veterinarian immediately. Important safety instructions: 1. Usage. The MPS-Protective Top Shirt should only be used on the advice/consultation of your veterinarian and only after a medical procedure, for medical conditions, and skin problems. In case of emergency or doubt, consult your veterinarian immediately! Do not use the shirt if it is damaged or if it contains loose parts. 2. Supervision. Your pet must always be under the supervision of a responsible adult while wearing the MPS-Protective Top Shirt. Regular checks for cleanliness, friction, constriction, chafing, tripping hazard, or choking hazard are mandatory. 3. Adjustment period. Allow your pet to get used to the shirt before a surgery and have your veterinarian put the shirt on your pet during anesthesia. If in doubt, always consult your veterinarian for advice. If your pet tends to chew of gnaw on materials, this product is not recommended. Therefore, always keep your pet under supervision. 4. Unusual behavior. In case of unusual behavior from your pet, which may include but is not limited to walking backward, sneezing, coughing, or incontinence, remove the MPS-Protective Top Shirt immediately and consult your veterinarian. 5. Visible issues. Check for proper fit and any skin irritation daily. Report skin irritation or any other problems to your veterinarian immediately for proper treatment and care. 6. Fit. Ensure the right fit to avoid safety risks. A shirt that is too large can cause tripping or choking hazards; a shirt that is too small can cause friction or skin irritation. Always measure your pet as described in the manual, which is available through the link at the bottom of the page. The MPS-Protective Top Shirt fully covers the dog's chest and chest area and is designed to have a snug fit. It is made of lightweight and breathable material. The MPS-Protective Top Shirt offers a gentle pressure, if desired, around the chest thanks to the adjustable closure with elasticated lower band. This pressure has a calming effect for most animals. Make sure there is always room for 1 to 2 fingers around the legs, chest, and neck. Check at least once a day for any irritation around the legs, chest, and neck. 7. Wear time. Never use the shirt for more than eight (8) consecutive hours unless prescribed by your veterinarian. Check the problem area or surgical zone every day. Remove the shirt every few days, wash it, and let it air dry. For constant protection, we recommend using a second MPS-Protective Top Shirt. See the washing instructions in the user manual, available in the link below.	
Disclaimer: While every effort has been made to ensure that the information in this manual is accurate and complete, Medical Pet Shirts is not liable for errors in or incorrect use of the manual and the product. We reserve the right to modify the manual. Warranty: Our products are guaranteed against manufacturing defects. In case of a product defect, please contact your veterinarian or the place of purchase.	Want to know more about this product? Visit: links.medicalpetshirts.com/pdp-top-shirt-dog Scan the QR code for detailed information about usage, sizes and more. 

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