MIXX[®]

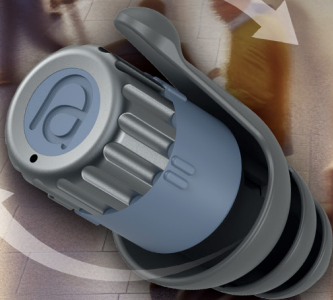
assistive audio

User Guide

[FRE] Guide de l'utilisateur [DEU] Benutzerhandbuch [ITA] Guida per l'utente [ESP] Guía del usuario

[NED] Handleiding [FIN] Käyttöopas [DAN] Indhold [POL] Instrukcja obsługi

[SWE] Användarhandbok [NOR] Brukerveiledning [POR] Guia do utilizador

ADAPTIVE

HEARING PROTECTION EAR PLUGS

protection for your ears in all situations

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S (5-9mm), M (6-10mm) and L (7-11mm) fully reusable
flared silicone earplugs. Supplied in a compostible bag

Ear Caps
(1 Unit)

Carry Loop & Case
(1 Unit)

Key Features

FRE Caractéristiques clés **DEU** Hauptfunktionen **ITA** Caratteristiche principali **NED** Hoofdfuncties
ESP Características clave **NOR** Nøkkelfunktioner **POR** Características principais **SWE** Nyckelfunktioner
FIN Keskeiset ominaisuuudet **DAN** Nøglefunktioner **POL** Najważniejsze właściwości

Effortlessly adapt to any environment with 23-4dB (NRR) noise reduction. Mode 1 protects your hearing while maintaining clear sound, making it perfect for concerts or lively spaces. Mode 2 offers full occlusion to block maximum noise, ideal for noisy environments like factories, workshops, or when you need to focus without distractions. Mode 3 keeps you aware of your surroundings, ideal for engaging in a conversation without needing to remove your earplugs.

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Fitting: Ensure the earplugs are fitted correctly for maximum protection. Incorrect fitting can reduce their effectiveness, increasing exposure to harmful noise.

Noise Protection: These earplugs reduce noise by approximately **23-4dB (NRR)**, depending on the mode. Suitable for everyday use, but we recommended using them for music, study and travel environments. They are not intended for extremely high-noise environments.

Care & Maintenance: Always keep your earplugs clean to prevent ear infections or irritation. Regularly clean the silicone tips as instructed in the care section. The product may be adversely affected by certain chemical substances. Only use agents that are not known to be harmful to the user. Refer to the manufacturer for further information. **Impulse Noise Warning:** These earplugs protect against impulse noise (e.g., sudden loud sounds) but may not be enough for extreme impulse noise like gunfire or blasts. Use additional protection in high-risk environments.

Choking Hazard: This product contains small parts. Keep out of reach of children and pets to avoid the risk of choking or accidental swallowing. **Do Not Modify:** Any modification of the earplugs may reduce their effectiveness and void the warranty. **Storage:** Store in a cool, dry place away from direct sunlight, moisture, and extreme temperatures to maintain performance, before and after use. **Obsolescence Date:** 2 years from the date of purchase. **Proper Fit, Use, and Inspection:** Ensure that the earplugs are fitted, adjusted, and maintained in accordance with the manufacturer's instructions. Wear the earplugs at all times in noisy surroundings. Regularly inspect them for serviceability and replace them if they show signs of wear, damage, or contamination. **WARNING:** Sudden or fast removal of earplugs out of the ear canal may damage the ear drum. **WARNING:** Failure to follow the fitting, use, and maintenance instructions provided may severely reduce the effectiveness of the earplugs. This could lead to inadequate noise protection and increased exposure to hazardous noise levels, potentially resulting in hearing damage.

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These earplugs are designed for repeated use, with durable, high-quality materials that can be cleaned and reused over and over again. To ensure optimal performance and hygiene, it's important to keep your earplugs clean. **Remove the Silicone Tips:** Gently pull the silicone tips off the ear caps before cleaning. **Clean the Tips:** Use mild soap and water to clean the silicone tips, do not use harsh chemicals, as this may damage the product. Rinse thoroughly and allow them to dry completely before reusing. **Cleaning the ear Caps:** Wipe the ear caps with a clean, dry cloth. Do not submerge the earplugs in water or use harsh chemicals, as this may damage the product. **Reassemble:** Once the tips are fully dry, securely reattach the ear plugs to the ear caps.

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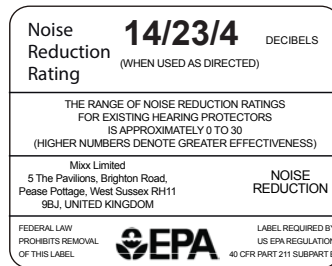
This product has been classed as Personal Protective Equipment (PPE) by the European PPE regulation (EU) 2016/425 and has been shown to comply with this regulation through the Harmonised European Standard EN352-2:2020: Ear-plugs.

This product has also been tested and certified to:

- ANSI S3.19-1974 standard.

CAUTION:

1. The environmental noise level as measured at the ear is 92 dBA.
2. The NRR is (value on label) decibels (dB).
3. The level of noise entering the ear is approximately equal to [92dB(A)—NRR] dB(A).

For noise environments dominated by frequencies below 500 Hz the C-weighted environmental noise level should be used. Improper fit of this device will reduce its effectiveness in attenuating noise. Consult the enclosed instructions for proper fit. Although hearing protectors can be recommended for protection against the harmful effects of impulsive noise, the Noise Reduction Rating (NRR) is based on the attenuation of continuous noise and may not be an accurate indicator of the protection attainable against impulsive noise such as gunfire.

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POL Deklaracja zgodności CE **SVK** CE-forsäkran om överensstämmelse

This Declaration of Conformity is issued under the sole responsibility of the manufacturer.

Manufacturer: Mixx Ltd, 5 The Pavilions, Brighton Road, Pease Pottage, West Sussex, RH11 9BJ, UNITED KINGDOM

Product name: Adaptive

Model/type: AA-ADA-NP-494

Trademark/brand: Assistive Audio

Manufacture Date: Nov/2024

Description of accessory, components and software required for intended use: No description available.

We, Mixx Ltd, hereby declare under our sole responsibility that the product described above is in conformance with the relevant directives and that the relevant documentation to support this exists.

The object of the declaration described above is in conformity with the essential requirements of the following directives and regulation:
Regulation (EU) 2016/425 (personal protective equipment)

The standards and/or technical specifications specified below have been applied for declaring conformity with the directives in this Declaration of Conformity: EN 352-2:2020

The PPE is subject to the conformity assessment procedure based on quality assurance of the production process (C2) under surveillance of the notified body Alienor Certification **CE** 2754, ZA du Sanital, 21 Rue Albert Einstein, 86100 CHATELLERAULT - France

John Whitehead
Chief Technical Officer
Mixx Ltd
24th October 2024

The **CE** mark indicates that the product meets EU safety, health or environmental standards. 2754 is the registered number of the notified body responsible for Module C2. SNR is the average noise attenuation of the product. EN352-2:2020 is the applicable CE standard for earplugs. MIXX Limited. Is the manufacturer. There are no spare parts for this product.

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+44 (0) 1293 560353

support@mixx-audio.com

Check out more of our Assistive Audio range, at assistive-audio.com

Sleep

Safety

Music

Flight

Découvrez d'autres articles de notre gamme d'audio d'assistance, à l'assistive-audio.com /
Weitere Informationen zu unserem Assistive Audio-Sortiment finden Sie unter assistive-audio.com /
Scopri di più sulla nostra gamma di audio assistivo, su assistive-audio.com /
Echa un vistazo a más de nuestra gama de audio de asistencia, en assistive-audio.com /
Sjekk ut mer av vårt utvalg av lydthjælpermidler på assistive-audio.com /
Veja mais da nossa gama de Áudio Assistivo, em assistive-audio.com /
Bekijk meer van ons assortiment ondersteunende audio op assistive-audio.com /
Tutustu tarkemmin avustavien äänien valikoimaamme osoitteessa assistive-audio.com /
Se mere af vores udvalg af lydthjælpermidler på assistive-audio.com /
Zapoznaj się z naszą ofertą systemów wspomagających dźwięk na stronie assistive-audio.com /
Kolla in mer av vårt sortiment av ljudhjälpmedel på assistive-audio.com