

Fitting Instructions

1. Remove inlay and place the M-Pace/L-Pace inside the shoe.
2. Place inlay over the top of the footplate. The inlay can be custom, off the shelf, or supplied with the user's footwear.
3. Insert foot into the shoe with laces loosened, feed the strap through the slot in the calf band and fasten the strap.
4. Check fit:



- Perform trial walking as above. Ensure the knee is not hyperextended.
 - Check footwear is not overly distorted.
 - Ensure the patient is comfortable and there are no areas of pressure which may lead to lasting sores, or ulcerations.
5. Make adjustments if needed:
 - The strap can be adjusted to the required size by removing the fastening tab and trimming the strap with scissors, or
 - By adjusting the position of the double-sided hook and loop fastener at the other end of the strap.
 - The inlay can also be bonded to the M-Pace/L-Pace footplate to secure its position, if desired.



Intended use

- Pace products are designed to promote dynamic walking in patients with drop foot or other gait abnormalities.
- Pace products should be used only as directed by a medical practitioner.

Footwear selection

- For optimal performance and durability of the Pace range, footwear should have a firm heel counter, non-slip flexible sole, and enough room to accommodate the orthosis.
- Boots and high heels should not be worn.
- Pace products must not be used without footwear.

Maintenance

- The products should be regularly cleaned by wiping with a damp cloth and hand washing the removable pad and strap.



Important

- The product must NOT be used for running, squatting, kneeling, climbing ladders, or climbing stairs by loading the footplate unevenly.
- Discontinue use immediately if you notice any excessive skin redness, abrasion, or irritation.
- Any serious incident should be reported to the manufacturer and the relevant competent authority.

Disposal

- Please dispose of your M-Pace/L-Pace in accordance with local and national laws and regulations.

Warranty

- Hardware: 2 years
- Soft goods: 90 days



Trulife Limited
41 Amos Road, Sheffield
S19 1BX, United Kingdom
www.trulife.com



MOSS
Schiffgraben 41, 30175,
Hannover, Germany

