
Nipple Shields with Sterilizer Case

20 mm | 24 mm


Nipple shields should be used under the care of a medical professional or lactation consultant. Before first use and after every use, wash nipple shields in warm soapy water, rinse clean, sterilize, and air dry.

WARNING! Sterilizer case and water will be HOT after microwaving. USE CAUTION.


Rest nipple shields on holders inside case. Add water to fill line (7 tsp / 35mL), close lid securely, and microwave (using a 700-1250 watt microwave) for 3 minutes and 30 seconds. Carefully remove sterilizer from microwave and drain water from either end of case. Open lid and allow shields to cool for 5 minutes, then wipe out any remaining water before closing lid to store.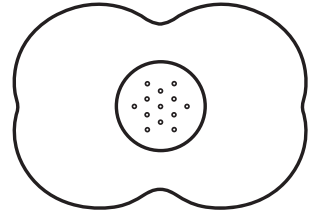

- 1 Check that the nipple shield is not inside out. Dr. Brown's logo should be visible in the correct direction from the outside.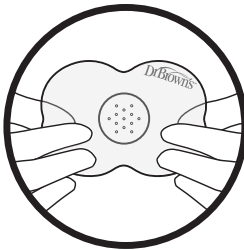


- 2 Center holes over your nipple. Gently stretch nipple shield and apply to breast.

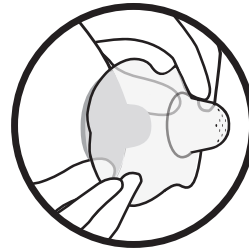- 3 If needed, apply a very small amount of nipple balm to the inside wings of the shield to help stick to skin.

- 4 Once feeding begins you may need to rotate shield to ensure that baby's nose is free from obstruction and to maximize skin-to-skin contact.

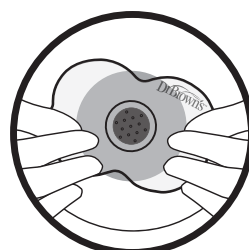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