⚙️ Installation with Clamp-On Bracket

1. Ideal for ComfyGO Mobility Electric Wheelchairs:
 - A. IQ-8000 Series, IQ-9000 Series, and IQ-7000 Series
2. Open the clamp and place it around the handlebar.
3. Align the controller with the brass inserts.
4. Use the short screws to fix the controller to the bracket.
5. Use the long screw, and tighten the bolt at the bottom using the Allen wrench.

⚙️ Installation with Screw-On Bracket

1. Ideal for ComfyGO Mobility Electric Wheelchairs:
 - A. Phoenix Carbon Fiber Series, X-9 Series, X-7 Series, X-6 Series, and X-Lite Series
2. Align the controller's back and screw it in securely using the shorter screws.
3. Fix the bracket directly onto a flat bar or panel using the long screws.

🔌 Charging

1. When turned on, 5 battery LEDs will light up, then turn off after 5–6 seconds if unused.
2. If only 1 LED shows, please charge it as soon as possible.
3. Use the provided USB-A to USB-C cable.
4. Charging port is on the back of the controller.
5. Full charge takes ~2–3 hours.

🧑 Basic Operation

1. Joystick: Move to control direction.
2. + / – Buttons: Adjust speed.
3. Power Button: Turn controller on/off.
4. Horn Button: Activates horn.

⚠️ Safety & Warnings

- Do not use in rain or wet conditions.
- Ensure mount is tight before use.
- Only charge with certified USB chargers.
- Keep out of reach of children
- Read the wheelchair's user manual for safety rules that apply to the caregiver controller.


🔄 Pairing Instructions

1. Turn on the wheelchair, then turn on the caregiver controller.
2. Press and hold the – (speed decrease) button on the caregiver controller until the red LED appears.
3. Then press and hold the – button on the wheelchair controller until the red LED appears or starts blinking.
4. Press the horn button on the wheelchair controller.
5. Both devices will automatically turn off after successful pairing.
6. Turn them on again and enjoy using the caregiver controller.

❗ If it doesn't work, repeat the steps carefully or contact us using the information at the bottom of this page.

United States of America & Canada

- info@ComfyGoMobility.com
- +1 844 755 5055
- Visit comfygomobility.com/pages/contact or scan the QR code with your smartphone to fill out the help form.

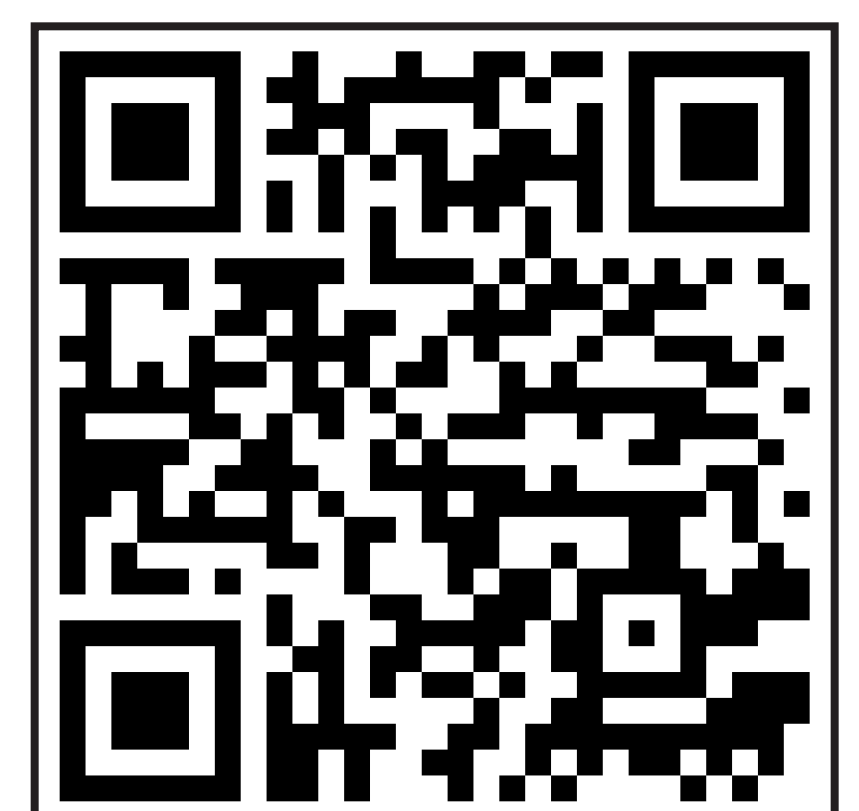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