ADVANCED
BIONICS

POWERFUL CONNECTIONS

M Zn-Air Battery Pak

CI-5501

instructions for use

English, Français, Español, Deutsch, Nederlands, Italiano

Contents

PURPOSE AND SCOPE OF IFU	5
LIMITATIONS AND CONTRAINDICATIONS	5
PRECAUTIONS, CAUTIONS, AND WARNINGS	5
UNDESIRABLE SIDE EFFECTS	6
PRODUCT DESCRIPTION	7
INDICATIONS FOR USE	8
INTENDED PURPOSE	9
<i>Intended Users</i>	10
<i>Intended Use Environment</i>	10
COMPATIBILITY	10
OPERATING INSTRUCTIONS	10
<i>Inserting Zn-Air Disposable Battery Cells (Non-Tamperproof Cartridge)</i>	10
<i>Inserting Zn-Air Disposable Battery Cells (Tamperproof Cartridge)</i>	13
CARE AND MAINTENANCE	17
<i>Storing the M Zn-Air Battery Pak</i>	17
<i>Cleaning and Maintenance</i>	17
<i>Recommended Operating and Storage Temperature Ranges</i>	18
<i>Expected Life</i>	18
<i>Ingress Protection (IP) Ratings</i>	18

<i>DISPOSAL AND HAZARDOUS MATERIALS</i>	<i>19</i>
<i>AVAILABLE PRODUCT OPTIONS AND MODEL NUMBERS</i>	<i>19</i>
<i>CLINICAL BENEFITS</i>	<i>20</i>

SYMBOLS TABLE

Labeling symbols and their meanings

	European Community Mark of Conformity.
	Date of Manufacture
	Manufacturer
	Model number
	LOT number
	European authorized representative
	Unique Device Identifier
	Medical Device
	Type of Protection: BF
	Suitable temperature range for transport and storage
	See Instructions for Use
	Fragile

	Do not get wet
	Suitable relative humidity range for transport and storage
	Caution (refer to "Cautions & Warnings" section for further information)
	Dispose of in accordance with applicable national and local regulations
	MR Unsafe

PURPOSE AND SCOPE OF IFU

This IFU is designed to help cochlear implant recipients, and their caregivers if applicable, to understand the use and care of the M Zn-Air Battery Pak. It is also meant to be used by hearing care professionals trained in the fitting of cochlear implants to counsel the recipients on the use of the M Zn-Air Battery Pak and to troubleshoot problems.

Limitations and Contraindications

As the M Zn-Air Battery Pak is a component of the HiResolution™ Bionic Ear System, the following contraindications stated for the HiResolution Bionic Ear System are applicable: Deafness due to lesions of the acoustic nerve or central auditory pathway; cochlear ossification that prevents electrode insertion; absence of cochlear development.

Precautions, Cautions, and Warnings

- *This device should be used only by the individual for whom it is prescribed.*
- **CHOKING HAZARD:** *Contains small parts that pose a hazard of inhalation, choking, or ingestion. If any parts are swallowed or inhaled, consult a physician or hospital immediately.*
- *Do not leave children unattended with or allow children to play with the M Zn-Air Battery Pak or battery cells.*
- *Do not allow children to place the M Zn-Air Battery Pak or battery cells in the mouth or nose.*
- *Do not connect the M Zn-Air Battery Pak to the M Battery Charger. The charger is to be used with the rechargeable M Batteries only.*

- *Remove your sound processor and consult your cochlear implant professional if uncomfortable sounds are heard or in case of discomfort, pain or skin irritation.*
- *If a battery shows visible signs of leakage, dispose of it in accordance with local regulations.*
- *Do not allow leaking battery fluid to come into contact with skin, mouth, or eyes, as this fluid may cause a burn.*
- *High levels of static electricity may damage electronic components of your sound processor or implant. Care should be taken to avoid exposure of the system to static electricity.*
- *In the event that you experience any issues with your product, please contact your cochlear implant professional or the manufacturer. Do not attempt to service or modify the Naída CI M, Sky CI M, or its accessories. Doing so may compromise system performance and will void the manufacturer's warranty. Products should be serviced only at Advanced Bionics and damaged products should be returned to Advanced Bionics.*


Undesirable Side Effects

Undesirable side effects of your Naída CI M or Sky CI M system may include skin irritation and discomfort from pressure on the ear, device overheating, or overly loud sounds.

Product Description

The M Zn-Air Battery Pak is a disposable power source for the Naída™ CI M and Sky CI™ M Sound Processors. The type of disposable battery to be used with the M Zn-Air Battery Pak is a 675 power implant batteries (P675). Regular 675 hearing instrument batteries will not be powerful enough for a cochlear implant. The suggested manufacturer and battery is PowerOne P675 Implant Plus battery.

The M Zn-Air Battery Pak is available with non-tamperproof and tamperproof cartridges.

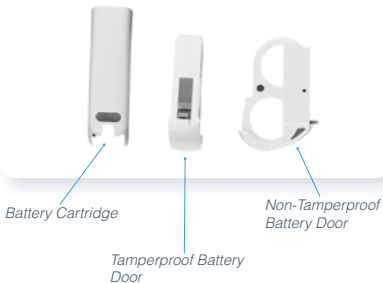INDICATIONS FOR USE

The M Zn-Air Battery Pak is an external component of the HiResolution™ Bionic Ear System, which is intended to restore a level of auditory sensation to individuals with severe-to-profound sensorineural hearing loss via electrical stimulation of the auditory nerve.

Adults

- *18 years of age or older.*
- *Severe-to-profound bilateral sensorineural hearing loss or severe-to-profound unilateral hearing loss.*
- *Post-lingual onset of severe or profound hearing loss.*
- *Limited benefit from appropriately fitted hearing aids, defined as scoring 50% or less on a test of open-set sentence recognition (HINT Sentences).*

Children

- *12 months through 17 years of age.*
- *Severe-to-profound bilateral sensorineural deafness or severe-to-profound unilateral hearing loss.*
- *Use of appropriately fitted hearing aids for at least 6 months in children 2 through 17 years of age, or at least 3 months in children 12 through 23 months of age. The minimum duration of hearing aid use is waived if x-rays indicate ossification of the cochlea.*
- *Little or no benefit from appropriately fitted hearing aids. In younger children (< 4 years of age), lack of benefit is defined as a failure to reach developmentally appropriate auditory milestones (such as spontaneous response to name in quiet or to environmental sounds) measured*

using the Infant-Toddler Meaningful Auditory Integration Scale or Meaningful Auditory Integration Scale or $\leq 20\%$ correct on a simple open-set word recognition test (Multisyllabic Lexical Neighborhood Test) administered using monitored live voice (70 dB SPL). In older children (≥ 4 years of age), lack of hearing aid benefit is defined as scoring $\leq 12\%$ on a difficult open-set word recognition test (Phonetically Balanced-Kindergarten Test) or $\leq 30\%$ on an open-set sentence test (Hearing In Noise Test for Children) administered using recorded materials in the sound field (70 dB SPL).

Intended Purpose

The M Zn-Air Battery Pak is an accessory of an auditory active-implantable system, the HiResolution Bionic Ear system. The HiResolution Bionic Ear system is intended to provide auditory sensation via electrical stimulation of the auditory nerve for individuals with severe to profound bilateral or unilateral sensorineural hearing loss. Severe hearing loss is defined as audiometric thresholds greater than or equal to 70 dB HL, but less than 90 dB HL. Profound hearing loss is defined as audiometric thresholds greater than or equal to 90 dB HL. The external components work together with the implant to bypass the damaged part of the inner ear and convert sound picked up by the microphone or streamed via wireless communication into electrical signals that are used by the cochlear implant to enable hearing.

The M Zn-Air Battery Pak is intended to power the Advanced Bionics Naída CI M and Sky CI M Sound Processors.

Intended Users

The intended users of the M Zn-Air Battery Pak are users of the Naída CI M or Sky CI M Sound Processors, their caregivers if applicable, and hearing care professionals.

The recipient, or their caregiver, should, at a minimum, be capable of changing the battery, connecting the earhook, placing and removing the sound processor from their ear, and placing and removing the headpiece from the implant site.

The hearing care professional should be trained in the use and fitting of the cochlear implant system.

Intended Use Environment

The intended use environments of the M Zn-Air Battery Pak are healthcare and daily living environments. The M Zn-Air Battery Pak is designed to be worn daily during the user's normal waking hours.

Compatibility

Naída CI M Zn-Air Battery Pak is compatible with Naída CI M and Sky CI M Sound Processors from Advanced Bionics.

Operating Instructions

Inserting Zn-Air Disposable Battery Cells (Non-Tamperproof Cartridge)

Holding the battery cartridge in one hand, gently but firmly push down on the lever on the back of the battery cartridge. This will cause the Zn-Air battery drawer to pop out of the bottom of the cartridge. Pull the battery door out of the cartridge with the other hand.

Remove the Zn-Air cells from their package, making sure to remove the sticker from the flat positive (+) side of the cell. Removing the sticker allows air to enter and activate the battery, so be sure to use the battery immediately after peeling off the sticker.

Place two Zn-Air cells, positive (flat) side up, in the two openings in the battery drawer.

Once batteries are flush with the top of the battery drawer, gently slide the battery drawer back into the battery cartridge. If the cells are inserted incorrectly, the battery drawer will not close.

Then begin to attach the battery cartridge to the Naída CI by holding the sound processor in one hand. Rotate RF cable upward to avoid bumping it upon battery placement.

Align the battery cartridge connector with the processor housing side of the connector.

Slide the battery onto the processor until you feel it click into place.

Inserting Zn-Air Disposable Battery Cells (Tamperproof Cartridge)

Using the same tool provided for the M T-Mic™ microphone or M Earhook removal and replacement, insert the pointed end of the tool into the small hole located on the spine of the Zn-Air cartridge.

With the tool inserted in the hole, gently push in and downward until you feel the Zn-Air drawer release and pop out the button of the Zn-Air cartridge. If you do not have the tool present, any thin, pointed device should work to release the drawer.

Set the tool aside. From here you can grasp the Zn-Air drawer with your hand and gently pull it the rest of the way out, so you are able to insert the Zn-Air cells.

Remove the Zn-Air cells from their package, making sure to remove the sticker from the flat positive (+) side of the cell. Removing the sticker allows air to enter and activate the battery, so be sure to use the battery immediately after peeling off the sticker.

Place two Zn-Air cells, positive (flat) side up, in the two openings in the battery drawer.

Once batteries are flush with the top of the battery drawer, gently slide the battery drawer back into the battery cartridge. If the cells are inserted incorrectly, the battery drawer will not close.

Then begin to attach the battery cartridge to the Naída CI by holding the sound processor in one hand. Rotate RF cable upward to avoid bumping it upon battery placement.

Align the battery cartridge connector with the processor housing side of the connector.

Slide the battery onto the processor until you feel it click into place.

CARE AND MAINTENANCE

Storing the M Zn-Air Battery Pak

When not in use, store M Zn-Air Battery Pak in the equipment case provided.

Cleaning and Maintenance

Wipe the outside of the M Zn-Air Battery Pak with a soft, dry cloth.

Recommended Operating and Storage Temperature Ranges

CONDITION	MINIMUM	MAXIMUM
Operating Temperature	0°C (32°F)	45°C (115°F)
Storage Temperature	-20°C (-13°F)	55°C (131°F)
Relative Humidity	0%	95%

Expected Life

The expected life of the M Zn-Air Battery Pak is 1 year.

Ingress Protection (IP) Ratings

The following product configurations have an IP rating of 54 (protection against dust; protection against failure due to water splashes from all directions):

- The Naída CI M90 sound processor with an M Battery or M Zn-Air Battery Pak, an M T-Mic or M Acoustic Earhook, and a Slim HP, Slim HP Mic, or Slim HP Standard.
- The Naída CI M90 sound processor with an M Zn-Air Battery Pak, an M Earhook, and a Slim HP, Slim HP Mic, or Slim HP Standard.

DISPOSAL AND HAZARDOUS MATERIALS

Dispose of the M Zn-Air Battery Pak and Zn-Air batteries in accordance with applicable national and local regulations. Hazardous materials: not applicable to this product.

AVAILABLE PRODUCT OPTIONS AND MODEL NUMBERS

PRODUCT OPTION	MODEL NUMBER
Sand Beige	CI-5501-120
Chestnut	CI-5501-130
Silver Grey	CI-5501-140
Velvet Black	CI-5501-150
Alpine White	CI-5501-110
Lava Red	CI-5501-280
Blue Ocean	CI-5501-260
Majesty Purple	CI-5501-270
Caribbean Pirate	CI-5501-190
Precious Pink	CI-5501-250
Resin Beige	CI-5501-240

CLINICAL BENEFITS

The Intended Clinical Benefit of the M Zn-Air Battery Pak as part of the HiResolution Bionic Ear System is to:

- *provide useful hearing to individuals with severe-to-profound hearing loss via electrical stimulation of the auditory nerve.*
- *provide a combined type of stimulation, electric plus acoustic, when sound processors are enabled with the Acoustic Earhook.*

Users of the M Zn-Air Battery Pak in the European Union should report any serious incident to their local competent authority and to Advanced Bionics.

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©2021 Advanced Bionics LLC. All Rights Reserved.
Advanced Bionics LLC