WITHINGS SCANWATCH HORIZON

QUICK INSTALLATION GUIDE

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EN | FR | ES | IT | CZ

THANK YOU FOR CHOOSING WITHINGS SCANWATCH

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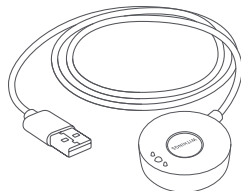
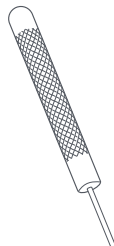
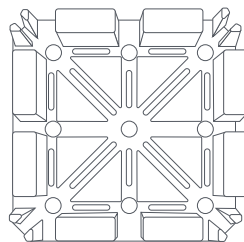
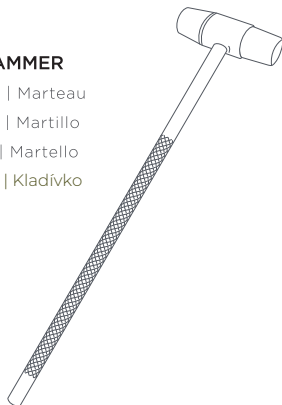
Avvio rapido |

Rychlý start

VISIT GO.WITHINGS.COM TO GET THE APP AND FOLLOW THE INSTRUCTIONS.

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INSTALL YOUR WATCH USING THE APP AND FOLLOW THE GUIDE TO DISCOVER ALL THE FUNCTIONALITIES OF YOUR PRODUCT.

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
Come indossare ScanWatch |

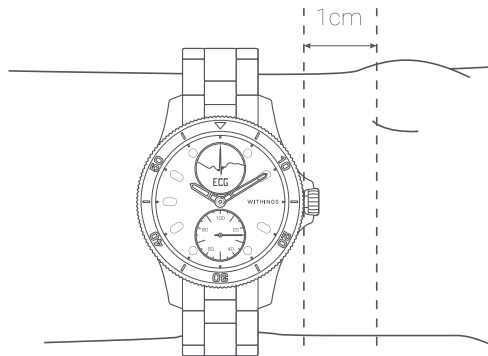Jak SCANWATCH nosit

THE WRISTBAND SHOULD BE COMFORTABLY TIGHT AND WORN 1 CM AWAY FROM THE WRIST BONE

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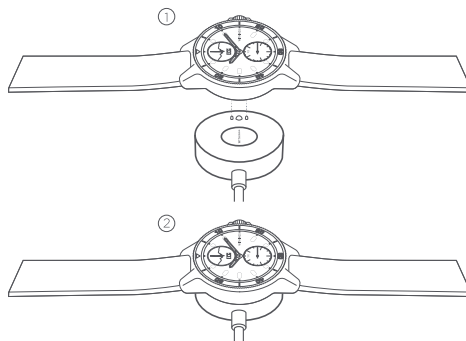
RECHARGE USING THE CABLE FOUND INSIDE THE BOX. BATTERY LIFE: UP TO 30 DAYS + 20 MORE DAYS IN POWER RESERVE MODE (TIME AND ACTIVITY TRACKING ONLY).

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
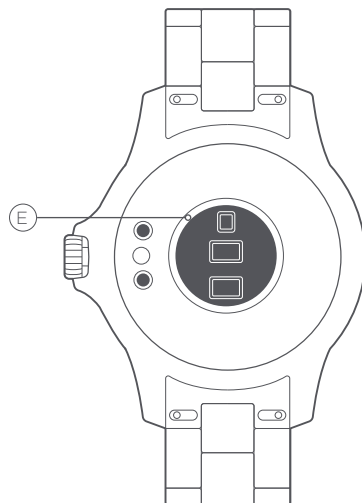
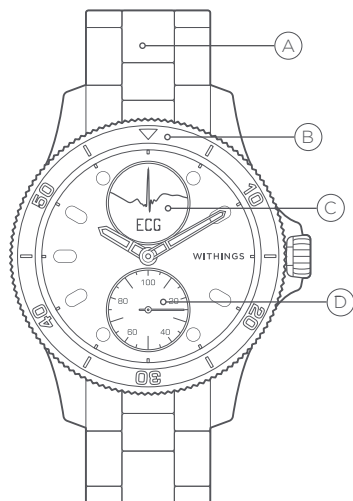
BAND TOOLKIT

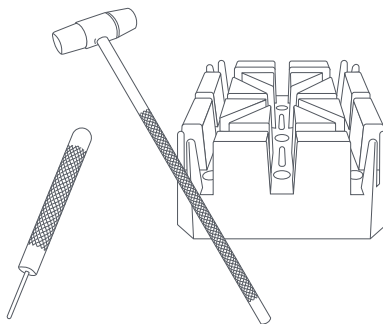Note: This toolkit is only suitable for a metal wristband with a folding clasp.

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
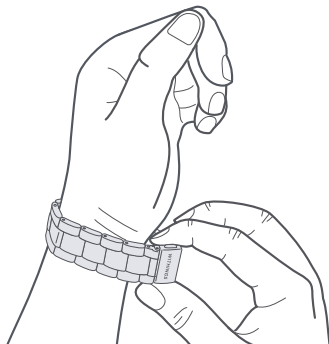
ADJUST THE WRISTBAND ON YOUR WRIST TO ESTIMATE THE NUMBER OF MESH LINKS TO BE REMOVED. NOTE THAT ONE OF THE LINKS IS SMALLER IN ORDER TO ADJUST THE BRACELET MORE FINELY.

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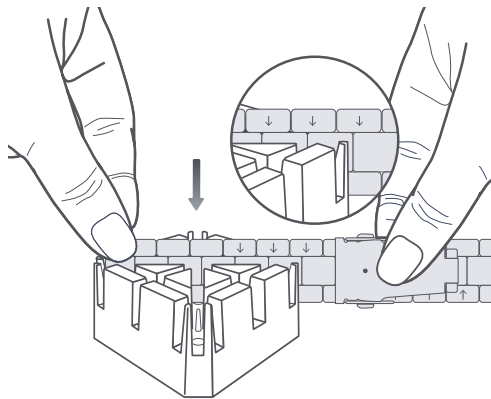
POSITION THE WRISTBAND IN THE HOLDER. THE ARROWS INSIDE THE WRISTBAND SHOULD POINT DOWNWARD.

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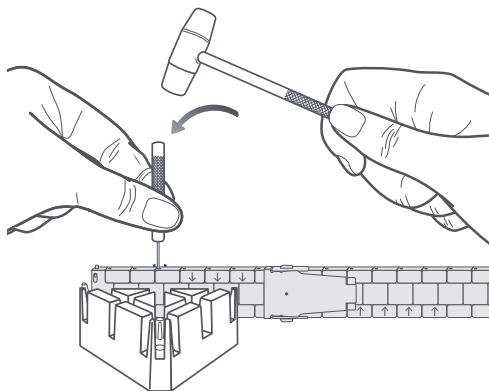
USE THE PIN PUNCH AND THE HAMMER TO PUSH OUT THE PIN THAT SUPPORTS THE MESH YOU WANT TO REMOVE.

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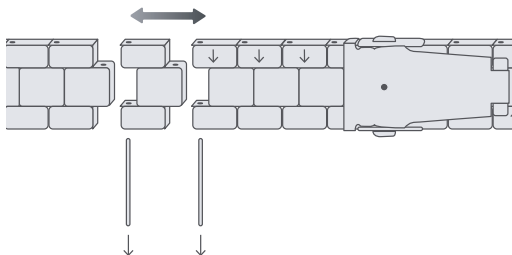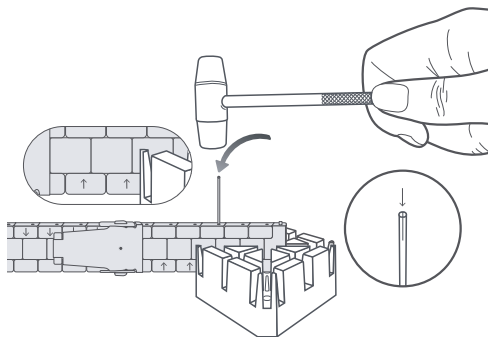
REASSEMBLE THE WRISTBAND BY REPLACING ONE OF THE PINS, THEN THE OTHER (IN THE OPPOSITE DIRECTION OF THE ARROWS) WITH THE HAMMER.

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Warranty

Garantie |
Garantia |
Garanzia |
Záruka

US | Withings One Year limited Warranty - Withings ScanWatch

Withings, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux ("Withings") warrants the Withings-branded hardware product ("Withings Product") against defects in materials and workmanship when used normally in accordance with Withings' published guidelines for a period of ONE (1) YEAR from the date of original retail purchase by the end-user purchaser ("Warranty Period"). Withings' published guidelines include but are not limited to information contained in technical specifications, safety instructions or quick start guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product's use.

UK | Withings 2 Year commercial guarantee Withings ScanWatch

Withings, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux ("Withings") warrants the Withings-branded hardware product ("Withings Product") against defects in materials and workmanship when used normally in accordance with Withings' published guidelines for a period of TWO (2) YEAR from the date of original retail purchase by the end-user purchaser ("guarantee Period"). Withings' published guidelines include but are not limited to information contained in technical specifications, safety instructions or quick start guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product's use.

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EN | This device complies with Part 15 of the FCC Rules and with Industry Canada licence-exempt RSS standard(s). Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation. Changes or modifications made to this equipment not expressly approved by Withings may void the FCC authorization to operate this equipment. Radiofrequency radiation exposure information: the radiated output power of the device is far below the FCC radio frequency exposure limits. Nevertheless, the device shall be used in such a manner that the potential for human contact during normal operation is minimized.

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Materiale |

Materiály

EN | List of materials potentially in contact with the skin during usage: Fluoroelastomer, silicone, leather, sapphire glass and stainless steel.

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Besoin d'aide ? |
¿Necesitas ayuda? |
Ti serve aiuto? |
Potřebujete pomoci?

➔ withings.com/support/scanwatch

For cleaning instructions, please visit
withings.com/support/scanwatch

EN | Withings hereby declares that the device Withings ScanWatch is in conformity with the essential requirements and other relevant requirements of Directive 2014/53/EU. The declaration of conformity can be found at: withings.com/compliance.

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«The crossed out wheeled bin symbol on your product, battery and/or accessories means that the item is classified as electrical and electronic equipment. Such items should not be mixed with general household waste, and must be taken to dedicated collection points at the end of their working life for proper treatment, recovery and recycling. This way you help save valuable resources and promote recycling of materials.»

Frequency Range : 2402.0 - 2480.0 MHZ
Maximum output power : -0.8 dBm

QIG_HWA09_77

WITHINGS

ScanWatch | Proactive Health Tracking | V1.1 - Jul. 2021

© 2021 Withings. All rights reserved.