




#### DOWNLOAD SMART APP

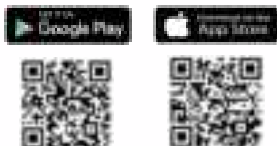

- Download and Install the CleanSpace Smart App onto your mobile device.
- Refer to the [CleanSpace SMART App Quick Start Supplement](#)
- Follow the steps below to update CST respirator firmware

#### PAIR CST RESPIRATOR

*Important: Confirm Bluetooth is turned ON in mobile device settings.*


- On main page select PAIR UNIT (UNIT PAIRING for iOS).
- Switch the CST respirator to STANDBY mode by pressing the 'ON' button once.
- All CST respirators within Bluetooth range and in STANDBY mode will be displayed on the screen in order of signal strength.
- Select the CST respirator that requires pairing.
- When paired the CST respirator will beep twice and the SET UP & CHECK page will automatically display.

**Note: Donning is not required to select 'Dashboard' for updating firmware.**



#### UPDATING CST FIRMWARE

- When paired, the DASHBOARD will display the CST respirator firmware version number in the Connection Status box. For the latest CST respirator firmware version number refer to the [CleanSpace website troubleshooting section](#).
- Select FILTER INFORMATION (Android) or Filter Change (iOS), located at the bottom of the DASHBOARD page.
- A 'FIRMWARE UPDATE' message will automatically display after a few seconds if the firmware requires updating. If no message appears then the firmware is already up-to-date.
- Follow the on-screen instructions to update CST respirator firmware and remain on this page until the firmware update is complete.
- Continue to follow the on-screen instructions to enable (activate) the updated firmware on the CST respirator.
- The firmware will now display the updated version number on the DASHBOARD (Android) / Filter Change (iOS) page.

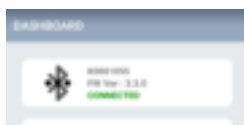

Connection Status displays the current Firmware version of the paired respirator.



Firmware Update Notification will automatically appear on **Filter Information** page when an update is required.



When the green bar indicates the firmware is updated, follow the 3 step process to complete the firmware update.



Connection Status displays the updated Firmware version of the paired respirator.

