Somna Collar

Somna Collar

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Somna Collar

Somna Collar is a daytime product that is developed to relieve tension in the body and shoulders. The collar can be used both when sitting still and when moving. The collar helps to ensure better posture, which promotes circulation and balance in the body. Use the collar with the reinforced side down.

Somna Collar complies with all necessary requirements, regulations and directives for medical devices. Dispose the product at an applicable disposal location.

For more information and accessories, go to www.abilia.com or scan the QR code.

Scan me!

Maintenance

 **CAUTION:** Use the included laundry bag. If the laundry bag is not used, Somna Collar can break.

Textile: 55% Polyester, 45% Cotton

Filling: 100% Polyester

Do not dry clean. Do not heat.

Important information

 **WARNING:** Read the following warning instructions before you use the product.

- Do not place Somna Collar on the throat when lying down.
- Do not heat Somna Collar.

Serious incidents

If a serious incident related to the product occurs, report this to the manufacturer and the competent authority of your country. A serious incident directly or indirectly led, might have led or might lead to the death of a patient, user or other person; the temporary or permanent serious deterioration of a patient's, user's or other person's state of health; or a serious public health threat.

Abilia AB, Råsundavägen 6, 169 67 Solna, Sweden
Phone +46 (0)8-594 694 00 | info@abilia.com | www.abilia.com

ABILIA