Contact Us Anytime, Anywhere

Visit www.rakiie.com for 24/7 customer service, or email us at support@rakiie.com. This product comes with a 12-month warranty. If there's anything missing, not fitting, or not meeting your expectations, contact us for an exchange or refund.

Meet Rakiie

We're always at your service for your satisfaction! We're all about safe sports and active living. Rakiie braces are here to prevent injuries and enhance your performance. Enjoy your activities more confidently with Rakiie! Beyond products, we're committed to your joy and well-being.

24/7 Live Chat: www.rakiie.com
E-mail: support@rakiie.com
Guarantee: 12-month warranty

Usage Instructions:

Usage Instructions:

Please read these instructions carefully before use.

- 1 Loosen the elastic bands on both sides of the product.
- 2 Put on the product on your shoulders, move the shoulder brace in front of your shoulder upward a little so that the back waist structure will be lower.
- 3 Pull it down flat to adjust the waist. Wrap the right strap around the left strap and secure with the Velcro.
- 4 Pull the adjustable shoulder strap forward, adjusting it to your desired tightness. Secure the strap handles by attaching them to the waist strap surface.
- 5 Wrap each compression strap around your waist and secure it to the front of the orthotic belt.
- 6 Attach the Velcro straps to the shoulder straps on both sides. Adjust the Velcro straps as needed to relieve pressure under your armpits based on your personal preference.


Note:

- 1 Do not use on areas other than the waist.
- 2 Do not wear while sleeping.
- 3 This product is not suitable for pregnant women.
- 4 The recommended wearing time is 2-6 hours, and the wearing time can be gradually increased with the wearing date.
- 5 Please wear it again regularly when wearing it for a long time to prevent skin problems or circulatory disorders caused by long-term pressure.
- 6 When not wearing the orthotic belt, please fix the elastic band on both sides of the front of the orthotic belt to prevent it from sticking to other fabrics and facilitate repeated wearing.
- 7 This product is not suitable for use as a medical device.

Washing Instructions:

- 1 Hand wash only, avoid using bleach, and use a neutral detergent.
- 2 Do not wash with light-colored clothes, as the color may fade or transfer to other fabrics.
- 3 Wash separately from other clothes to prevent wear or fasteners opening.
- 4 Avoid using a dryer or iron, hang to dry in a cool place.

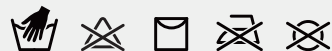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CET Product Service Ltd.