

Your quick guide to using NovoPen® 6 or NovoPen Echo® Plus

Reliable insulin
dose recording
happening here




NovoPen® 6))
NovoPen Echo® Plus))




Smart pens. Smarter care.

Prepare your NovoPen® 6 or NovoPen Echo® Plus

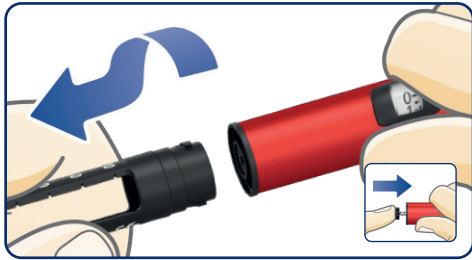


Remove the cartridge holder

Pull off the pen cap. Twist off the cartridge holder. The piston rod may stick out of your pen. If so, push it in until it stops.

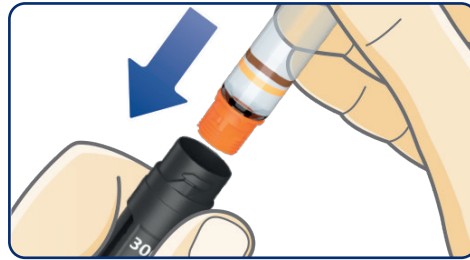

Insert new cartridge

Slide a new insulin cartridge into the cartridge holder with the threaded end first. Twist the cartridge holder back on.

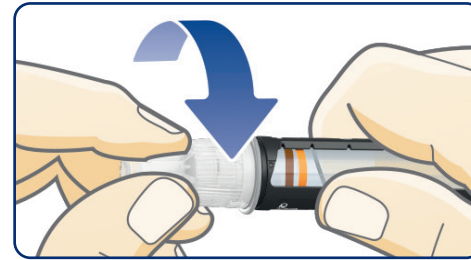

Attach a new needle


Tear off the paper tab. Push the needle straight onto the pen. Turn until it is on tight. Pull off both needle caps. Save the outer needle cap.

Inject



⚠ ALWAYS test the insulin flow

Select 2 units and press the dose button until the dose counter shows 0. The insulin flow test is complete when you see insulin squirt and there is no longer a gap between the piston rod head and the piston.

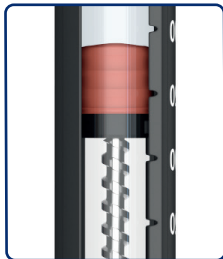

Select your dose

Pull out the dose button. Make sure the dose counter shows 0 before you start. Turn the dose button to select the dose you need.



Inject your dose


Insert the needle. Press the dose button until the dose counter shows 0. Count slowly to 6 before you remove the needle from your skin. Safely remove and throw away the needle. Put the pen cap back on.

Check your last dose



Pull out the dose button

Make sure the dose counter shows 0 before you start.

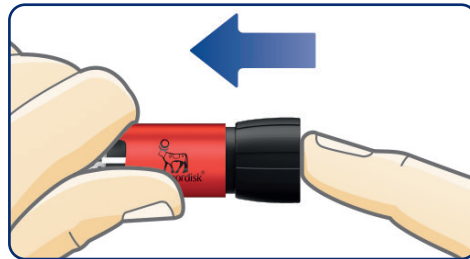

Push the dose button back in

The dose memory will now show the details of your last dose.



Read your dose memory

The example above shows 12 units injected 1 hour, 52 minutes and 3 seconds ago.



Order here!



PRAXISDIENST
Medical Supplies since 1953

Keep a personal injection log

If you have a smartphone, tablet, personal computer, blood glucose monitor or another device that supports Near Field Communication (NFC), you can easily transfer your insulin dosing information from your pen to your device. Just make sure to download a compatible app or other software that can read and display this information.



NFC location may vary by phone model.



NovoPen® 6 and NovoPen Echo® Plus are registered trademarks of Novo Nordisk A/S.
© Novo Nordisk Allé, DK-2880 Bagsvaerd, Denmark.
The N Mark is a trademark or registered trademark of NFC Forum, Inc. in the United States and in other countries.
HQ22NP600036; Approval date: August 2022

NovoPen® 6) NovoPen Echo® Plus)