

ACTIVESHOP®

INSTRUCTION MANUAL

Electric cosmetic chair Sillon Classic 3 motors

Model: C501A



Importer:

ACTIVESHOP Sp. z o.o.
Ul. Graniczna 8b, building
DC2A 54-610 Wrocław,
Poland
www.activeshop.com.pl
Rev. A/04/2024



***BEFORE USING THE PRODUCT, PLEASE READ THE INSTRUCTION MANUAL*.**


This manual contains detailed information on the operation and maintenance of the device. Before using the device, read the enclosed manual. Use the device as described. Keep this manual and make sure that all persons who may use the device are familiar with it.

Sillon electric cosmetic chair is a professional equipment that will be great for any beauty salon. Ideal application for massages, cosmetic and also physiotherapeutic treatments. The chair is designed to provide full comfort while lying on it during the treatment. The chair consists of four parts and a removable cushion. On both sides of the seat are handles on which you can hang the control remote control so that it is always at hand.

Load capacity of the product: 150 kg


The maximum load capacity of the seat body is 150 kg. Do not lift more than the given weight that the chair can support.

Do not jump on the backrest and do not put a lot of pressure on the armrests, so as not to damage the structure of the product.

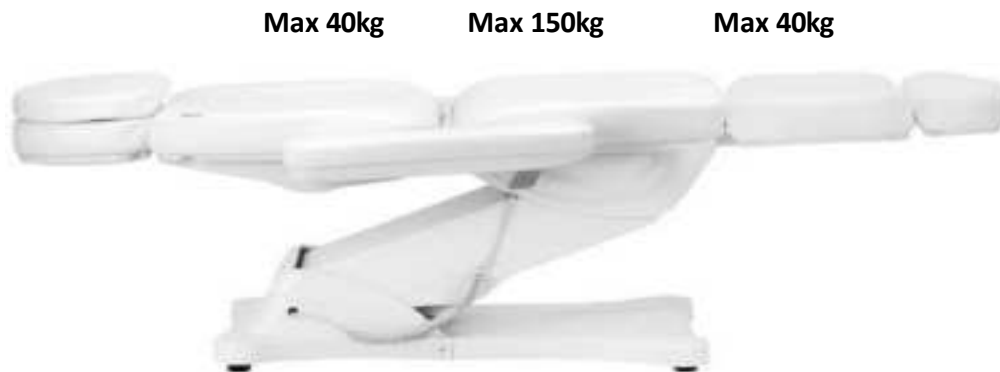

Note

Cardboard and outer cartons are used for packaging, with a total of 10 layers of paper to ensure the protection of your products. We recommend keeping the original packaging for future use, especially for returns during the warranty period. For safety reasons, two people are needed to lift the chair out of the carton. When lifting, stand on both sides of the chair's headrest and footrest. Remember, the chair can only be lifted from the headrest and footrest positions, not from either side of the armrests.

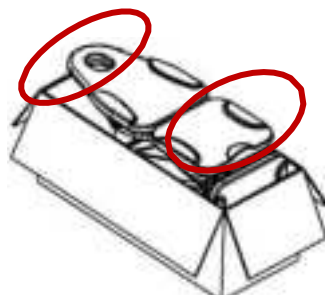

Before using the chair, please check whether the package is intact and has not been damaged during transportation. If there is any problem, please notify the transportation company or

After-sales personnel within 2 days of receiving the device. Make sure all accessories are complete at the time of delivery. If you find missing accessories at the time of delivery, contact the after-sales service in time.

The electric beauty chair must be used in a closed, dry room. The temperature in the room should be from +10° to -40 °, and the relative humidity should be from 30% to 75%. If the electric cosmetic chair has been exposed to temperatures below 0 °, wait at least 5 hours before using it so that the device has time to reach room temperature.

Before using the electric cosmetic chair, read the instructions and perform the following steps:

- (1) Make sure all packaging materials have been removed.
- (2) Make sure the chair is in working order.
- (3) Place the electric beauty chair in the area where it will be used.
- (4) Plug the power plug into a suitable outlet.


Tip! Children and inexperienced persons must not use this device. Children must be supervised and must not play near the electric cosmetic chair. For safety reasons, unplug the device and do not leave it unattended. Also unplug the power cord.

Dimensions of the device:





Construction of the device:

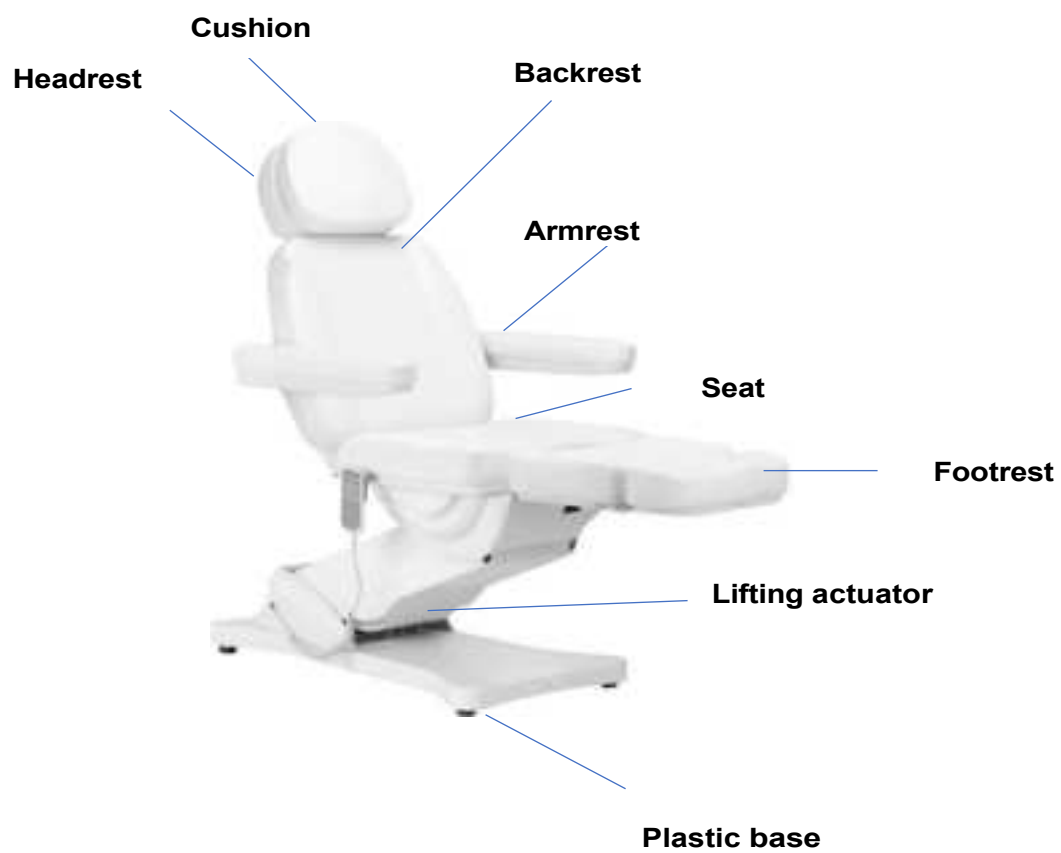

Warning

1. The footrest is not designed to support the full weight of an adult.
2. Do not sit directly on the armrest and footrest.
3. Do not use the electric gynecological examination chair near flammable gases.
4. Do not lift more weight than the chair can support.
5. Make sure the distance between outlets does not exceed 2 meters.

Note

1. For safety reasons, always plug the power cord into a grounded outlet.
2. Do not tie the power cord to the device. Lifting or pulling the power cord may damage it. Make sure the power cord can be easily removed in an emergency.
3. Before moving the electric beauty chair, remove the power cord so that it does not get stuck between some parts of the chair.
4. If the power cord is damaged, immediately disconnect the cord, do not use the chair and contact an authorized service center. Use only the original power cord.
5. Do not let a person who is not performing the procedure control the control remote control.
6. Do not place any objects under the chair.
7. Make sure there is enough clearance around the chair so that it can be folded and unfolded freely.
8. Make sure there are no sharp or harmful objects around the chair that could damage the device.
9. Do not modify the device yourself. If the chair does not work properly, stop using it and contact an authorized service center.

Use of the device:

1. Connect the power cord to the external power supply 110V~240V.

Armrest installation method:

The seat must be in a horizontal position to install the armrest to it. Slide the armrest into the hole located on the side of the backrest. The armrest should be firmly attached. Twist it to the right or left so that it goes into place properly. Do the same with the other armrest.


The armrest is perpendicular to the ground (regardless of the angle of the seat). It can be rotated 180°.




1. Manual control function (lifting height: the bed is 58~ 82 cm from the ground)



2. Manual control function (back lift angle 0-80 °, leg lift angle 0-90 °)

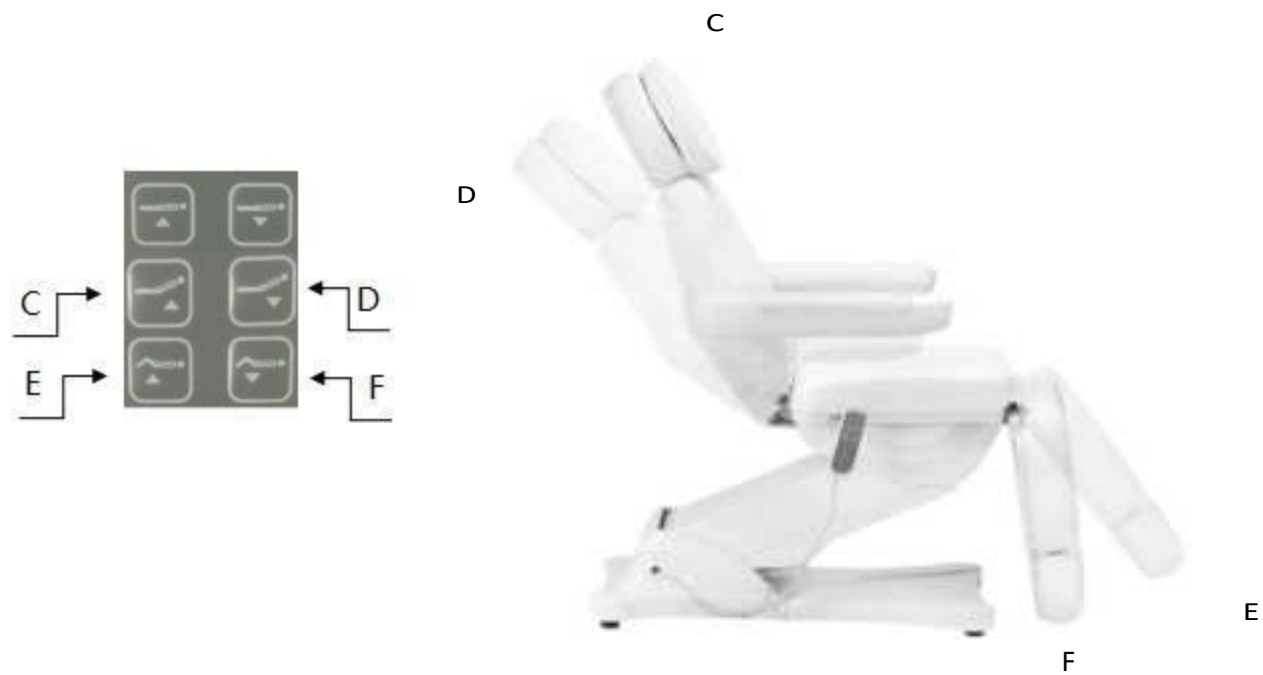

3. Retractable footrest function (footrest, length: 0~ 12cm)

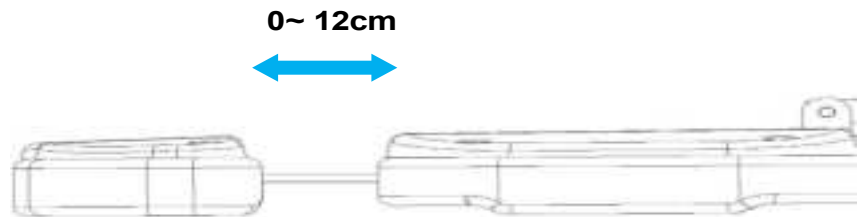

Cleaning and disinfection of the device:

- Unplug the device before cleaning.
- To keep the surface of the chair in good condition, clean it regularly. Remember to clean and disinfect the chair after each use. It is recommended to perform a thorough disinfection once a month.

I Metal and plastic surfaces

- To clean metal and plastic surfaces and controls wipe them with a damp cloth. To clean hard-to-reach areas or corners, a small brush will work well. Do not use too much liquid to clean the device. Then disinfect the device with a suitable liquid.

II Stainless steel surfaces

- Cleaning with a mild detergent solution will work best. Ammonia and most solvents can be used to remove tough stains. Avoid chlorine-based solutions.

III Painted or chrome-plated metal surfaces

- Do not use water jets (showers, high-pressure water cannons) for cleaning.
- Do not clean in high temperature and high humidity, and do not expose the device to conditions that may lead to liquid contamination.
- Do not use solvents and acids for cleaning.

IV Chair surface

- Wipe the surface of the seat evenly with a special leather cleaner, then wipe it with a clean, soft cloth.

Maintenance

Unplug the device before maintenance.

1. Repairs should be performed by experienced and manufacturer-authorized personnel. Repairs by unauthorized personnel may cause injury or damage to the device, for which the manufacturer is not responsible.
2. Use only original spare parts approved by the manufacturer.
3. Do not use the electric beauty chair or accessories if they are not working properly.
4. All maintenance and repair work must be documented.

Maintenance measures

Check the controls and their wiring once a week. Check the condition and function, of the following items at least once every six months:

1. Power cord and its attachment
2. Motor wiring
3. The control box and its wiring
4. Whether accessories are attached
5. Normal operation of the locking system

If you find any defects, stop using the equipment and contact an authorized service center. Only experienced or authorized personnel can open or replace the actuator / control unit.

If any part of the equipment is damaged, unplug the power cord and stop using it. Then notify the service center.

Troubleshooting

Problem	Solution methods
Manual control does not respond	Check if the interface connecting the manual control line is loose (Check if there is a problem with the circuit board) remember to mark the original position of the plug.
Motor does not respond	Switch the position of the motor plug

To replace the control box and order other spare parts, contact the service department, describe the problem before contacting:

1. Model
2. Date of purchase

SAFETY MEASURES

1. The device is a professional equipment for use in beauty parlors. Operation of the device only by qualified cosmetic personnel avoids undesirable consequences during treatment.
2. Do not disassemble the device or attempt to use it for purposes other than those indicated in this manual. All servicing and repairs must be performed only by an authorized service technician (ACTIVESHOP Ltd. company).
3. Do not install or operate the device near liquids, wet places or with wet hands - so that no liquids get into the device.
4. Immediately unplug the device from the outlet and contact the authorized service provided by ACTIVESHOP Ltd. in the following cases:
 - (a) liquid has gotten into the device,
 - (b) the device emits a strange odor, smoke or makes strange sounds,
 - (c) the power cord has been damaged,
 - (d) the device has been knocked over or its housing has been damaged.
5. Do not lay any objects on the cables. Do not install the device where someone may step on the power cord.
6. For safety purposes, unplug the power cord from the outlet each time you finish working with the device.
7. Do not insert anything into the opening of the device. If a foreign object gets into the hole, remove the cord from the outlet and contact the distributor or manufacturer.
- (8) Do not place the device on an unstable cart, shelf or vehicle. If dropped, the device may be damaged.
9. In order to guarantee the established useful life of the device, it is recommended to maintain an interval of one minute before pressing the power button again (between switching on and off and vice versa).
10. The power supply must comply with the specifications on the nameplate. Otherwise, there is a risk of incorrect operation of the device its damage and even burning.

CAUTION:

1. In case of damage to the power cord or other malfunctions, contact an authorized service center - this will avoid danger.
2. This equipment may be used by children at least 8 years of age and by persons of diminished physical and mental capabilities and persons with inexperience and lack of familiarity with the equipment, if supervision or instruction on how to use the equipment in a safe manner is provided so that the associated risks are understood.
3. The equipment is not intended for play by children. Do not allow children to play with the equipment.
4. Do not leave the appliance unattended when children or persons without sufficient knowledge of the use of the appliance are in the vicinity.
5. This electrical appliance is not a toy. Improper use or contact with the electrical appliance may pose a health risk or lead to death, so use and store the appliance out of the reach of children.

DISPOSAL OF THE DEVICE

Dispose of used electrical appliances, fixtures and packaging to raw material reprocessing centers in accordance with environmental protection principles. Do not dispose of electrical appliances in household waste containers. Follow the rules for disposal of this type of equipment. In order to protect the environment

According to Directive 2012/19/EC on waste electrical and electronic equipment, waste electrical equipment should be collected separately, and then handed over to collection points for waste electronic and electrical equipment. At the end of their useful life, as well as old appliances contain full-value recyclable components, so they should be handed over for reprocessing. Therefore, they should be handed over to appropriate collection points, as well as they can also be handed over to ACTIVESHOP Ltd.

STORAGE AND TRANSPORT

The device should be transported at a temperature in the range of (-) 20 °C to (+) 50 °C only in its original packaging. Store the device in a well-ventilated and relative humidity of no more than 75%. Operating temperature from (+) 10 °C to (+) 40 °C. During storage and transportation processes, the device must not be thrown or dropped, exposed to mechanical shocks or atmospheric agents.

