





LUNDERG®

QUICK SETUP GUIDE

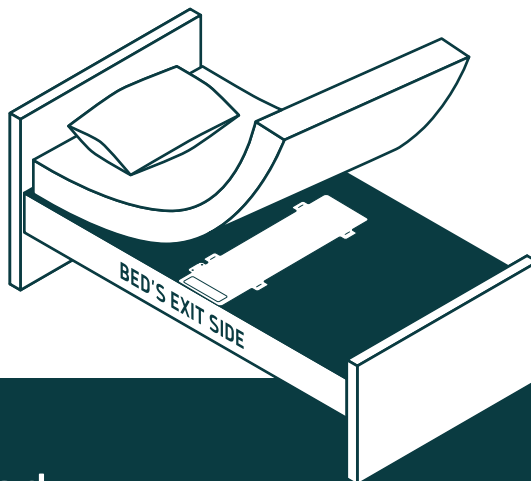

Under Mattress Bed Alarm Replacement Pad

If you have any issue or question, contact
us directly at questions@lundberg.com

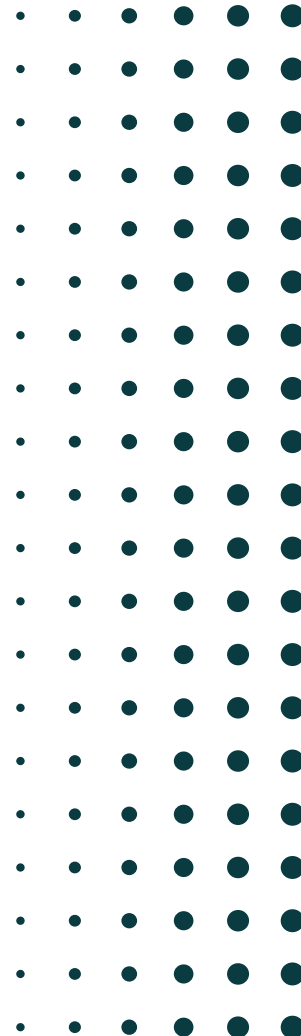

STEP 1

TURN ON THE MONITOR



Install batteries
following +/-



Close cover with
screwdriver



Turn ON

STEP 2


PAIRING



Hold "PAIR" button
for 2-6 sec until
status light turns
STEADY GREEN



Locate pocket
on the bed pad
& open it

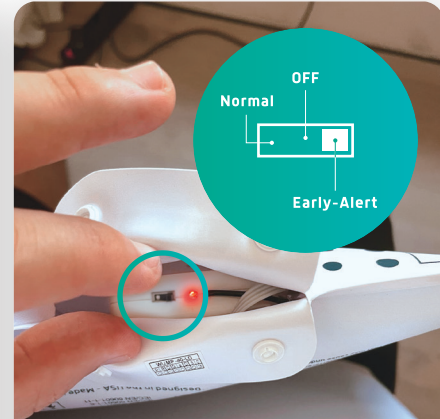

Set to Early-Alert
(right) and close
pocket. Pad will
start beeping!

COMPLETE WITHIN 40 SECONDS!

STEP 3

PAD PLACEMENT



Place bed pad under mattress (Early-Alert side to the edge)



Place mattress down & wait until pad stops beeping (10 sec to 2 min depending on mattress)

If pad was not placed before it stopped beeping, turn OFF bed pad, then ON (right) and repeat this step.

STEP 4

TESTING



Lay down on
the bed



Sit on the edge of
the bed and then
exit bed



If Alarm didn't sound, switch OFF bed pad first,
then go back to step 1, and PAIR the device.



ALARM MUST SOUND

Cancel alarm by
FIRMLY pressing
"STOP" button

READY

THE SYSTEM IS **READY TO USE**

This is just a quick guide to get you started. To unlock the full potential of your system, please read the user manual and check the full instructional video!



Full Guide Video
www.Lunderg.com/underfullvideo

SUPPORT

QUESTIONS?



Contact us at
questions@lunderg.com

We will reply you within
24 hours, 7 days a week!

LUNDERG®



LUNDERG®

Lunderg.com