


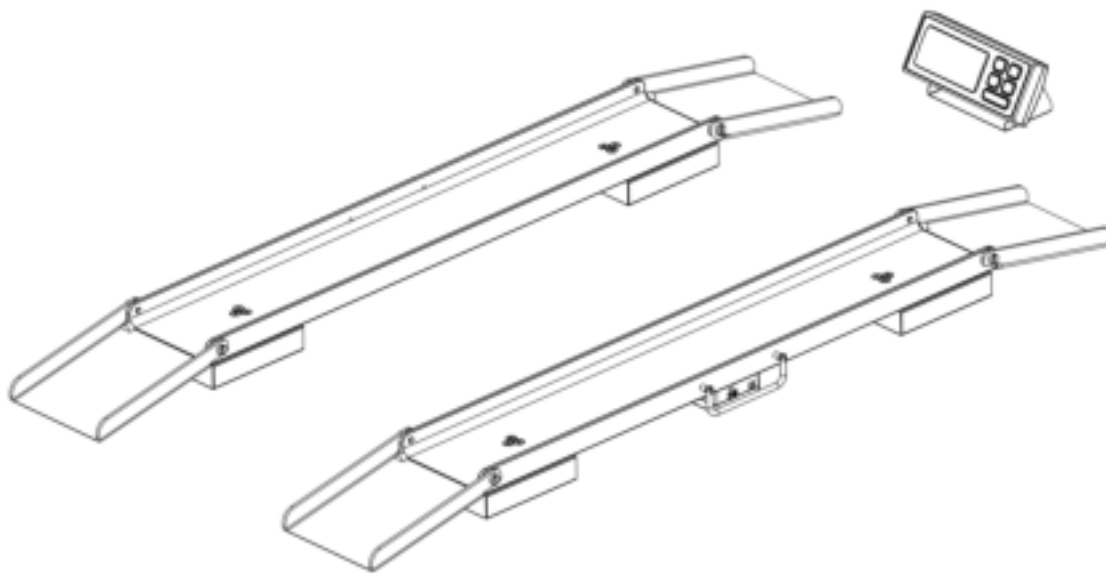

Professional Weighing Equipment
Wireless Wheelchair Scale
Operation Manual
LWC-P Series



Use factory adapter only

**NOTE: THIS PRODUCT HAS BEEN
CALIBRATED IN THE FACTORY.
PLEASE REFER TO THE MANUAL
BEFORE RE-CALIBRATING
PRODUCT!**

Table of Contents

<u>Section</u>	<u>Page</u>
Declaration of Conformity	3
Customer Service	4
Introduction	4
Safety	5
Weight Scale	6
Functions	7
Features	8
Application & Conformity	8
RS232 Data	9
Getting Started	10
Delivery	10
Assembly & Installation	11
Working Mode	12
Cautions	12
Application Menu	13
Program Option	14-15
Calibration	16-17
Maintenance & Service	17
Transportation & Storage	18
Warranty	18

Declaration of Conformity

Declaration of conformity for apparatus with CE mark

We hereby declare that the product to which this declaration refers conforms to the following standards.


Electronic scale: Vet Scales series

Imperial version


Metric version

LWC-M 800

LWC-M 400

Mark applied	EU Directive	Standards
	2004/108/EC	EN 61326-1: 2006

Signature:

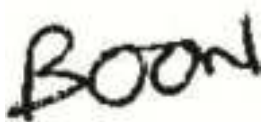

Boon Lim, R & D Manager

Date: 28. 08. 2019

LW Measurements LLC, 620 Carlson Court, Rohnert Park, CA 94928

Customer Service

USA

LW Measurements LLC,
620 Carlson Court,
Rohnert Park, CA 94928
USA

Tel: +1-707-542-2185
Fax: +1-707-542-3285

<http://lwmeasurements.com>

EUROPE

LW Measurements Europe S.L.
Poligono Industrial La Vega, Calle Los Barrios,
Edificio 610, 11380 Tarifa . Cadiz . Spain
Phone : +34 956 682 192
Mobile : +34 680 581 408

<http://www.lwmeasurements.eu>

Introduction

What you should know about these Operating Instructions:

Tree® Professional Weighing Equipment products are simple to operate.

Nevertheless, you should read through these operating instructions in their entirety, so that you can make optimum use of the full potential and the diverse possibilities of the weighing machine in your daily work.

These operating instructions contain guidance in the form of pictograms and keyboard diagrams, which should help you in finding the required information:

For the labelling of potential hazards and advice, please see Safety below.

Safety

Representations and symbols

Important instructions, which involve safety, are highlighted with the appropriate mark:



Safety recommendations

When using the weighing equipment in surroundings with increased safety requirements, the corresponding regulations must be observed.

The weighing scale may only be used with the power adapter supplied. Before connecting the power adapter to the scale, the user must ensure that the operating voltage stated on the power adapter is compliant with the mains voltage. If not, please contact Customer Service at the address above.

If the power adapter or its cable is damaged, the weighing scale must immediately be disconnected from the electricity supply (pull out the power adapter).

If there should be any reason to believe that it is no longer safe to operate the scale, it should be immediately unplugged from the electricity supply (pull out power adapter) and secured against inadvertent operation.

In carrying out maintenance work, it is essential to follow the recommendations set out in maintenance and servicing.

The weighing scale must not be operated in an area subject to explosion risks.

Care must be taken when weighing liquids to ensure that no liquid is spilled into the inside of the scale or into connections on the rear of the equipment or the power adapter. If liquid is spilled on the scale, it must immediately be unplugged from the main electricity supply (pull out power adapter).

The weighing scale may be operated after it has first been inspected by a service technician.

These operating instructions must be read by each user and should be easily accessible at the workplace at all times.


Weight Scale

Construction & Functions

The weighing scale consists of the following parts;

- The Scale body (1)
- The Wireless indicator (2)
- The adapter (3)
- Operation manual.

Figure 2.1 Your weighing machine

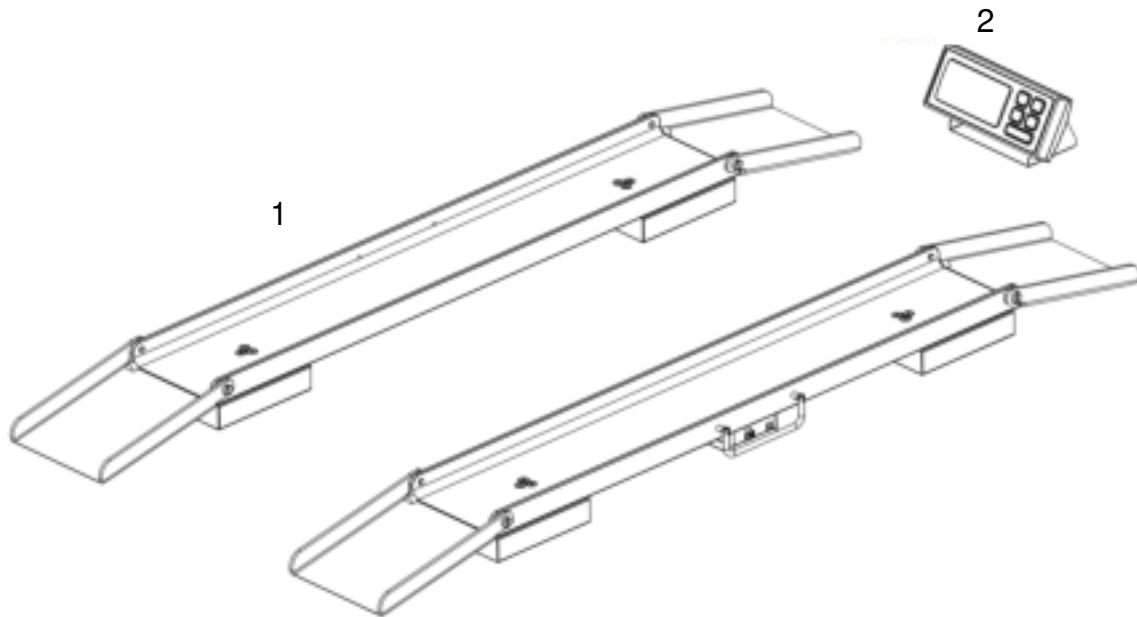

Functions

The Large Wheelchair Scale is a high-quality electronic precision weighing machine with the following specifications

Metric Specifications

Model number	Capacity	Graduation	Platter Size
LWC-P 400	400kg	0.1kg	14.8 x 95 cm
Shipping weight	17.5 kg		
Package (Standard carton)	102 x 26 x 26 cm		
Operating Temp.	5 – 35 C°		
Power source	7.4V rechargeable lithium battery		
	12V/1A AC/DC adapter		

Imperial Specifications

Model number	Capacity	Graduation	Platter Size
LWC-P 800	800lb	0.2 lb	5.8 x 37.4 in
Shipping weight	38.5 lbs		
Package (Standard carton)	40.2 x 10.3 x 10.3 in		
Operating Temp.	40-95 °F		
Power source	7.4V rechargeable lithium battery		
	12V/1A AC/DC adapter		

Features

- Auto backlight
- Auto shut-off
- Wireless indicator
- Display options: 5 1/2 digital LCD
- Peak hold function
- Hold function
- Animal weighing mode
- ZERO range (0% - 4% of full capacity)
- TARE range (4% - 100% of full capacity)
- AnyCal calibration software
- Optional calibration unit: kg or lb
- RS232 connection
- Stability indication
- Auto zero tracking
- USB connection

Application & Conformity

The weighing scale may only be used for the weighing of solid-materials and of liquids filled into secure containers.

The maximum capacity load of the weighing scale must never be exceeded, otherwise the weighing scale may be damaged.

In using the weighing scale in combination with other devices as well as with devices produced by other manufacturers, the appropriate regulations for the safe use of the relevant attachments and their application in accordance with instructions must be observed.

The weighing scale has been manufactured and tested in accordance with the standards and recommendations set out in the declaration of conformity.

The power adapter supplied for the weighing scale complies with the appropriate electrical protection class.

The following applies to LWC-M Scale series

Power supply:

Input: 110 or 230V AC (+/-15-20%); 50 to 60Hz

Output: 12V DC 1000mA

Allowable ambient conditions


Temperature: 5°C - 40°C

Relative humidity: 25% - 85%, non-condensing

If you have any questions on the technical data or require detailed technical information on your scale, please contact your technical representative.

RS232 Data interface

1. Connector: DB9

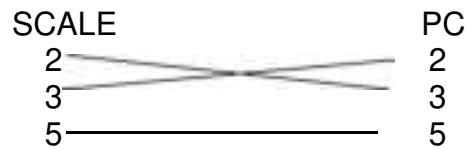

2. Transmissions Settings

Mode: Simplex Asynchronous Serial
Data Bit: 8
Stop Bit: 1

Baud Rate: 9600
Parity Bit: None
Data Format: ASCII

3. Transmission Information Format : 20 Byte , blank=20H

1~2	3	4~13	14~18	19	20
'W: '	P	DATA	UNIT	CR	LF

W: Start of Data Transmission=57H+3AH

P: Polarity '+' = 2BH = Positive '-' = 2DH =Negative

DATA: '12.345' = 20H+20H+20H+20H+31H+32H+2EH+33H+34H+35H

UNIT: 'lb' =6CH+62H+20H+20H+20H

'Kg' =6BH+67H+20H+20H+20H

CR: = 0DH

LF: = 0AH

Getting Started

The scale is packaged in an environmentally-friendly carton, which provides optimum protection for the scale during transportation.

We suggest that you keep the original packaging in order to avoid damage if you are shipping or transporting the scale to a different location. It is also the best way to keep it in the best conditions if it will not be used for an extended period of time.

In order to avoid damage, please follow the instructions provided below, when unpacking the scale:

- Unpack the scale carefully.
- When outside temperatures are very low, the scale should be stored for a couple hours and kept in its box in a dry room at normal temperature, so that no condensation settles on the unit when opening the box.
- Check the scale immediately after unpacking for any external visual damage. If there is any damage on scale, contact customer service immediately.
- If the scale is not to be used immediately after purchase, it should be stored in a dry place where fluctuations in temperature are low. (Reference pg.).
- Read through these operating instruction, before you work with the unit and pay attention to the Safety recommendations (reference Safety pg. 6).

Delivery

Inspect delivery for completeness immediately on unpacking all components.

Checklist for complete delivery

	Component delivered present yes / no
Scale body	
Wireless indicator	
Rubber mat	
Weighing pan	
Power adapter	
Operating manual	

Assembly & Installation

The weighing scale is delivered in a partly dismantled condition. Assemble the individual components in the following sequence:

- Place your machine on a level, clean, and dry surface to obtain accurate readings.

Connecting the AC Adapter

The following Safety recommendations must be observed when connecting the scale to the mains:



The Scale should only be connected to the mains voltage socket with the power adapter supplied. Check before connecting the power adapter to the mains socket, that the operating voltage stated on the power adapter complies with the local mains voltage. If the operating voltage is not the same as the mains voltage, the power adapter must not be connected to the mains socket and contact customer service.


Placement of Scale

The location in which the scale is placed is very important in order for the scale to work to its full potential. Certain conditions can affect the capabilities of the scale, conditions like: the presence of air flow, variations in temperature, and direct sunlight. Please follow the recommendations given below in choosing a location to place your scale.

- Place the scale on a solid, firm and preferably vibration-proof, horizontal base
- Make sure that the weighing machine cannot be shaken or knocked over
- Protect from direct solar radiation
- Avoid drafts and excessive temperature fluctuations
- Avoid placing the scale near or on any magnetic surfaces

Working Mode

1. Press the power switches of the two scales to turn on the power of the scales.

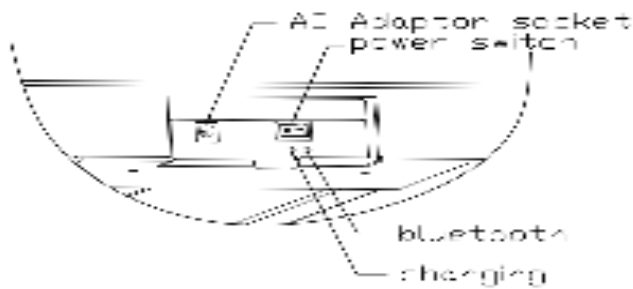

2. Press [ON/OFF] key on the indicator to turn on the indicator. The bluetooth LEDS on the indicator and the scale body both change from flashing to bright, indicating that the indicator and the scale body are connected successfully. The indicator will display a string of self-checking characters from 'PL=== ' to "PL---" "Pr---" "33333" for a few seconds and "-----" then 0.00, now the scale enter the WEIGH mode.

Cautions

1. if you press the [on/off] key on the indicator to turn on the indicator, the bluetooth indicator on the indicator will always flash, and the indicator will always display the self-test character of "PL = = =" or "Pr = = ="

The reason for this phenomenon may be that the power switch of the left scale body or right scale body isn't turned on. Another possibility is that the battery of the scale body is too low, please charge in time.

2. When the sign  is lit, it indicates the indicator is low battery.


When the sign  is Slow flicker it indicates the left scale body is low battery.

When the sign  is Fast flashing, it indicates the right scale body is low battery.

Please charge the battery for 4 to 6 hours. When the charging LED turns from yellow to green, it indicates that the battery charging is completed. Please unplug the charger (adapter) in time, because overcharging will cause battery damage.


3. Only use the charger (adapter) supplied with the scale and indicator.
4. After charging, the battery does not last long, please contact the supplier to replace new battery.

Application Menu

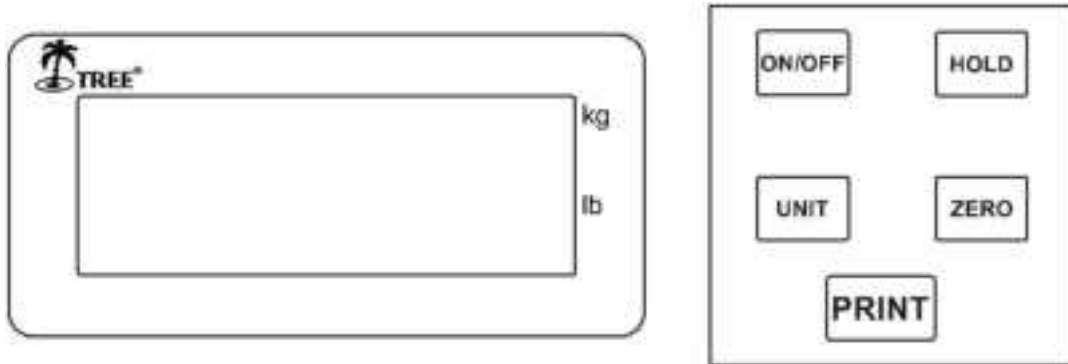

Display messages and key functions


Display messages:

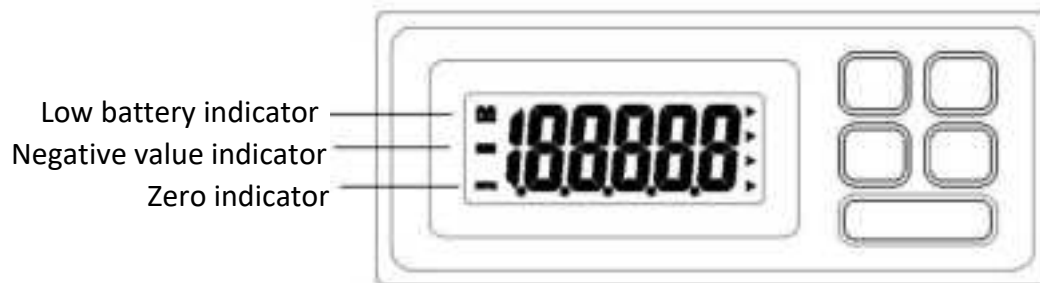

Key Functions:

- The ON/OFF key is to turn the Scale On or Off
- The HOLD key is to store the weight value.
- The UNIT key is to toggle different weighing units.
- The ZERO key: Zero feature if load weight below 4% of full capacity; Tare feature if load weight is between 4 to 100% of capacity.
- The PRINT key is to send data to external device through RS232 connector and USB.

Back view of indicator:

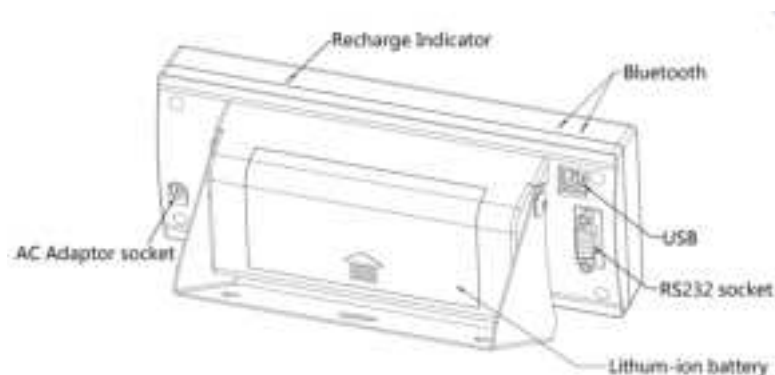

Program options

Interface Settings

- Press and hold the [HOLD] key, then press [ON/OFF] key to turn on the scale, the display will show A-ON or A-OFF.
- Press [UNIT] key the display will show L-ON, L-Au or L-OFF.
- Press [UNIT] key again to enter Parity mode, the display will show P-XXX.
- Press [ZERO] key to select KEY, STB, CON, OFF.
 - a) OFF stands for serial data output disabled.
 - b) KEY stands for manual mode by pressing [PRINT] key.
 - c) STB stands for automatic print when scale is stable
 - d) CON stands for continuous print.
- Press [ON/OFF] key to turn off the scale.

Setting Auto Shut off

- Press and hold the [HOLD] key, then press [ON/OFF] key to turn on the scale, the display will show A-ON or A-OFF, indicating that the scale enters the menu of setting auto shut off.
- Press [ZERO] key to select ON or OFF.
 - a) ON means auto shut off function activate.
 - b) OFF means auto shut off function inactivate.
- Press [ON/OFF] key to turn off the scale.

Setting Backlight

- Press and hold the [HOLD] key then press [ON/OFF] key to turn on the scale, the display will show A-ON or A-OFF.
- Press [UNIT] key to enter next menu, the display will show L-ON , L-OFF or L-Au, indicating that the scale enters the menu of setting backlight.
- Press [ZERO] key to select ON, OFF or Au.
 - a) ON means the backlight is always on.
 - b) OFF means the backlight is always off.
 - c) Au means auto backlight function activate, when the weight isn't zero, the backlight will be on; otherwise, it will be off.
- Press [ON/OFF] key to turn off the scale.

Setting power-on mode

- Press and hold the [HOLD] key then press [ON/OFF] key to turn on the scale, the display will show A-ON or A-OFF.
- Press [UNIT] key three times to the menu of setting the power-on display, and the display will show J-ON or J-OFF.
- Press [ZERO] key to select ON or OFF.
 - a) ON means showing zero when turn on the power.
 - b) OFF means showing weight placed when turn on the power.
- Press [ON/OFF] key to turn off the scale.

Hold function

- Press the [ON/OFF] key to turn on the scale, place the item on the platform.
- Wait for the readings to stabilize, press the Hold” key, the “ . “ indicator will show.
- Remove the item from the platform, the readings will still remain on the display.
- Press the “Hold” key to quit hold function mode.

Bluetooth reconnect

- When reconnect, first reconnect the left scale, and then reconnect the right scale.
1. Turn the switch of the left scale on,
 2. Turn the switch of the right scale off,
 3. Press and hold the [PRINT] key and then press the [ON/OFF] key to turn on the indicator, the display will show ‘PL===’,
 4. Press the [PRINT] key, the display will show AUTOP, “PL---” , The bluetooth LEDS on the indicator and the left scale body both change to bright, indicating that the indicator and the left scale body are bluetooth connected successfully
 5. Turn the switch of the right scale to the on position,
 6. press the [UNIT] key, the display will show ‘Pr===’,
 7. Press the [PRINT] key, the display will show AUTOP, “Pr---” , The bluetooth LEDS on the indicator and the right scale body both change to bright, indicating that the indicator and the right scale body are bluetooth connected successfully
 8. Press [ON/OFF] key on the indicator to turn off the indicator.

Calibration

Using an External Calibration Weight

Calibration is generally required when the weighing machine is initially installed or if the weighing machine is moved to a substantial distance from the original location. 30 minutes of warming up of the scale is always needed before initial calibration.

A reminder when operating our larger capacity scales:

The scales of the LWC-M series are calibrated at the factory. Usually no calibration is necessary unless major inaccuracies are observed. If calibration is deemed necessary, an appropriate test weight of at least 1/3 of the full capacity of the scale should be used to obtain accurate weighing.

Calibration procedure:

When calibrating, first calibrate the left scale, and then calibrate the right scale. During calibration, the uncalibrated scale should be shut down. press the [on/off] key on the indicator to turn on the indicator,

1. Turn the switch of the left scale on,
2. Turn the switch of the right scale off,
3. Press and hold the [ZERO] key and then press the [ON/OFF] key to turn on the indicator, the display will show CAL-L,
4. Press the [HOLD] key, the display will show L0XXX calibration weight, press the [UNIT] key to select unit of measurement kg. or lb. press the [PRINT] key to move the flashing digit to right, press the [ZERO] key to increase the value of the flashing digit, press the [HOLD] key to confirm. (We recommend a calibration weight at least 1/3 of the full capacity to get an accurate weighing)
- 5.the display will show CAL and then the AD value, press the [UNIT] key when the stable indicator displayed, it will show the flashing calibration weight.
- 6.Place the known test weight on the platform L, press the [UNIT] key after stable indicator displayed, it will show ----, and then will show CAL-L, the left scale calibration is complete.
7. Turn the switch of the left scale to the off position, remove the test weight from the platform L.
8. Turn the switch of the right scale to the on position,
9. Press the [ZERO] key, the display will show CAL-r,
10. Press the [HOLD] key, the display will show r0XXX calibration weight,
- 11.press the [HOLD] key to confirm. the display will show CAL and then the AD value, press the [UNIT] key when the stable indicator displayed, it will show the flashing calibration weight.

12. Place the known test weight on the platform, press the [UNIT] key after stable indicator displayed, it will show ----, and then will show CAL-r, the right scale calibration is complete.

Press the [ON/OFF] key to turn the scale on to test if the weighing is accurate, if not, repeat above steps.

Maintenance and service


The weighing machine must be treated carefully and cleaned regularly. It is a precision instrument.



For maintenance-work, the scale must be disconnected from the power supply (remove power adapter plug from socket). Also ensure that the scale cannot be connected to the power supply during the work by a third party.

Make sure that no liquid spills into the scale while performing maintenance work. If liquid is spilled on the scale, it must be inspected by a service technician.

Regularly perform maintenance to the weighing pan and the weighing pan holder by removing any dirt or dust from under the weighing pan and on the weighing scale housing. Use a soft brush or a soft, lint-free cloth, moistened with a mild soap solution.

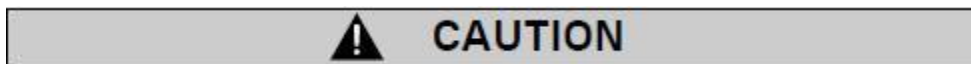

Never use solvents, acids, alkalis, paint thinners, scouring powders or other aggressive or corrosive chemicals for cleaning; these substances can cause damage to the surfaces of the scale housing.

Transport & Storage

Your weighing machine is a precision instrument, treat it carefully. Avoid shaking, severe impacts and vibration during the transportation. Make sure that there are no marked temperature fluctuations during the transportation and that the weighing machine does not become damp (condensation).

If you would like to take the weighing machine out of service for an extended period, disconnect it from the electricity supply, clean it thoroughly (refer to Maintenance & Service) and store it in a place which meets the following conditions:

- No violent shaking, no vibrations
- Minimum temperature fluctuations
- No direct solar radiation
- Minimum moisture

Warranty

The products are under warranty against factory defects for a period of two (2) years from the date of shipment.

For Customers within the lower 48 states of the continental United States. LW Measurements will pay for freight both ways for the first 30 days after purchase. After 30 days expire the customer is responsible for shipping the product back to us. After the product is received we will inspect it and as necessary we will repair or replace and will ship the product back to the customer at our expense.

Any new scales returned for warranty must be properly packaged in the original box. If they are not properly packed in the original box, the customer pays for shipping cost. If we determine there is a factory defect, we will pay for the shipping back. If we determine that it is not a factory defect, the customer will pay shipping.

For Customers outside the lower 48 States, including Mexico, Canada, Puerto Rico, Hawaii, Alaska and all other countries, customers must pay for shipping.

Our warranty does not cover misuse or neglect including but not limited to battery or water damage, overloading, and chewed or cut wires. If the product is found to have been misused or damaged by the customer, LW measurements is not responsible for the cost of return.

For warranty claims please go online to lwmeasurements.com and fill out the warranty submission form or call your customer service representative.