BIOTRACE™ Auto Read Pro Biological Indicator Reader (73400)

QUICKSTART GUIDE


161373-01 Rev. C
2024-06

REF 73400

R3

- Features
- Caractéristiques
- Eigenschaften
- Caratteristiche
- Características
- Características
- Funktioner
- Funksjoner
- Funktioner
- Ominaisuudet
- 特徴
- özellikler
- функции
- 특징

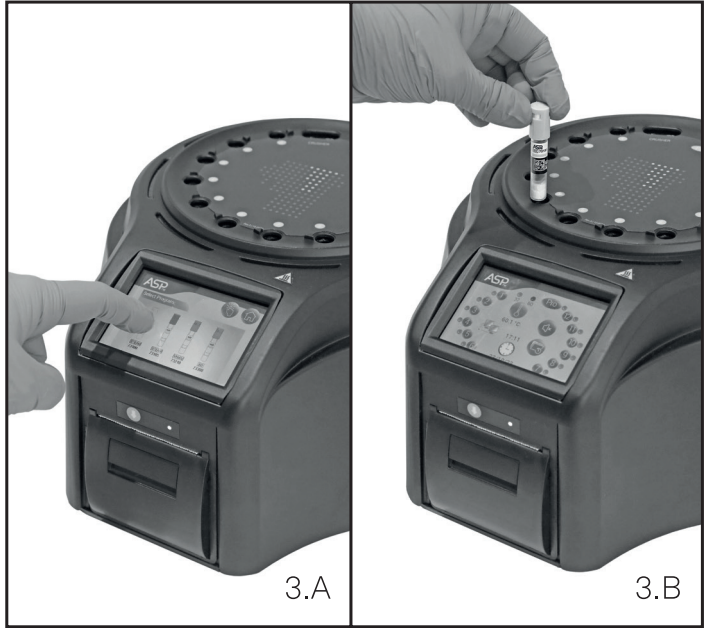BIOTRACE™ Auto Read Pro Biological Indicator Reader (73400)

Features

- Protective Cover
- Incubation Area for 12 SCBIs and 1 Protein pen. Ampoule Crusher
- LCD Touch Screen Display
- Thermal Printer
- Printer's paper indicator light
- Paper Traction Button
- Cavity for paper
- Input for power source supply (12 Volts DC)
- Hole for External Temperature Control
- Ethernet port (RJ45)

Main screen features

- PRO Configuration Button (reserved for future use)
- SCBI Position Configuration Button
- Position status Indicator
- Date and Time Indicator
- Remaining Incubation Button
- Alarm Cancellation Button
- Incubation Temperature Selection Button
- Incubation Results History Button
- Temperature Stability Indicator

ASP Information Button

- Network connection indicator / connection indicator

Product description

BIOTRACE™ Auto Read Pro Biological Indicator Reader (73400) has been designed for the incubation and automatic readout of Advanced Sterilization Products, Inc. (ASP) Self-Contained Biological Indicators (SCBI) appropriate for sterilization and environmental disinfection process control applications. BIOTRACE™ Auto Read Pro Biological Indicator Reader (73400) allows two different incubation temperatures, 37 °C (98.6 °F) or 60 °C (140 °F). Microorganisms contained within the SCBIs vary depending on the sterilization process for which they have been designed, therefore, incubation temperatures and incubation times differ depending on the SCBI being used. BIOTRACE™ Auto Read Pro Biological Indicator Reader (73400) allows the selection of independent incubation times for each of the positions. BIOTRACE™ Auto Read 4H EO Biological Indicator (73300) should be incubated at 37 °C (98.6 °F), while those used for the monitoring of the rest of the sterilization processes should be incubated at 60 °C (140 °F).

Note: Simultaneous incubation of SCBIs for monitoring Ethylene Oxide (EO) processes along with SCBIs for monitoring other sterilization processes is not possible.

BIOTRACE™ Auto Read Pro Biological Indicator Reader (73400) allows easy and rapid detection of positive and negative SCBIs, using advanced fluorescence techniques. A fluorescence result can also be visually evidenced by culture medium color change when performing extended incubations. Due to the high sensitivity of the fluorescence results, the option to perform or not an extended incubation depends on the internal protocols of each laboratory or hospital. Read the instructions for use of the SCBI for more information.

BIOTRACE™ Auto Read Pro Biological Indicator Reader (73400) also provides a printed ticket each time an incubation is completed for recording the results. This allows for easy result management, for documentation compliance and safe-keeping.

Indications for use

United States of America

BIOTRACE™ Auto Read Pro Biological Indicator Reader (73400) incubates at 60 °C and 37° C and read the BIOTRACE™ SCBI for fluorescent results at the times prescribed in the User Manuals.

Outside the United States

BIOTRACE™ Auto Read Pro Biological Indicator Reader (73400) incubates at 60 °C and 37° C and read the BIOTRACE™ SCBI for fluorescent results at the times prescribed in the User Manuals.

Instructions for use

Setting Up your Auto Read Device

Place the BIOTRACE™ Auto Read Pro Biological Indicator Reader (73400) on a firm surface, free from vibrations, away from direct sunlight, currents of hot or cold air, chemical and corrosive or flammable substances. Do not place the Auto Read Device in a way that disconnection of the power supply's AC plug from mains could be difficult. Leave a space of at least 10 cm from the Auto Read Device to the closest wall. Do not move the Auto Read Device periodically or during its use. Connect the Auto Read Device to a secure and stable electrical mains outlet socket. Do not wet or heat the Auto Read Device. If liquid is spilled on the device, disconnect it and dry it immediately. Before turning on, verify that all reading positions are empty.

Using your Auto Read Device

1. Turn on your Auto Read Device by connecting the AC plug of the power supply to the mains and the jack at the other end of the power supply to the rear of the Auto Read Device. (See image 1)

2. Press the temperature selection button for 3 seconds to select the desired temperature. (See image 2)

3. Wait until the temperature stability indicator stops blinking.

4.A.1. To select the incubation program press the button of the corresponding position. Then press the configuration button. A screen will display the different options. Select the SCBI that you are planning to incubate. (See image 3.A)

4.A.2. Place the SCBI in the chosen position. The incubation will start automatically. (See image 3.B)

5. Wait until the incubation is completed. The position status indicator light will turn green to indicate a negative result or red to indicate a positive result. If a positive result is detected, press the alarm button to cancel the alarm. Do not remove the indicator until the ticket with the result of the incubation is printed.

Disposal: Dispose of the SCBIs in accordance with your country's waste-disposal guidelines.

Connectivity

To connect your Auto Read Device to a personal computer (PC) via Ethernet, connect an Ethernet cable with a RJ45 connector to the rear of the Auto Read Device, then connect the other end of the cable to an Ethernet switch or router. Afterwards, connect your PC to the same Ethernet switch or router using a suitable Ethernet cable. Connecting the Auto Read Device to a PC allows the use of the SM Cloud™ Web Application. Please refer to the User's Manual for more information.

BIOTRACE™ Auto Read Pro Biological Indicator Reader (73400) incubation programs

The following incubation programs are available for your Auto Read Device.

Temperature	Time	Incubation Program
60 °C	20 minutes	20 minutes at 60 °C
60 °C	30 minutes	30 minutes at 60 °C
60 °C	1 hour	1 hour at 60 °C
37 °C	4 hours	4 hours at 37 °C

To select the appropriate incubation program for the Indicator you are planning to use, follow each Indicator's Instructions for use.

Note: Select the appropriate incubation program before the incubation of any Indicator.

Troubleshooting

Refer to the user manual for a detailed troubleshooting routine. Please visit efu.asp.com or scan the QR code to access the Auto Read Device user manual.

You may see one or more of these symbols on the packaging or labeling of this product

- Humidity limitation
- Temperature limit
- Caution
- Consult instructions for use or consult electronic instructions for use
- Caution, hot surface
- CE Mark
- Direct current
- For indoor use only
- Keep away from sunlight
- Batch code
- Manufacturer
- Recycle electronic equipment

For a glossary of symbols used, go to efu.asp.com

Safety Information

To avoid risks and/or damaging the device:

For indoor use only.
Do not place the Auto Read Device in a room exposed to direct sunlight or to high luminous intensity lamps.
Do not place the Auto Read Device near any electrical device that emit strong electromagnetic waves.
Do not use the Auto Read Device on leaned surfaces or on surfaces that are subject to shocks, vibrations, temperature or high relative humidity.
Disconnect the power cord before cleaning.

Do not use abrasive, cleaning or disinfectant cleaners.
Do not immerse into any liquid. Do not pour any liquid inside.
Make sure the Auto Read Device is connected to an appropriate electrical mains outlet socket.
Use only the power supply (AC power adapter), power supply AC plug, and power supply cords. Verify that all the included elements are in good condition on daily basis. If any of them is damaged, discontinue their use. The use of cables, adapters, cords, and/or power supplies different from the ones included may cause fires, electrical shocks, or even physical injuries.
Do not plug any device into the Auto Read Device's Ethernet port other than devices compliant with IEC 60950-1, IEC 62368-1 or comparable, with safety extra-low voltages on its Ethernet ports. Ask a qualified technician to verify device compatibility. Attaching any other device to the Ethernet port may damage the Auto Read Device and may not be safe for the user.
Do not attempt to repair the Auto Read Device by yourself, that could lead to major and irreversible damages to the device. In case of device malfunction, contact your local distributor for further assistance.

To reduce the risk of using incompletely sterilized loads:

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To avoid the risk of injury, related to glass fragments produced when crushing the glass ampoule inside the SCBI tube:
Cool the SCBI during the indicated time before crushing the ampoule.
Do not handle the SCBI excessively since this might cause the glass ampoule to burst.
Wear safety gloves and glasses when removing the SCBI from the sterilizer, pressing the SCBI's cap, and crushing the SCBI's ampoule.
Do not use your finger for crushing SCBI. Use the Ampoule Crusher instead.

To avoid a potentially hazardous situation:
Avoid contact with the hot metal block inside each incubation position.
Do not insert your fingers, or any other element, inside the incubation positions.
Place only compatible indicators inside the incubation positions.

To avoid SCBIs from absorbing fluorescent particles:
Avoid direct contact between the SCBIs and Chemical Indicators or Tapes before the SCBI incubation.
Note: Do not use this product in a manner not specified by ASP, otherwise the protection provided by the product might be affected.
Only ASP authorized personnel can access or service the internal components of the Auto Read Device. Parts or components inside the Auto Read Device should not be manipulated by the user.

Operating conditions
Power supply specifications

Input parameters
Voltage range (100-240)
Frequency 50/60
Current 1
Output parameters
Voltage 12
Current 3

Units
AC Volts
Hertz
Amperes
Units
DC Volts
Amperes

ASP recommends the use of uninterruptible power supply (UPS) instead of voltage stabilizers, since they fulfill two functions: to stabilize and maintain the energy during a power outage.

Environmental operating conditions

Environmental conditions
Altitude 3500 (maximum)
Operation temperature 10-30 (50-86)
Relative Humidity 30-80
Installation/Over-voltage Category II
Degree of contamination 2
Storage temperature 5-40 (41-104)
Voltage 12

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Units
Meters
Celsius (Fahrenheit)
%

Device safety compliance
BIOTRACE™ Auto Read Pro Biological Indicator Reader (73400) complies with the following standards and directives:

Electrical Safety
IEC 61010-1
IEC 61010-2-010
Low Voltage Directive 2014/35/EU
EN 61326-1

Electromagnetic Compatibility (EMC)
EMC Directive 2014/30/EU
RoHS Directive 2011/65/EU
WEEE Directive 2012/19/EU

European Commission
EMC Directive 2014/30/EU
RoHS Directive 2011/65/EU
WEEE Directive 2012/19/EU

BIOTRACE™ Auto Read Pro Biological Indicator Reader (73400) does not represent photobiological risk and does not generate dangerous optical radiation in any of its normal operation conditions as per the requirements of IEC 62471 Standard. Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.

Cleaning and maintenance
Cleaning and decontamination of external surfaces
Disconnect the power supply cable and the Ethernet cable from the Auto Read Device. If the device is hot, wait until it has cooled down before handling it.
Clean the external surfaces of the device with a microfibre cloth moistened with a solution of mild dish washing detergent and water. Wring the cloth so it is damp but not dripping before cleaning, and wipe the outer surfaces of the Auto Read Device. Afterwards, moisten a clean microfibre cloth with water only, and repeat the procedure until all traces of detergent are removed from the external surfaces of the device. After cleaning, allow the Auto Read Device to dry for at least 1 hour before connecting the power supply cable or the Ethernet cable again.

This cleaning procedure can be followed whenever considered appropriate.
This cleaning procedure must be followed every time a spill occurs on any of the external surfaces of the Auto Read Device. If further cleaning is required, or if you have doubts about the cleaning agents you may use, please contact your local distributor.

Note: Do not clean the internal parts of the device.

Note: Do not pour or immerse the device into any liquid. Do not allow any liquid to run inside the device during its cleaning.

Maintenance

The Auto Read Device does not require routine maintenance.

Warranty

Products are warranted to be free from defects in materials and workmanship when properly installed, maintained and used for their intended purpose as set forth in the applicable product labeling and/or the User's Guide. The warranty only applies to the original purchaser.

Term

The warranty period for the BIOTRACE™ Auto Read Pro Biological Indicator Reader (73400) will be 1 (one) year from the date of installation, and may never be extended beyond 5 (five) years from the product's date of manufacture.

Limitation of liability

ASP shall not be held liable for any loss or damage that result from the unsuitable use of the equipment, negligence or user's full responsibility.

Technical assistance

If you are located in the United States and have questions about the BIOTRACE™ Auto Read Pro Biological Indicator Reader (73400), please call ASP Professional Services at 1-888-783-7723. Internationally, call your local ASP Customer Support Representative. You may also wish to visit our website at www.asp.com

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