



CPAPMAN.COM
BEST SERVICE SINCE 1997



X-PLOR₂

CARTRIDGE REPLACEMENT

By:  belluscura®



Cartridge Replacement

PART 1

Removing Cartridge.



1 Turn off the X-PLOR® by pressing the power button for 2 seconds.



2 Remove the X-PLOR from the carrying bag, if applicable.



3 Remove the battery from the X-PLOR.



4 Place the X-PLOR on its side to access the underside of the POC.



5 Pull the cartridge out of the POC by pulling on the cartridge pull handle.




6 Remove the cartridge completely from the X-PLOR and throw away.



Cartridge Replacement

PART 2

Installing Cartridge.

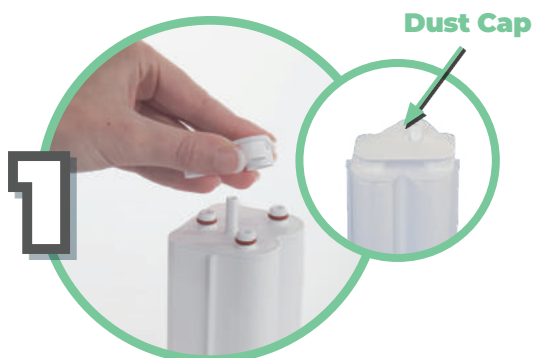

Remove cartridge from mylar bag. The new cartridge has a dust cap on top. Remove the dust cap. Make sure there is no dust or debris where the dust cap was located.



Insert new cartridge top first into the X-PLOR. Do not leave the cartridge ends exposed; the cartridge should be inserted into the X-PLOR as soon as the used cartridge is removed.



Push the cartridge into the X-PLOR so it is fully inside.



The cartridge pull handle should lay flat once the cartridge is secured.



Connect charged battery to the X-PLOR.



Turn on the X-PLOR by pressing the power button for 2 seconds. Set the flow setting to your prescribed setting and use normally.

