

# Getting started with the CareLink™ Connect app

Compatible phone? Check your device at <https://www.medtronic-diabetes.com.au/mm780g-supported-devices>



GET IT ON  
Google Play

Download on the  
App Store

1. Open the app and follow the on-screen instructions. You may be warned that your device is untested but can still continue

2.



Log into your CareLink™ Personal care partner account. Sign up if you do not already have one

**Note:** Your country must be the same as the person you will be following

4.



Enter the CareLink™ Personal username of the person you wish to follow\*

They must accept your connection request in their CareLink™ Personal account within 24hrs



3.



Read the Warning screen and accept by tapping Next

5.



Once accepted, you can set up your notification settings in the menu. You will not receive any notifications\*\* until these are set



Automatic updates to mobile operating systems should be deactivated – if one becomes available, check your local website to see if it is supported before installing

Representative patient profile, Google Play is trademark of Google LLC and App Store is a trademark of Apple Inc.

\*See local website for supported devices - must be using and logged into the MiniMed Mobile app

\*\*Internet connection required for live data and notifications - Phone screens may look different depending on the pump system and mobile phone used.

# Notifications


Notifications are colour coordinated:

**Alarms** are red

**Alerts** are yellow

Reminders are grey


**Messages** are blue

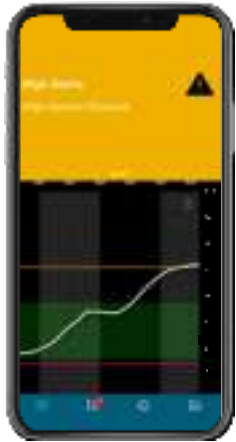

## Important Information:

- Notifications must be set in the Settings menu before any will be received.
- Having your phone on **Do Not Disturb, Mute** or **Vibrate** may cause you to miss notifications
- The CareLink™ Connect app lets you follow one person per account. There is a help section in the Main menu


## Repeat notifications

- Some notifications can now be set to repeat when received at certain times you specify if they have this symbol 
- The notification sound can be personalised and will repeat every 60 seconds until the app is viewed



# Navigating the app




 Home Screen



 History



 Time in range



 Device Status

# Notification privacy

You can select if you want to see health related information in your notifications or have it hidden.




Privacy option switched on:



Or

Privacy option switched off:

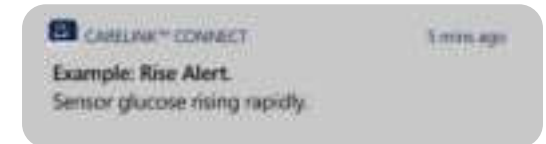

## ALWAYS FOLLOW THE DIRECTIONS FOR USE

12050-122022 <https://www.medtronic-diabetes.com.au/>

Medtronic Australasia Pty Ltd, 2 Alma Road, Macquarie Park, NSW 2113, Australia. Tel: +61 2 9857 9000, Fax: +61 2 9889 5167, Toll Free: 1800 777 808

Medtronic