
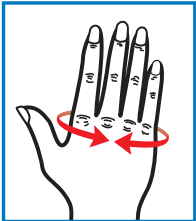
Comfort Cool® Thumb CMC Abduction Orthosis Instructions

Instructions for care Hand wash mild detergent in cool water and air dry.
Instrucciones para el cuidado Lavar a mano con un detergente suave en agua fría y dejar secar al aire.
Instrucciones de soins Lavez à la main avec un détergent doux et à l'eau froide, et faites sécher à l'air.
Pflegeanleitung Mit mildem Reinigungsmittel in kühlem Wasser waschen und an der Luft trocknen lassen.
Skötselanvisningar Handtvättas i svalt vatten med ett mild tvättmedel. Lufttorkas.

NC79490	NC79491	XXS	4½" – 5½"	(11 – 14 cm)
NC79492	NC79493	XS	5½" – 6½"	(14 – 17 cm)
NC79494	NC79495	S	6" – 7"	(15 – 18 cm)
NC79494-P	NC79495-P	S+	6½" – 7¼"	(17 – 18 cm)
NC79496	NC79497	M	7" – 8"	(18 – 20 cm)
NC79496-P	NC79497-P	M+	7¼" – 8½"	(18 – 21 cm)
NC79498	NC79499	L	8" – 9"	(20 – 23 cm)
NC79498-P	NC79499-P	L+	8½" – 9½"	(21 – 24 cm)

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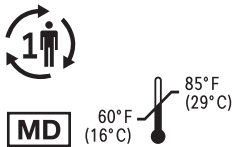North Coast Medical, Inc.
135 E Main Ave, Suite 110
Morgan Hill, CA 95037 – U.S.A.
EC REP ICON (LR) Limited
South County Business Park
Leopardstown, Dublin 18
D18 XSR3, Ireland
UK REP ICON (LR) Limited
500 South Oak Way
Green Park, Reading
RG2 6AD, United Kingdom
UK CA
CH REP Allenspach Medical AG
Dornacherstrasse 5
4710 Balsthal, Switzerland
CHN/156

Thumb CMC Abduction Orthosis
Órtesis para abducción CMC del dedo pulgar
Orthèse en abduction CMC de pouce
Daumen-CMC-Abduktionsorthese
Abduktionsortos för tummens grundled

Helps to treat thumb contractures and conditions, especially where the thumb is held abducted against the palm.
Not made with natural rubber latex.

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Intended Purpose: Provides support to the thumb and CMC joint for pain relief, stiffness, and inflammation.

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Indications: Thumb deformity and contractures due to neurological conditions, median nerve injuries, arthritis, medial subluxation of the thumb CMC, joint instability, sprain, tendinitis, or repetitive motion injuries.

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Contraindications: Loss of circulation and skin irritation or allergy to polyester, nylon, neoprene, or spandex.

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- Slide thermoplastic insert **fully** inside strap's pocket (1A). Trim insert if needed. Carefully heat orthosis in microwave in **20 second intervals** until insert is soft (1B).

Microwave CAUTION: Insert piece will be very hot and sticky and cannot be removed after heating. Any exposure of thermoplastic insert may result in burning the skin.

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Optional heating method: Immerse orthosis strap containing plastic insert into 160°F (71°C) water until soft. Proceed to instruction number 2 for fitting.

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- Secure orthosis onto the hand with thumb sliding through the thumb hole and secure the hook and loop. Place thumb in comfortable abduction/extension. Align warm, pliable strap over the CMC joint. Pull the strap firmly, following palm's curved hook fastener to apply supportive tension along the first metacarpal (2).

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- For greater thumb tip motion, trim along the thumb edge. DO NOT cut at or below the sewn seam stop (4).

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- Bring strap through web space without stretching it. Fasten end to small hook piece (3). Hold thumb in place until insert hardens. Insert can be reheated and reshaped for progressive positioning if needed.

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