# Etac My-Loo

med kantstopp / with brackets / mit Klammern

1A15172.3 2022-10-07

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
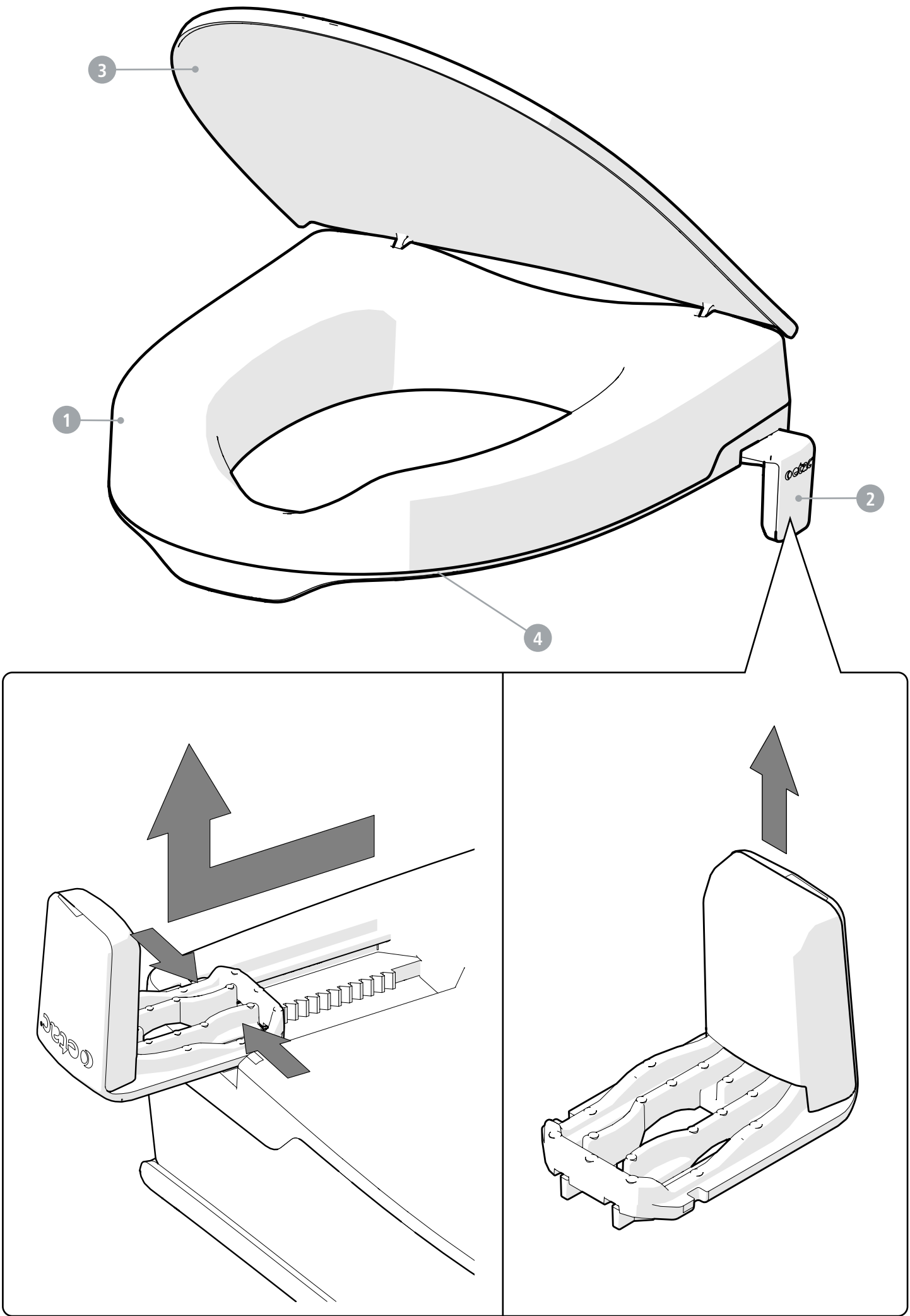
General

- The product is suitable for reuse.
  - Original spare parts from Etac should always be used.
  - Remember that the service life of the product is limited (see
- manual). This is particularly important when prescribing the product to a user who is expected to use it for an extended period of time.

Washing

- Clean the product with normal nonabrasive cleaning agents (pH value of between 5-9). Rinse and dry.
  - Disinfect the product with an alcohol-based disinfectant, 70%.
- The product can withstand cleaning in a sterilizer at 85 °C for 3 minutes and can be mechanically dried at 70 ° for 10 minutes.
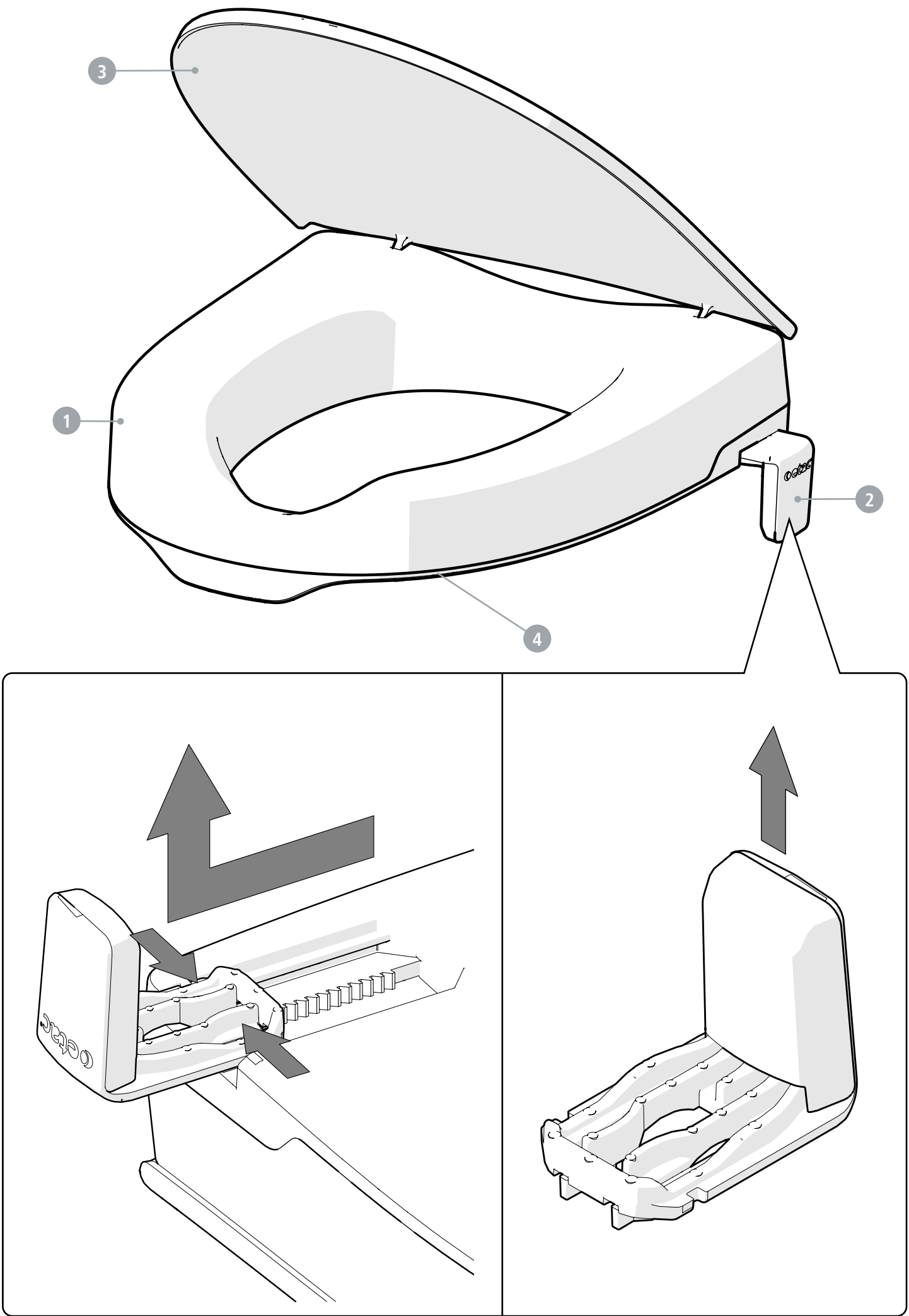
| 1                                                                                                                                | Raiser          |
|----------------------------------------------------------------------------------------------------------------------------------|-----------------|
| Check so that there are no cracks or damages.                                                                                    |                 |
| 2                                                                                                                                | Edge stops      |
| Check so that the snap function is ok and that the edge stops can be adjusted.                                                   |                 |
| Check so that there are no cracks or damages.                                                                                    |                 |
| When cleaning in sterilizer, disassemble the edge stops and clean in upright position (see illustration) at 85 °C for 3 minutes. |                 |
| 3                                                                                                                                | Lid             |
| Check so that the lid is properly fixed and can be folded.                                                                       |                 |
| Check so that there are no cracks or damages.                                                                                    |                 |
| 4                                                                                                                                | Friction pieces |
| Check so that the friction pieces are properly fixed and that there are no damages.                                              |                 |

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