

Instruction for use – Wrist braces

Caution: recommend that the product be fitted and adjusted by a licensed or otherwise qualified health professional. Users with diabetes, sensory loss, skin, known contact allergies, ingested acids, or poor circulation must be extra cautious and check for pinching, chafing, or problems when using the brace. Always check material content below. Contact your prescriber or CalliFi Questions about the product or use.

Area of use

This brace supports the lower hand, wrist and forearm during any type of activity, especially those that require precision. It is to be used when the need for precision or protection from or against an injury exists.

Application

1. The brace must be straight, split, level and fit the wrist, forearm and hand. It must accommodate shape and angle of the wrist, hand and forearm. Place it back in the center point. Split part must be lined in towards first joint. Picture 1
2. Open all loops and straps. It is also the hand strap. It is not put on in the brace and attached to picture 2 or 4.
3. Close all back and loop straps and set and form so that the brace fits comfortably. It is not to be



Out of brace. Look and loop straps for adjustment of pressure approximately. Adjust it to use and loop straps when necessary. It is important that the brace fit comfortably without pinching or chafing. Select fitted strap sizes.

Material

Straps: nylon, elastane. **Split:** cotton, neoprene, elastane. **Comfort:** cotton, neoprene, polyester, elastane. **Neoprene:** used for forearm, palm side, palm side. **Soft:** polyester, elastane, elastane, neoprene, foam, foam.

Washing instructions

Clean the back and loop straps and forearm first joint. If first joint has one from the split pocket. Fit the wrist and loop straps. Use a warm, dry towel. Let the brace dry.



Size

Size	Dimensions	Size	Dimensions	Size	Dimensions
XS	10-12 cm	M	14-16 cm	L	18-20 cm
XS	12-14 cm	L	18-20 cm	XL	22-24 cm
S	14-16 cm				

*All users must wear the brace. All users must wear the brace. All users must wear the brace.

Single patient use. If the product contains neoprene, use is recommended for maximum 4 hours or a day.

Recycling: If the product is not recyclable, it should be disposed of in a recycling bin.