QUICK START GUIDE

AUTOMATIC DIGITAL BLOOD PRESSURE MONITOR

REF MDS5001

Please watch the following User Video on your computer by typing in the link <https://www.medline.com/sku/item/MDPMD5001>

QUICK START GUIDE


AUTOMATIC DIGITAL BLOOD PRESSURE MONITOR

REF MDS5001

Please watch the following User Video on your computer by typing in the link <https://www.medline.com/sku/item/MDPMD5001>

Step 1 – Install Batteries

- A) Pull battery cover off.
- B) Install four (4) “AA” batteries.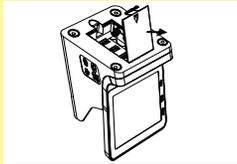


Step 2 – Set Date and Time

- A) Hold down SET until the year starts flashing in the top right corner.
- B) Release the SET button.
- C) Press MEM repeatedly to change the year.
- D) Press SET once to change to month selection. Press MEM repeatedly to change the month while the number is still flashing.
- E) Repeat step D three (3) times for the day, hour, and minute.
- F) Press SET once to change to Talking Language selection. Press MEM repeatedly to switch between SP OF (Off), SP 01 (English), and SP 02 (Spanish).
- G) Press START/STOP once.


Step 3 – Cuff Placement

- A) Place cuff on left arm at heart level (1 in. above elbow joint).
- B) Align tubing with middle finger to ensure placement of Artery Mark (Ø) over the brachial artery.
- C) Plug the air tube connector into the air tube socket on the left side of the monitor.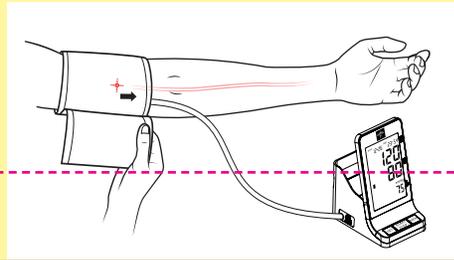


Step 4 – Body Placement

- A) Sit with feet flat on floor.
- B) Place arm, palm-up on flat surface.
- C) Ensure cuff is level with heart.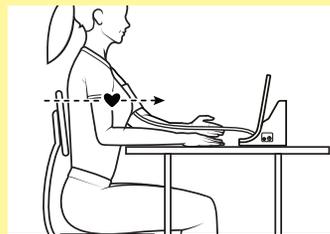

Step 5 - Begin Measurement

- A) Press START/STOP to turn monitor on and automatically begin measurement.

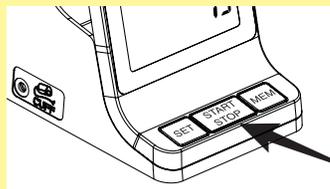Troubleshooting Guide

1) Does Not Turn On

- Check that batteries are inserted.
- Check that the battery polarities (+ and - symbols) are positioned in the correct direction.
- Try inserting new batteries.

2) Inaccurate Reading*

- Refrain from eating, smoking and drinking (especially alcoholic beverages) before a measurement. These activities can affect your blood pressure.
- Check the cuff for a snug fit that can fit two fingers between the cuff and your upper arm.
- Ensure that you are not moving, speaking or repositioning during measurement.
- If taking multiple measurements, ensure you are waiting at least 5 minutes between readings.

3) Inflation Problem

- Ensure the air tube connector is fully plugged into the air tube socket on the left side of the monitor.
- Ensure that cuff is snug enough to where it will not slide up or down the arm.
- Confirm that you are remaining still during measurements.
- Ensure the cuff is fully deflated before taking a measurement.
- Make sure cuff hose remains straight and unknicked during measurement.

*Other things can affect blood pressure values besides those listed here. **IMPORTANT!**

You can STOP the inflation or deflation of the cuff at any time by pressing the START/STOP button.

Please see User Manual for more details.

If you have any questions or are experiencing difficulties, please call the Medline Service Center at 1-800-MEDLINE (1-800-633-5463). V1 RJ20DGA

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