Auria™
iConnect
CI-5751

Instructions For Use
English, Français, Español, Deutsch, Nederlands, Italiano

Warnings and Cautions

1. Small Parts Hazard: The iConnect earhook, the battery door and the iConnect Battery may cause choking if swallowed.
2. Batteries are toxic when swallowed. Keep them out of reach of young children. If batteries are swallowed, please seek immediate medical attention.

Intended Use

The Auria iConnect is an adapter intended to allow a cable-free connection to a miniature personal FM receiver worn directly at ear level.

Package Contents

The Auria iConnect package comes with an iConnect earhook and two spare battery compartments.


Device Description

The Auria iConnect (CI-5751-XXX*) is available in two sizes and three colors.

CI-5751-100	<i>Auria iConnect Standard Silver Metallic</i>
CI-5751-120	<i>Auria iConnect Small Silver Metallic</i>
CI-5751-200	<i>Auria iConnect Standard Dark Sienna Metallic</i>
CI-5751-220	<i>Auria iConnect Small Dark Sienna Metallic</i>
CI-5751-300	<i>Auria iConnect Standard Beige</i>
CI-5751-320	<i>Auria iConnect Small Beige</i>

** The X's denote model number variants related to product color and size.*

Both sizes of iConnect contain a battery compartment, processor connector and personal FM connector. The standard iConnect has a standard earhook while the small iConnect earhook can be shaped for a custom fit around the ear by bending the earhook to the desired position.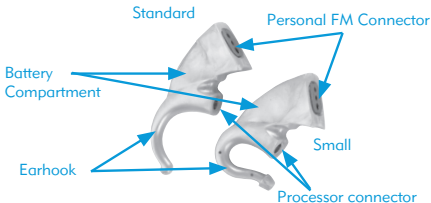

FM Receiver Compatibility

The Phonak MLxS and MLxi FM receivers are compatible with the iConnect. For easier access to the controls, the MLxS should have the pin orientation illustrated in Figure 1. Consult the MLxS user reference materials for instructions on how to orient the pins.

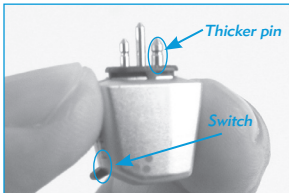
Figure 1. Proper pin orientation

iConnect Battery

The iConnect can only be used with a standard size 10 Battery. Do not use any other battery.

Using the Phonak MLxS or MLxi with the iConnect Earhook

To use the MLxS or MLxi receivers with the iConnect, follow these steps:

1. Attach the Phonak FM Receiver to the iConnect Adapter

First, turn off all equipment. If using the MLxi, ensure that it has been programmed for the Advanced Bionics Harmony/Auria sound processor. Contact your FM professional if you are uncertain whether your MLxi has been programmed for use with your processor. When attaching the receiver to the iConnect earhook, be sure to line up the thicker pin with the wider socket hole (Figure 2). Forcing the pins into the holes will result in damage to the unit.

Figure 2. Attach the receiver to the iConnect.

2. Attach the iConnect to the Auria or Harmony Processor

The iConnect can be used in place of other earhooks. To place the iConnect, align it with the processor and gently push straight on to the processor until it snaps into place.

3. Operate the FM System

Turn the volume control down on the Auria or Harmony processor and set the program switch to the dedicated FM program* position. Turn on the FM transmitter, then the FM receiver, then the processor. Use the 1-dot setting for the MLxS (Figure 3). Gradually increase the volume on the sound processor to the typical use setting.

Figure 3. Turn on the MLxS.

**In order to use the iConnect, the processor must contain a program set up for auxiliary input. A 50/50 or 30/70 Mic/Aux mix is recommended for use in the classroom. Switch to this program when using the processor with the iConnect. Please contact your center for fitting advice if necessary.*

4. Removing the iConnect from the Processor

To remove the iConnect, first turn off the FM receiver. Then, turn the earhook slightly more than a quarter turn in either direction until it pops off.

Replacing the Battery

To replace the battery, open the battery compartment (Figure 4) and remove the old battery by tilting the iConnect (Figure 5).

Figure 4. Open the battery compartment.

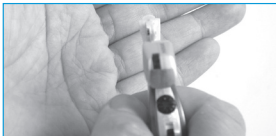
Figure 5. Remove old battery.

To open the small iConnect battery door, use a small, flat screwdriver (i.e. a hearing aid screwdriver) to lift the door.

Do not attempt to open the battery compartment further than shown on Figure 4.

Remove the protective foil from the new battery. Hold the battery as shown in Figure 6 and insert the battery, into the compartment bevel end (negative) first. Then gently shut the battery compartment until it clicks into place (Figure 6).

Do not use excessive force when handling the battery compartment.

Figure 6. Insert the battery.

If there is excessive resistance when closing the compartment, make sure that the battery is correctly inserted. The compartment will not close completely if the battery is improperly inserted.

Dispose of depleted batteries appropriately.

Replacing the Battery Compartment

When the battery compartment is subject to high force, it may fracture. A new battery compartment snaps on easily (Figure 7).

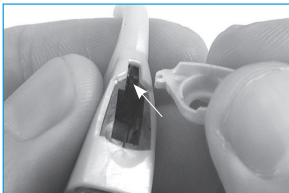
Figure 7. Install a new battery compartment.

Maintenance and Protection

To avoid unnecessary battery depletion* when the iConnect is not in use, remove the battery from the compartment and store it in a safe place.

** After the protective foil has been removed, Zinc air batteries begin to drain even when not in use in a device. To confirm battery life, use a hearing aid battery tester. Consult your hearing professional.*

The Auria Carrying Case insert functions as a dri-aid kit. Put the iConnect in the Auria Carrying Case regularly to help prevent moisture build up.

Avoid the following:

- Exposing the iConnect to extreme temperatures (below 32 °F/ 0 °C or above 122 °F/ 50 °C). For example, do not leave in a closed car on a hot day or near a heater or radiator.
- Dropping the iConnect on a hard surface.
- Opening the battery compartment further than shown in Figure 4.

Cleaning

If necessary, the iConnect can be cleaned with a slightly dampened cloth or tissue. Do not use soap or solvents.

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©2022 Advanced Bionics LLC. All Rights Reserved.
Advanced Bionics LLC
Advanced Bionics GmbH