

For further assistance, please
visit [Tenovi.com/support](https://tenovi.com/support).

tenovi

© 2025 Tenovi Co.
All rights reserved. Republication, duplication or modification
in whole or in part is prohibited without prior written permission
from Tenovi. All trademarks are owned by or licensed to
Tenovi, Co. unless otherwise noted.
Revision 05.01.2024

1

Power Your Gateway


Plug your Gateway into a power outlet with the provided power cord and wait until the LED on the Gateway turns **RED**.



2

Insert Batteries

Insert batteries into the back of the Peak Flow Meter.

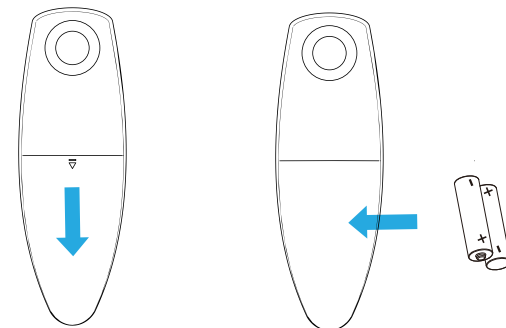

Connect To Gateway

3


Stand within 20 feet of the Tenovi Gateway, press the power button for 3 seconds and wait for the Tenovi Gateway to start flashing yellow.



Displaying Measurement

5


Your measurement will be displayed on the Peak Flow Meter screen and the Gateway LED should turn green. Press the power button and start at Step 3 if you need to take another reading.

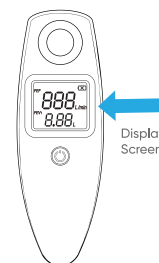

Take A Measurement

4

Take a deep breath and then exhale quickly into the Tenovi Peak Flow Meter.

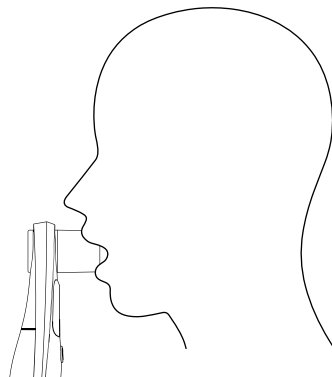

Daily Measurements

6

Take one measurement each day unless advised otherwise by your healthcare provider. The LED will turn RED each morning and then turn GREEN once a measurement has been successfully transmitted.



Measurement Needed



Measurement Successful