



1) Download Corsano App with QR Code

For patient if you have received an invitation letter with QR Code

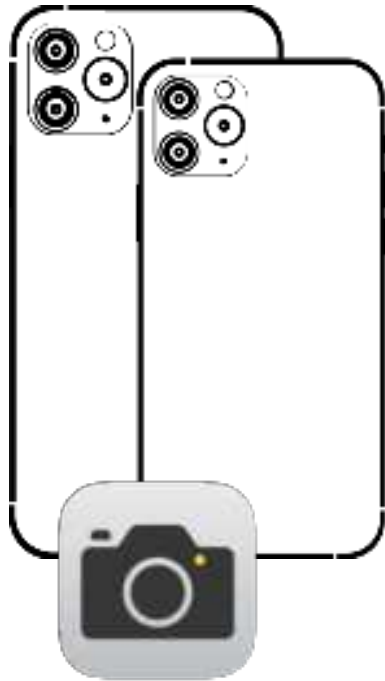

1) Open the
Camera App on
your mobile

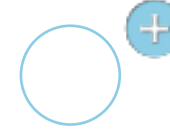

2) Take your letter
and scan the QR-
Code



3) The App Store will
automatically open.
Click to install Corsano
App




4) App ready to use

2) Registration with QR Code



For patient if you have received an invitation letter with QR Code

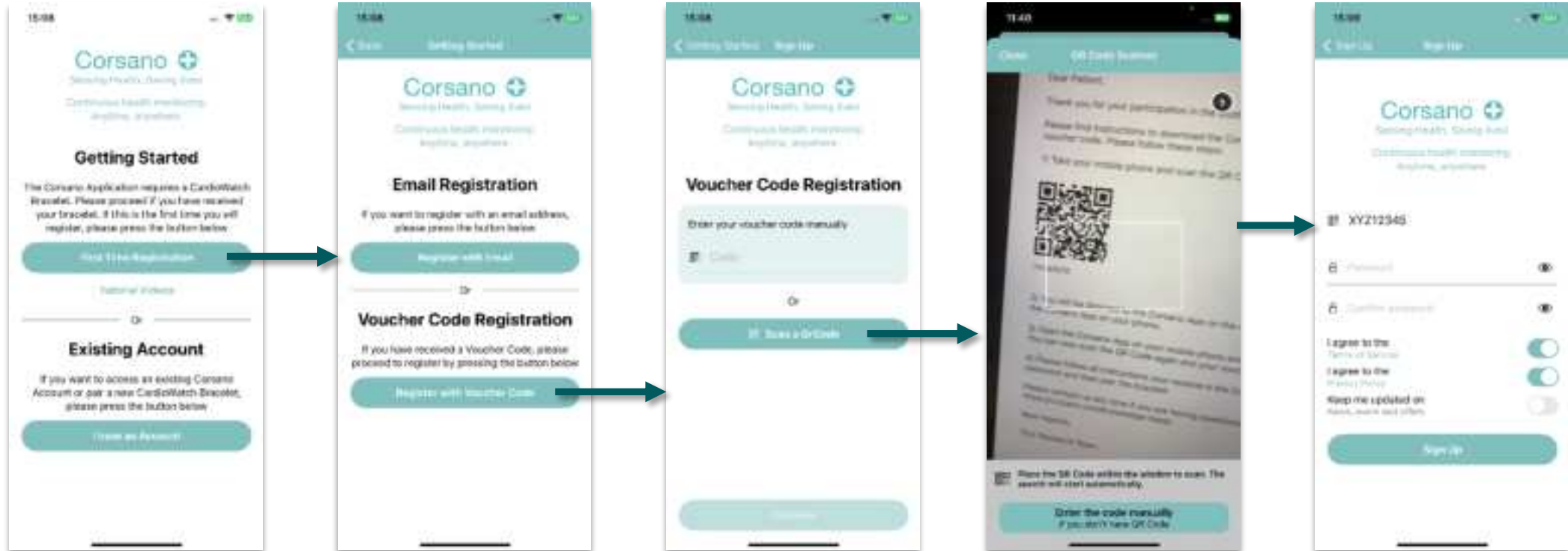

1) Click on
[First Time Registration]

2) Select the
option
[Register with
Voucher Code]

3) Click on
[Scan a QR Code]
to open the
camera

4) Scan the QR
Code with the
camera

5) Voucher code detected.
Set your own password to
finish the registration
process




Store password in
safe place!


3) Access Existing Account



For patient if you already have an account

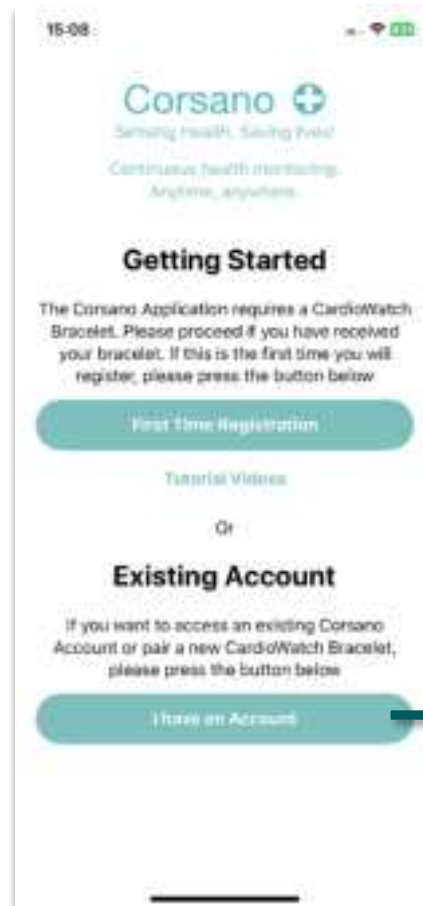

1) Press on
[I have an Account]

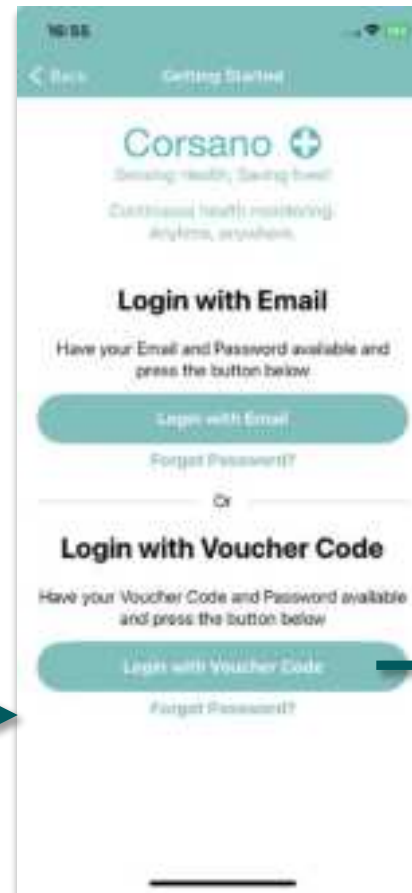

2) Select [Login with
Voucher]



3) Enter your voucher
code and your password