

# Phonak

# Target 10

## Phonak Target/ALPS User Guide



This guide contains details on using Phonak Target/ALPS available in Phonak Target fitting software. Phonak Target/ALPS allows you to enter data via Phonak Target directly into ALPS. This one step process avoids duplicate entry and allows you to manage all relevant Lyric fitting and subscription information in an easy way.

To use Phonak Target/ALPS effectively, make sure your computer has an internet connection.

For every Lyric fitting, Phonak Target will automatically connect you to ALPS. A wizard will guide you through all the necessary steps to complete a successful Lyric fitting. Please note that the Lyric device activation code must be entered the first time Phonak Target is launched.

More information is also be available under Web Help.

### Content

Create a new client in Phonak Target .....	2
Access an existing client in your ALPS account.....	6
Device exchange (including ear canal observation) .....	7
Tourist case: Device exchange for a tourist client .....	10
Tourist case: Your client has been refit by a different Authorized Lyric Partner .....	11
Auto Renewal of a subscription.....	12
Cancellation of an active subscription .....	13
Offline fitting and synchronization.....	14
ALPS home and reports .....	16
Enhanced log files for support .....	20
Information and description of symbols and System Requirements.....	20

## Create a new client in Phonak Target

To create a new client in Phonak Target:

1. Click on [Instruments] and select Lyric devices in [Hearing aids]
2. Click on [Lyric configuration] and click on [Select device] to open the Phonak Target/ALPS wizard.



The ALPS button indicates your current connection status:

- Green – connected
- Black – disconnected




To connect into ALPS, enter your user name and password to log into your Lyric account.

We strongly recommend that you select the [Automatic login] function. This time-saving feature enables you to remain logged into ALPS without the need to reenter your credentials. That only applies if you're not working on a shared workstation.



In case you have forgotten your password, click on the link [forgot password] and change it in [www.phonakpro.com](http://www.phonakpro.com) directly.

You will be asked to [Add new client to your ALPS account]. Your account information is shown on this screen, that is very helpful for providers working with multiple logins.

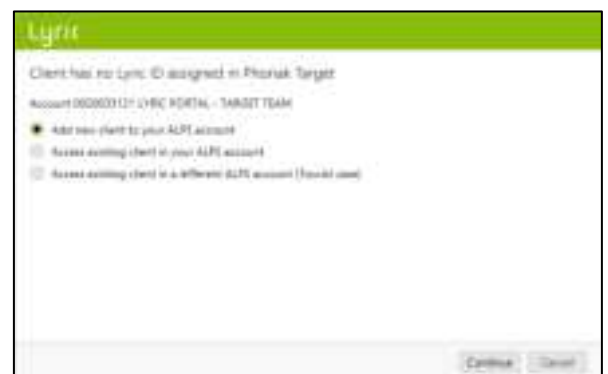

In case you don't intend to login with the last login shown, choose [Switch login] under the ALPS button and login with another username.



You can now see the client's first and last name, date of birth and gender as entered in Phonak Target.

The additional information (date of birth and gender) is optional. You can uncheck [Submit additional information] if you do not wish to transmit this data.

For new subscriptions you are then asked to confirm that the client understands and gave consent that his/her data will be transferred to Sonova for processing. If the box is not ticked, the fitting cannot continue.

To select the appropriate Lyric device, choose the measured seal size and pick the first device in the list as it is the closest to expire. Devices about to expire within 30 days are marked red.

Please note that expired devices can no longer be fit. To return an expired device, use the [device return process] in ALPS, select the serial number and the reason for device return as "expired" and send it back to Phonak.

Select the measured insertion depth.

Choose the subscription duration you want to create for your client. Make sure to click the check box, in case the customer is using Lyric for the treatment of tinnitus.

Lyric

Enter client details

First name:

Last name:

Date of birth:

Gender:

☒ Submit additional information

☐ I hereby confirm that I fully informed my client that his personal data will be transferred to the hearing aid manufacturer for processing purposes.  
My client consented to such transfer

Lyric

Select Lyric device from consignment stock

Formal device seal size: 40L

Expiry date: 14.03.2023

Device#	Model	Seal size (mm)	Expiry
2107447341	Lyric4	40L	20.04.2023
2107447342	Lyric4	40L	04.04.2023
2107447343	Lyric4	40L	04.04.2023
2107447344	Lyric4	40L	04.04.2023
2107447345	Lyric4	40L	11.04.2023
2107447346	Lyric4	40L	17.02.2024
2107447347	Lyric4	40L	17.02.2024
2107447348	Lyric4	40L	17.02.2024
2107447349	Lyric4	40L	17.02.2024

Lyric

Select measured insertion depth

Initial device insertion for this ear:

1 mm  
4 mm  
5 mm  
6 mm  
7 mm  
8 mm  
9 mm  
10 mm  
11 mm  
12 mm  
13 mm  
14 mm  
15 mm

Lyric

Select a subscription

Subscription type:

Payment type:

Start date:

☐ Lyric is used for tinnitus treatment

Phonak Target confirms the device selection. You can get [More information] about the subscription in this window or choose [Close]. For binaural fittings, you are asked to [Proceed with other side] which leads you directly to selecting another device from your consignment stock.




Below the start and end date of the subscription and actual days remaining to the trial end date are shown.

It is now possible to extend a trial once for an amount of 10 days. The new end date will then be shown in Target/ALPS.



In case a wrong serial number has been chosen, it can be returned to the stock by clicking "return device to stock".

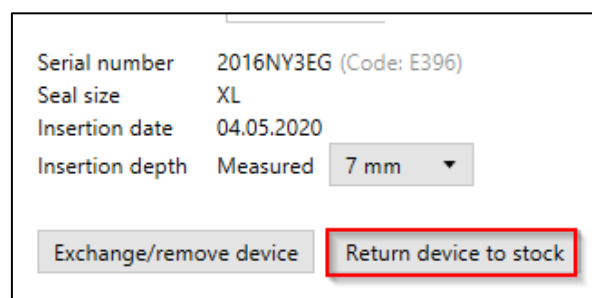

You have the possibility to choose between Gain (Real ear and Coupler 0.4cc) and Output display (SPL real ear and coupler 0.4cc) when fitting Lyric. Click on the [Output] menu to select your desired output curve display. When selecting the output curve display, you may also choose to show or hide the hearing level and UCL.



In case the achieved insertion depth varies from the measured one, a measured and an achieved insertion depth can be entered in [Fitting]. A warning will be shown if the achieved insertion depth is higher than the measured one.



By clicking [Save to database] at the end of the session, your fitting data will be synchronized with ALPS.




In case you need to have more notes stored around the patient, fill them in on the new notes section.



A new note can be added and Target allows a "remind me again in next session" function.




All note are then visible under [Client, Notes].

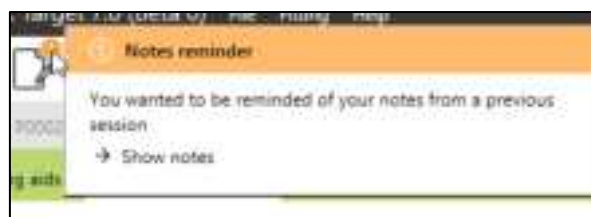

The following reports can be printed from the Print menu:

For Professional:

- Fitting Report
- Lyric Datasheet

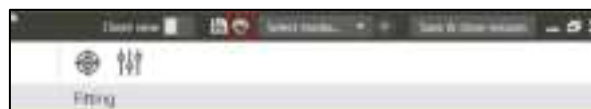

For Client:

- Hearing Aid Instructions (SoundLync)
- Communication tips
- Diary of observation
- Understanding hearing loss



The curves reports can be printed as follows:

Choose [Setup], [Reports], [Curve type] and choose between Gain or Output. The selected curve type will then be visible on the curve report.

## Access an existing client in your ALPS account

If the client exists in ALPS but not in Target (or in case of a first access within a satellite office), you can [Access existing client in your ALPS account].

First, create a new client in Noah and enter the client's data. This will be transferred into Phonak Target. Proceed until you reach the screen where you are asked to add or access an existing client.



Enter the correct Lyric ID and/or the first and last names as stored in ALPS.

To retrieve the Lyric ID you can open the ALPS portal in a separate browser or get it from the printed Lyric ID card. Hit search to find P# (Lyric ID).



The Lyric ID has to be entered only if you are connecting an existing client for the first time. When opening the session again, you will automatically be connected to the ALPS data.

By selecting the search button the P# will populate, if the P# does not populate, this means the patient is not an existing patient or the name does not match the ALPS information.

If the data of the client was found, the client subscription information screen will show you all the relevant information.




Information	Right	Left
Subscription	Yes (with icon)	No subscription
Start date	06.06.2019	
End date	06.06.2020	
Remaining devices	11	
Device	lyric 5	
Serial number	1400000101	

'Continue' and 'Cancel' buttons are at the bottom right.

## Device exchange (including ear canal observation)

To exchange a Lyric device, please log into Phonak Target/ALPS, go to [Instruments], [Lyric configuration] and click [Exchange/remove device].

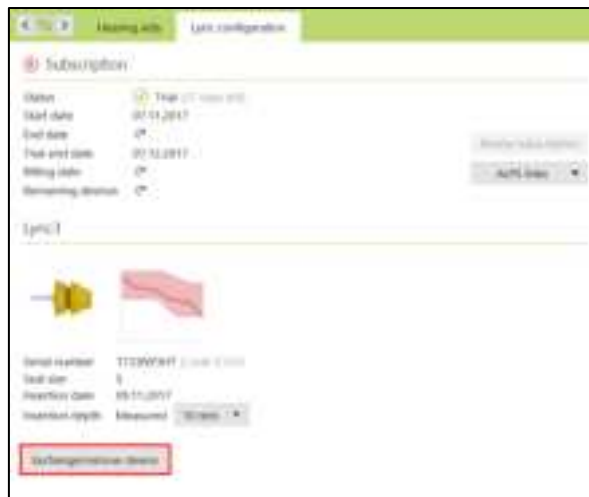

Select if the provider or the patient has removed the device and specify the removal date if not today. Choose the corresponding removal reason. The removal reasons are separated into four categories (Discomfort, Device related, Fitting related, Proactive removal, Self-replacement and Cancelled Subscription).



Hovering over individual removal reasons activates the "tooltips", which gives you more information to help you understand which reason to select.



Phonak Target will ask you for the status of the ear canal. You can choose between: [OK for immediate refit], [OK for refit after rest] and [Requires medical referral]. When choosing [OK for immediate refit], the device exchange can happen immediately. In case the ear canal was irritated and needed some rest or a medical referral, the system will inform you accordingly.\*

The comments for removal reason and ear canal observation are both optional.

In case [OK for refit after rest] or [Requires medical referral] is chosen, additional details about the state of the ear canal must be chosen. Additionally, if "other reason" is chosen, a comment to give more explanation is required.

To remove a second device, click on [Proceed with other side]. There are some reasons (scheduled removal, near end of device life and clinical proactive removal) that will give the second side the same reason as a default but it can be changed if the removal reason is different.

\*If you have chosen [OK for refit after rest] or [Requires medical referral], Phonak Target will inform you on the main screen.

Select device

1

OK for refit after rest (13.11.2017)  
Redness of tissue - Mild



Before you can exchange a device, it is mandatory to confirm that the [Ear is ok for immediate fit].

After confirming the ear is OK for immediate fit, choose another Lyric device to exchange.

Serial	Model	Stock size	Expiry date
170000000	Lyric 2	M	08.01.2016
170000001	Lyric 2	M	08.01.2016
170000002	Lyric 2	M	08.01.2016
170000003	Lyric 2	M	08.01.2016
170000004	Lyric 2	M	08.01.2016
170000005	Lyric 2	M	08.01.2016
170000006	Lyric 2	M	08.01.2016

The comments you entered are saved under [Client, Notes].

Please note that if the device has been worn longer than 30 days, the [Easy Refit] is enabled and will show default values:

- Removed by provider
- Removed today
- Removal reason: scheduled removal
- Ear canal observation: "ok for immediate refit"

Click [Remove] and then choose a new device accordingly. In case the criteria doesn't match the default value, please change it and continue.

## Tourist case: Device exchange for a tourist client

If you need to exchange a Lyric device for a client who was originally fit by another Authorized Lyric Partner, you need to get access to the fitting data of that client's ALPS fitting data. To do so you need the client's individual Lyric ID.

To start the tourist case, proceed as you would with a new client. Log into your ALPS account and select [Access existing client in a different ALPS account] (Tourist case)].

Enter the client's Lyric ID, and check if the name is displayed correctly (as printed on the Lyric ID card). Proceed with the exchange of the device, and adjust the fitting parameters if necessary. If the patient doesn't know their Lyric ID, it can be obtained by calling Lyric customer service.



The screenshot shows a web form titled "Lyric client" with a green header. The form contains the following fields: "First name" (with a dropdown menu), "Last name" (with a dropdown menu), "Lyric ID" (with a dropdown menu), "Date of birth" (with a date picker), and "Gender" (with radio buttons for "Male" and "Female"). Below these fields is a message: "Please contact Lyric customer support if the client's Lyric ID is unknown." At the bottom right are "Continue" and "Cancel" buttons.



The screenshot shows a web screen titled "Lyric" with a green header. The main text reads: "Client has no Lyric ID assigned in Phonak Target". Below this is a section titled "Access EXISTING LYRIC PORTAL - TARGET TEAM" with four radio button options: "Add new client to your ALPS account", "Access existing client in your ALPS account", "Access existing client in a different ALPS account (Tourist case)", and "Access data in Phonak Target will be updated from ALPS". The third option is selected. At the bottom right are "Continue" and "Cancel" buttons.

## Tourist case: Your client has been refit by a different Authorized Lyric Partner

If your client has been refit by a different Lyric Provider, you will be alerted that there is a mismatch between ALPS (tourist fitter) and Phonak Target (your fitting). This indicates that the serial number has been changed.

By clicking [Continue] the system will synchronize the newest serial number.

The screenshot shows the Lyric software interface with a green header. Below the header, an orange banner reads "Data conflict between ALPS and Phonak Target". The main area displays a comparison of data between ALPS and Phonak Target. At the top, it says "Device data in ALPS and Phonak Target do not match!". Below this, there are two columns: "Right" (ALPS) and "Left" (Phonak Target). The "Right" column shows "Subscription: Trial (30 days left)", "ALPS: 140899010 (Tourist refit)", and "Target: No serial number". The "Left" column shows "Subscription: Trial (30 days left)", "No serial number", and "No serial number". At the bottom, there is a message: "Serial number will be updated in Phonak Target" and "No action required". A "Continue" button is at the bottom right.

If the tourist Lyric fitter also changed the fitting parameters, a fitting parameter mismatch will be shown. You can solve the conflict by choosing the preferred option.

Please note that the ALPS data is the most up to date information from the tourist fitter and Phonak Target data is the data you entered yourself before the tourist fitting.

The screenshot shows the Lyric software interface with a green header. Below the header, an orange banner reads "Fitting parameter mismatch". The main area displays a comparison of fitting parameters between ALPS and Phonak Target. At the top, it says "Fitting parameters stored in ALPS and Phonak Target do not match!". Below this, there are two columns: "Right" (ALPS) and "Left" (Phonak Target). The "Right" column shows "New Phonak Target data: Last updated 18.08.2014 14:07" and "New ALPS data (refit): Last updated 18.08.2014 14:07". The "Left" column shows "Fitting parameters are identical". At the bottom, there is a message: "Clicking 'Continue' will resolve the conflict. Press 'Cancel' to resolve the conflict manually." A "Continue" button is at the bottom right.



## Auto Renewal of a subscription

Two months before the subscription expires, you'll get a renewal pop-up in Target/ALPS, that informs you when the subscription is up for automatic renewal.

Once the subscription is up for renewal, it will automatically be renewed for another year with the same payment method as chosen in the active subscription.


In case that is not wished by the customer, you still have the following choices before the subscription is automatically renewed:

1. Renew the subscription and choose another length of subscriptions (where applicable)
2. Cancel the renewal of the upcoming subscription

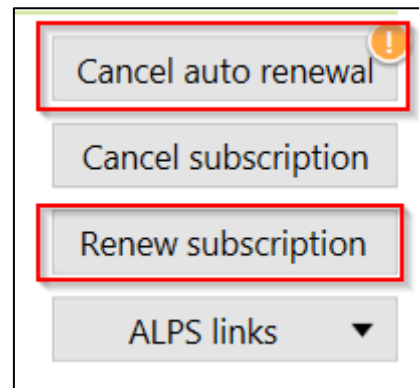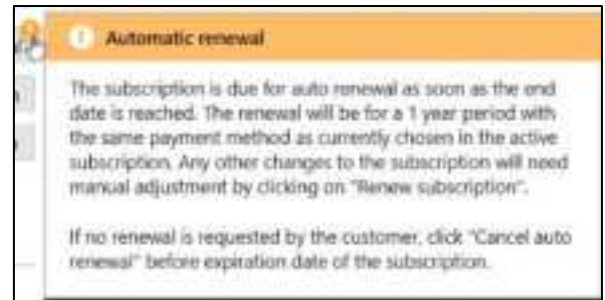

In case the customer doesn't want to renew the subscription any more, then it is mandatory to enter a non-renewal reason that is given when entering a cancellation of the upcoming subscription.



The renewed subscription will automatically be an active subscription without a trial.

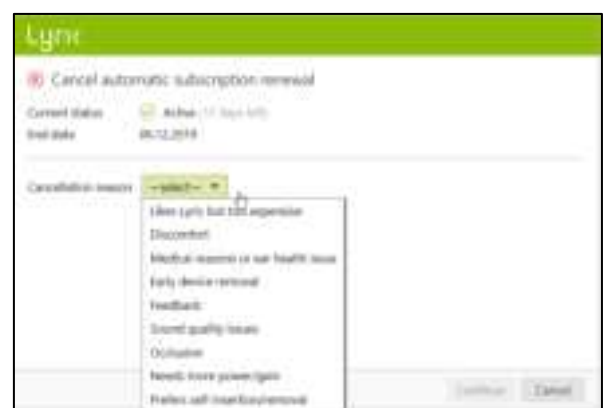

## Cancellation of an active subscription

If there is a need to cancel an active subscription, you can do so by clicking [Cancel subscription] on the Lyric configuration screen.

The system will make you aware that you are about to cancel an active subscription. To proceed, choose the corresponding cancellation reason and click Continue. The cancellation request will be sent to Phonak.

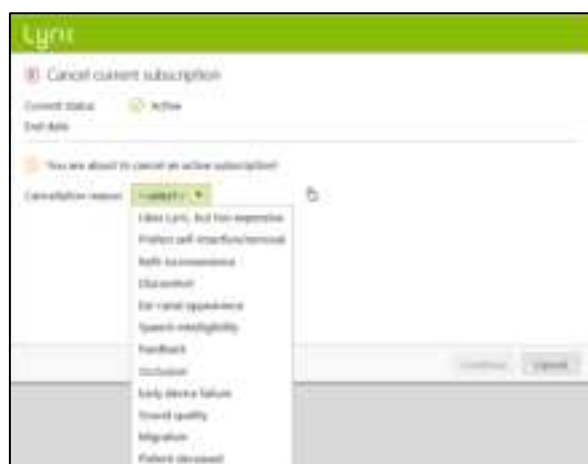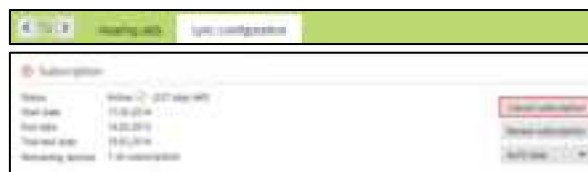

## Offline fitting and synchronization

There might be occasions where you have a client set up via Phonak Target/ALPS, however, due to various reasons (such as temporarily no internet connection or computer problems) you will have one or more sessions where you have to work offline. This means that the new data you have in Phonak Target is not synchronized with ALPS.



Removing a device in offline mode works the same way as in removing a device in online mode.

You are now asked to select a new device.




Enter the serial number as well as the code that is shown on the Lyric package. Enter the measured insertion depth. If needed, the fitting parameters can be changed.



Once the internet connection problem is resolved and you are able to use Phonak Target/ALPS again, you will be notified by an orange exclamation point [Fitting not saved to ALPS] message for each patient that was fit offline.



As soon as Phonak Target has started connecting into ALPS again, it will automatically synchronize the offline sessions.

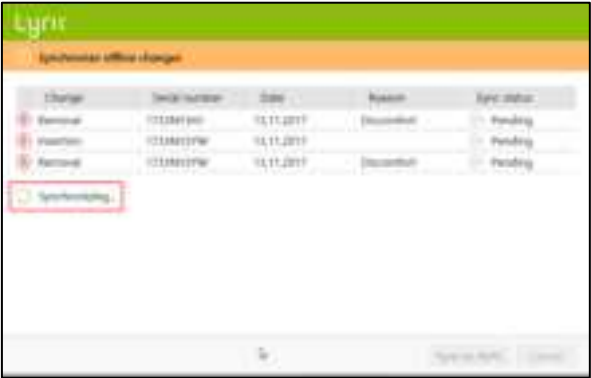

A green check mark will indicate that the synchronization has been successful.

After an offline fitting, logs on your desktop are available.




## ALPS home and reports

There are some actions that require a direct link to ALPS:

- Open ALPS
- Client details
- Device details
- Print Lyric ID card
- Reports

Alternatively, ALPS can also be opened via Phonakpro.com.

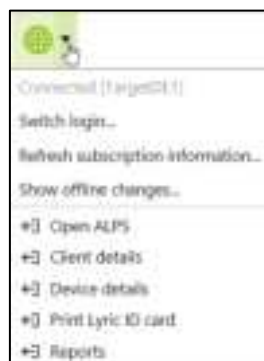

On the [Home Screen] you have an overview over:

- Inventory Audit
- Trials about to expire within 7 days
- Subscriptions about to expire within 90 days
- Devices to expire within 120 days





The section [Client] does give you an overview over all your clients. By clicking on the first name (marked in blue), you'll get to a level where you see:

- Print Lyric ID
- Subscription of that client
- Renew Subscription(s)
- Remove Devices (needed when cancelling the subscription)
- Client details (such as fitting parameter)

This screenshot shows a table with columns for Client ID, Name, Address, Phone, and Email. The first row is highlighted in blue, indicating it is the selected client.

Client ID	Name	Address	Phone	Email
1000000001	John Doe	123 Main St	555-1234	john.doe@example.com
1000000002	Jane Smith	456 Oak St	555-5678	jane.smith@example.com
1000000003	Bob Johnson	789 Pine St	555-9012	bob.johnson@example.com

By clicking on the blue subscription number, more details about the subscription are shown:

- Terms and Conditions
- Cancel Subscription
- Device History

This screenshot shows a form with fields for Subscription ID, Start Date, End Date, and Status. There are also buttons for 'View Terms', 'Cancel', and 'Renew'.

Subscription ID	Start Date	End Date	Status
1000000001	2023-01-01	2023-12-31	Active

When clicking on the blue device number, this information is available:

- Device State
- Device Code, Warranty & Use By Date
- Subscription/Insertion
- Removal and Return Information

This screenshot shows a form with fields for Device ID, Device Code, Warranty, and Use By Date. There are also buttons for 'View State', 'View Code', 'View Warranty', and 'View Use By Date'.


Device ID	Device Code	Warranty	Use By Date
1000000001	12345678	2023-01-01	2023-12-31

The [Stock] overview shows all devices in the respective account, sorted by device size. That list can also be exported to Excel.

This screenshot shows a table with columns for Device ID, Device Code, Device Size, and Device Status. The table is sorted by device size.

Device ID	Device Code	Device Size	Device Status
1000000001	12345678	XS	Active
1000000002	23456789	S	Active
1000000003	34567890	M	Active
1000000004	45678901	L	Active

Under [Subscriptions], all subscriptions with all states can be found. When working in a satellite office that is connected in the system, then all clients are visible but separated by account number. That list can be exported to Excel.

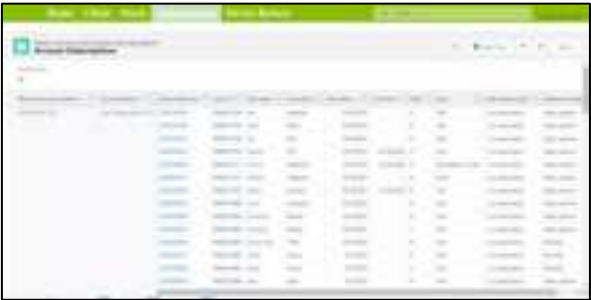

The [Device Return] section allows you to return devices to Phonak that are either

- New (overstock)
- Dead on arrival
- Wrong deliver
- Expired

Choose the devices that need to be returned to Phonak. Click on the respective device and choose a return reason. Click select and then print a return form that can be sent with a device to return (if needed).



Under your name, there are the following sections:

- Home
- My Profile
- My Account
- Logout



[My Profile] does allow you to change your language if needed.




Under [My Account] you can choose your current location (when working in different locations).



In case you'd like to get a weekly e-mail about the trials, subscriptions and stock about to expire, you can fill in up to five e-mail addresses. Also, choose how many months before expiration of the subscriptions you'd like to get a notification. For satellite offices, a report with a full overview over all offices can also be requested by ticking the box.



The [Dashboard] gives you an indication about how your business is doing. Explanation is found in more details in the legend within the dashboard.

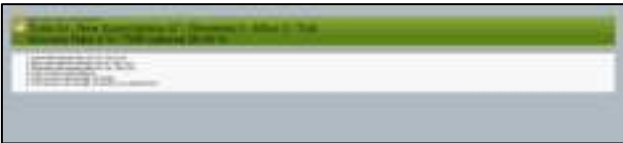

The [Device Analysis Form] and the end-user contract templates can be downloaded.




The [Search] field does allow you to search data such as subscription, client names, devices etc. directly.



## Enhanced log files for support

If you encounter a problem with Phonak Target, enhanced logging can help our Target software support team to find out the root cause.

Please save the log files on your desktop and/or send them by e-mail to your Lyric contact.

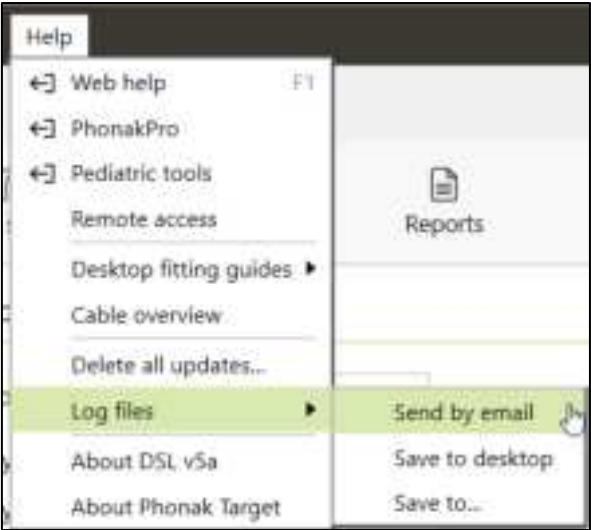


## Information and description of symbols and System Requirements

Information and the description of symbols and an overview of system requirements can be found in the Phonak Target User Guide

This user guide is applicable for Target 10.0 and later subversions of Target 10.0 fitting software. For previous versions of the fitting software user guide contact your local Phonak representative.



Sonova AG • Laubisrütistrasse 28 •  
CH-8712 Stäfa • Switzerland

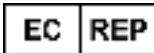

Sonova Deutschland GmbH  
Max-Eyth-Str. 20  
70736 Fellbach-Oeffingen • Germany

