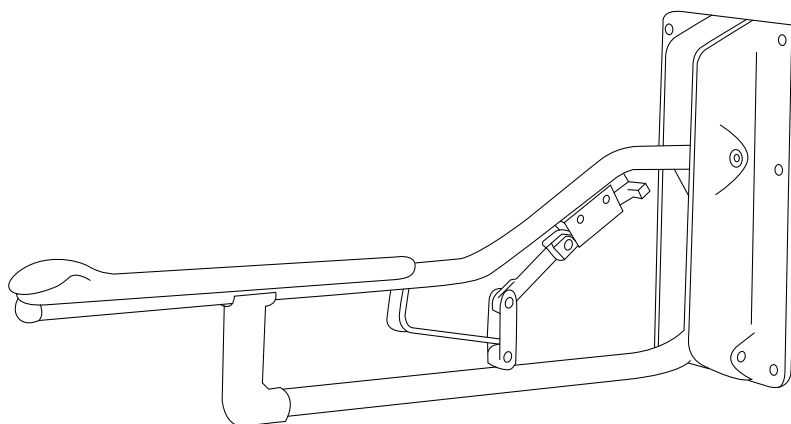
Etac Optima

låsbart
lockable
verriegelbar

Manual

78186E 19-10-29

etac®


A

Alla mått i mm

All measurements in mm

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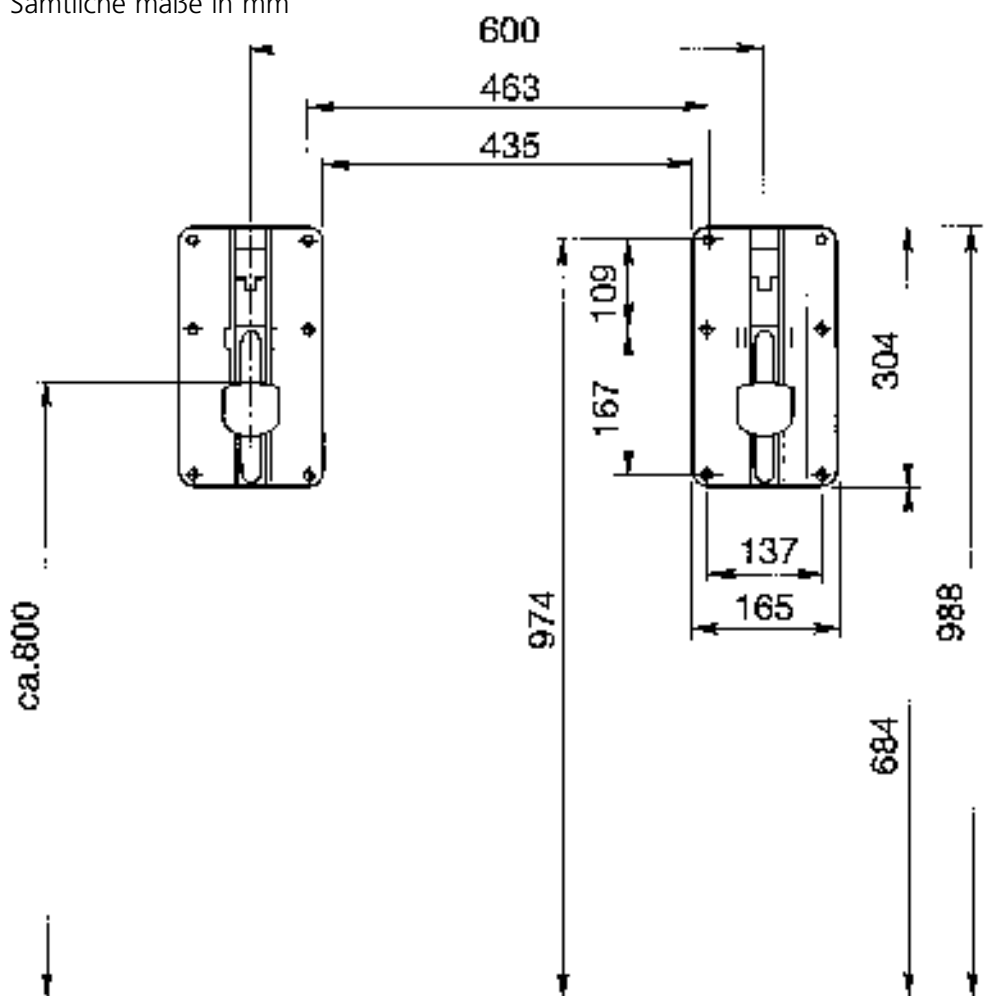B**C**

80303004	71,5 cm	21,5 cm	2,4 kg
80303007	91,5 cm	21,5 cm	2,8 kg

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Tekniske data..... Figur C

en**English****Intended use**

Optima is a range of toilet armrests that facilitates sitting down and getting up from the toilet. The armrests provide greater security when seated and support when you sit down/get up from the toilet. The armrests can be flipped up.

The product conforms to the requirements of the Medical Devices Regulation (EU) 2017/745.

Installation, use..... Figure A
Maintenance and safety..... Figure B

Max user weight: 130 kg.

Cleaning: Clean the product using a solvent-free cleaning agent with a pH value of 5-9 or with a 70% disinfectant solution.

Decontamination: Can be decontaminated at max. 85° C/185 °F for 3 minutes if required.

The materials used are corrosion-resistant.

The materials used are resistant to all general disinfectants.

A defective product must not be used.

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Guarantee, service life

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Service life 10 years. For complete information regarding the service life of the product, see www.etac.com.

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Etac A/S
R82, Inc.
Etac Supply Center AB