RAKIIIE

ANKLE SUPPORT BRACE

User Manual

Meet Rakiie

We're always at your service for your satisfaction! We're all about safe sports and active living. Rakiie braces are here to prevent injuries and enhance your performance. Enjoy your activities more confidently with Rakiie! Beyond products, we're committed to your joy and well-being.

Contact Us Anytime, Anywhere

Visit www.rakiie.com for 24/7 customer service, or email us at support@rakiie.com. This product comes with a 12-month warranty. If there's anything missing, not fitting, or not meeting your expectations, contact us for an exchange or refund.

24/7 Live Chat: www.rakiie.com
E-mail: support@rakiie.com
Guarantee: 12-month warranty

Product Diagram:

Upgraded Spring Sheet

Thickened metal on both sides ensures enhanced durability and stronger support.

Elastic Heel Design

Contours perfectly to the heel curve for a snug and comfortable fit.

Split 3D Silicone Pads

Dual-sided silicone pads gently massage and relieve pain, offering superior comfort.


Heel Support

Crafted from CR material to reduce strain on the heel tendon effectively.

Upgraded Velcro Strap



Made from premium watch-strap material for superior strength and comfort.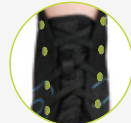

8-Hole Tie-Down Design

Allows freely adjustable tension for a personalized fit.

Wearing Instructions:

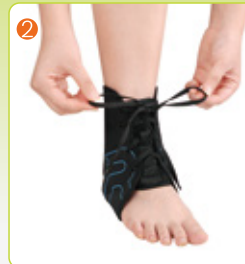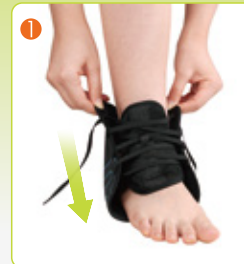Usage Instructions:

Please read these instructions carefully before use.

- 1 Loosen the laces and slide your foot into the brace.
- 2 Tighten the laces on both sides and secure.
- 3 Wrap the velcro strap across and under the foot.
- 4 Pull it diagonally to the top of the other side.
- 5 Then wrap and secure the velcro strap.

Note:

- 1 For prolonged use, please re-wear periodically to prevent skin issues or circulation problems caused by prolonged pressure.
- 2 Check the directions before use and follow the wearing instructions correctly.
- 3 This product is not intended to be used as a medical device.

Washing Instructions:

- 1 Hand wash only, avoid using bleach, and use a neutral detergent.
- 2 Do not wash with light-colored clothes, as the color may fade or transfer to other fabrics.
- 3 Wash separately from other clothes to prevent wear or fasteners opening.
- 4 Avoid using a dryer or iron, hang to dry in a cool place.

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CET Product Service Ltd.